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# Wi.Neb Go

Mod. P0905EM

## Portable and Ultra-Compact Aerosol Therapy Device

Congratulations on your purchase of Up-Wi.Neb Go and thank you for confidence in us. Our goal is to fully satisfy our customers by offering them cutting-edge systems for treating ailments of the respiratory system. Please read these instructions carefully and keep them handy for future reference. Use the device only as described in this manual. This is a home medical device designed to administer nebulizer medications prescribed or recommended by your physician upon assessing the patient's general conditions. Please visit [www.flaemnuova.it](http://www.flaemnuova.it) to see the entire range of Flaem products.

### IMPORTANT SAFEGUARDS


- This device is also intended for direct use by the patient
- Before using the device for the first time, and periodically throughout its life, check the power supply adaptor cable for any damage; if you detect any damage, do not plug it in. Take the device immediately to an authorized FLAEM service center or to your local dealer.
- Should your device fail to provide the expected performance, contact the authorised service centre for clarifications.
- The average duration provided for compressor is: 400 hours.
- Children and persons with reduced physical, sensory, or mental capabilities may only use this device under the strict supervision of an adult who has read this manual.
- Several parts of the device are small enough to be swallowed by children; therefore keep the device out of reach of children.
- Do not use the tubes and the cords of the device for purposes other than those specified as they may cause strangulation hazard; utmost attention must be paid to children and persons with special needs; usually, such persons are not capable of assessing the hazard correctly.
- This device is not suited for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrogen protoxide.
- The power adaptor of your Wi-Neb can be plugged into different types of wall outlets worldwide by using the appropriate adaptors. Always keep the power adapter and the power supply cord away from hot surfaces.
- Do not handle the power adaptor with wet hands. Do not use the device in humid environments (e.g. while taking a bath or shower). Do not immerse the device in water; should this accidentally occur, unplug the device immediately. Do not remove or touch the device immersed in water before pulling out the plug. Take the device immediately to an authorized FLAEM service center or to your local dealer.
- Do not wash the device under running water or submerge in water. Keep away from splashing water or other liquids. Do not charge the batteries outdoors.
- Do not expose the device to high temperatures.
- Do not put the device and power adaptor near heat sources, direct sunlight, or in hot environments.
- Do not obstruct or insert any objects in the filter or in its housing in the device.
- Repairs must be performed by authorized FLAEM personnel only, by complying with the information provided by the manufacturer.  
Any unauthorized repairs will void the warranty and may pose a safety hazard for the user.
- The Manufacturer, the Retailer and the Importer are responsible for the safety, reliability and performance of the device only if:
  - a) the device is used in accordance with the operating instructions
  - b) the electrical system of the premises where the device is used is in compliance with current laws.
- Interactions: The materials used that come into contact with medications have been tested with a wide range of medications. However, due to the variety and continuous evolution of medication, the possibility of a chemical interaction cannot be excluded. We recommend that once the medication has been opened you should use it as soon as possible and avoid prolonged contact of the medication with the nebulizer. The materials used in the device are biocompatible in accordance with the provisions of Directive 93/42 EC and subsequent amendments. However, the possibility of occurrence of allergic reactions cannot be entirely excluded.
- Do not use the device with the power adaptor connected only to the device and not plugged into the mains socket.
- The manufacturer must be contacted about any problems and/or unexpected events concerning operation and for any clarifications on use, maintenance/cleaning.
- The amount of time required between preservation conditions and use is approximately 2 hours.

**FOR IMPORTANT WARNINGS ON THE BATTERY SEE  
"DESCRIPTION AND OPERATION WITH BATTERY"**


## OPERATION WITH EXTERNAL POWER SUPPLY

### OPERATION WITH POWER ADAPTOR

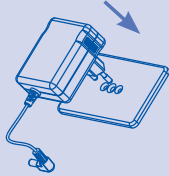
Plug the power adaptor into mains socket that is compatible with the device voltage. After using the device unplug it; the socket must be placed so as to allow for convenient disconnection from the power mains.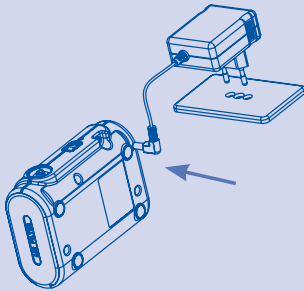


Insert the "L" jack of the power adaptor cable in the DC plug of the device.

### OPERATION WITH CIGARETTE LIGHTER RECEPTACLE

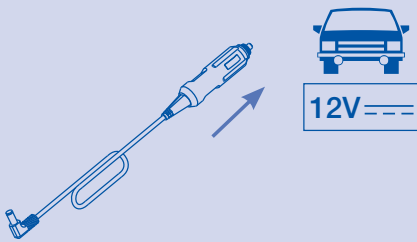Insert the cigarette lighter plug in the 12V DC cigarette lighter receptacle of the vehicle/ camper/ boat. Check the User Manual of the vehicle to make sure that the voltage supplied is 12V DC.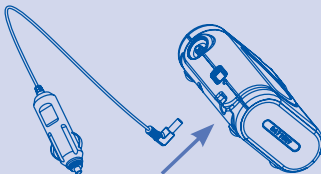


Insert the "L" jack of the cigarette lighter cord in the DC plug of the device.

## HOW TO CONNECT THE ACCESSORIES

Prior to each use, carefully wash your hands and clean your Wi Neb according to the instructions provided in the “CLEANING, SANITIZATION AND DISINFECTION” section. During the application you should use adequate protection against any drips. The average lifespan of accessories is 1 year; however, we recommend you replace the nebulizer every 6 months in intensive use (or earlier, if the nebulizer is blocked) to always ensure maximum therapeutic effect. The nebulizer and accessories are for personal use only to prevent possible risk of infection due to contamination. This device is suitable for the administration of


medical substances and not, for which the administration via aerosol is foreseen; these substances are to be in any case prescribed by the Doctor. In case of too thick substances, the dilution with a suitable physiological solution could be needed, according to the medical prescription. Use original FLAEM accessories only.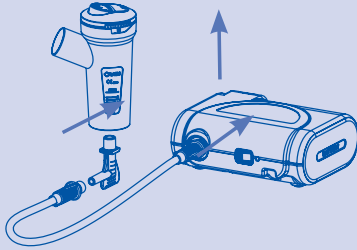


Connect one end of the connection hose (E) to the device.

Connect the other end of the hose (E) to the L-fitting (F).

Connect the L-fitting (F) to the nebulizer (G).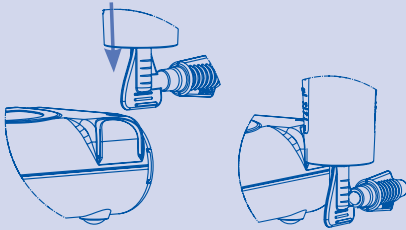

Insert the nebulizer (G) in its housing on the device (A).

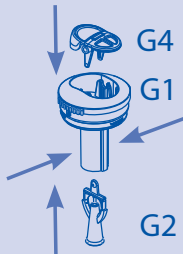
Insert the nozzle (G2) in the upper part (G1) pressing as shown by the 2 arrows. Insert the Speed selector with valve (G4) in the upper part (G1).

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
Close the nebulizer by simply turning the lid clockwise.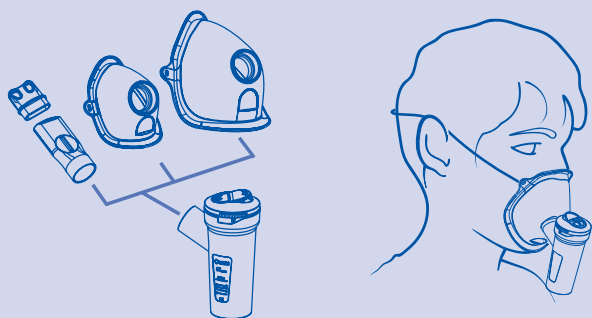


Connect the accessory desired.

If you use the face mask, fit it to the face as shown in the picture


RF7 Dual Speed Plus nebulizer provides 2 different ways to administer medication; consult your doctor for information on the most appropriate method to suit your needs.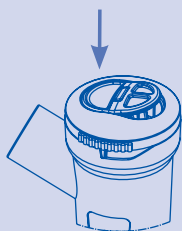

To speed up the inhalation therapy, press on the word MAX of the speed selector button.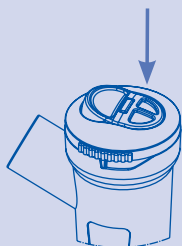

For a more effective inhalation therapy, press on the opposite side of the word Max located on the speed selector button; this allows you to inhale an optimal amount of the medication and reduces dispersion of the medication into the surrounding environment to a minimum thanks to the valve system provided in the nebulizer, mouthpiece, and mask.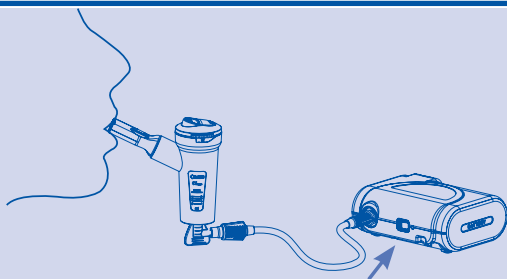

Turn on the device by pressing the on/off switch to the (I) position; the green LED will turn on.

## OPERATION WITH INTERNAL POWER SUPPLY

### DESCRIPTION AND OPERATION WITH BATTERY (OPTIONAL)

The rechargeable Lithium-ion battery you have purchased is partly charged. Therefore, before first use, you must fully charge it, which takes about 2 hours. The autonomy acquired will allow you to carry out several aerosol treatments.


The **lithium-ion** battery has been developed on well-proven and reliable technology. This technology, applied in the medical field for aerosol therapy devices, offers the following advantages:

- high energy storage capacity in a light and compact design (about 45 minutes of autonomy)
- low self-discharge rate (after the battery is fully charged, your device will have power for about 20 minutes or 40% less after 90 days of inactivity)
- high number of charge/discharge cycles

### IMPORTANT SAFEGUARDS FOR USE WITH BATTERY


- Although the battery pack is equipped with a safety system, when you remove it from the device, take care to keep it away from metal objects (e.g. a set of keys) that could cause short circuits. NEVER tamper with the battery pack contacts using, for example, hairpins, scissors, or tweezers.
- To get the most out of your battery you should always use the device at temperatures anywhere between 10° C to 40°C.
- Recharge the unit only when the battery is DISCHARGED; this condition is shown when the green LED on the device turns red. At this point **SHUT OFF THE DEVICE** and do **NOT** attempt to restart the device by pressing the power switch as this could damage the battery. Then proceed to recharge the battery.
- To recharge, use only the equipped power supply adapter (B) or the cable with the cigarette lighter connection (C), included with your Wi.Neb, connected to a 12 VDC receptacle of your vehicle or camper.
- When the battery is charging, the power supply unit and device may heat up without creating any problems.
- Do not recharge the battery outdoors because rain or water spray could damage the device as is not waterproof. Do not immerse the battery in water or get it wet. If the battery does get wet, do not put it in the device and contact an authorized service center or local dealer.
- If necessary, wipe off the battery case with a dry cloth.
- Do not recharge the battery with room temperatures that are less than 4°C or greater than 35°C.
- Do not place the device in areas where the temperature may exceed 35°C to prevent rapid deterioration of the battery.
- Do not expose battery to high temperatures e.g. near radiators, heaters, fireplaces, hobs, ovens, and microwave ovens, etc. and do not dispose of battery in fire.
- Do not expose battery to direct sunlight and do not store it in automobiles especially in hot weather.
- If the battery is subjected to mechanical shocks, it can be hazardous; if this should happen, we recommend that you take it to an authorized service center or to your local dealer to get it checked.
- The battery life at maximum efficiency is about 250 charge cycles, for an average life of 6 months, after which the performance tends to gradually diminish and you should replace the battery. Based on the information above, we recommend that you take full advantage of each battery charge to maximize the battery lifespan.
- If you do not plan to use your Wi.Neb for an extended period, fully charge the battery before storage, even with the battery inserted. Remember to recharge the battery at least once after every 3 months of inactivity to prevent battery damage.
- When not in use, always store your Wi.Neb in a cool and dry environment (not in the refrigerator) to prevent premature aging of the battery.
- Allow the device to reach the operating temperature specified in the Technical Characteristics section at least 24 hours before you decide to use it again or charge the batteries.
- The exhausted battery must be disposed of in specially labeled collection containers, or at any appropriate waste collection center.
- Do not attempt to open the battery case for any reason.

**INSTALLING THE BATTERY OR REPLACING EXHAUSTED BATTERY**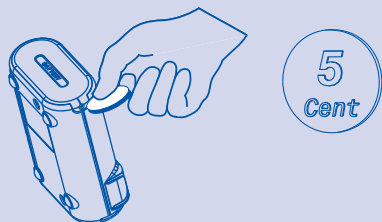

Dispose of batteries in accordance with local regulations. Do not dispose of batteries in normal household waste.


To remove the “**BATTERY**” cover, insert a small coin in the rear slot and gently pry it open.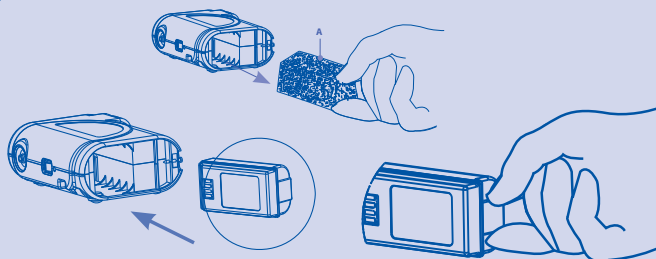

Before inserting the battery, remove the sponge silencer from the battery compartment. Insert the original Flaem battery available for purchase (cod. ACO377P) at your local dealer or authorized FLAEM service center.

**REPLACEMENT AND END OF LIFE**


Remove the exhausted battery and dispose in specially labeled collection containers, or contact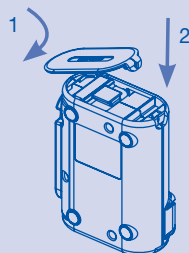


an appropriate waste collection center

Replace the “**BATTERY**” cover by 1 positioning the front part in the seat and 2 gently pressing on the rear part until it snaps securely into place.

**HOW TO CHARGE BATTERY (connection with power adapter)**

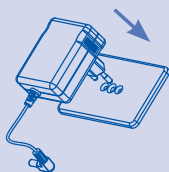
Plug power adaptor in the electrical receptacle. This must be positioned so that it is not difficult to make the disconnection from the mains.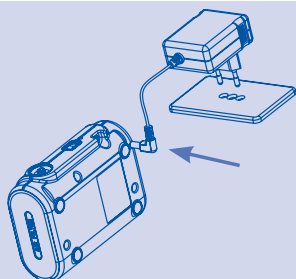


Plug the power adaptor cord in the DC outlet.

## HOW TO CHARGE BATTERY (connection with cigarette lighter cable)

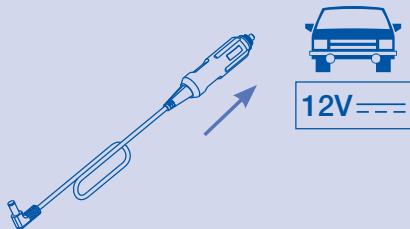
Insert the cigarette lighter plug in the 12V DC cigarette lighter receptacle of the vehicle/ camper/ boat. Check the User Manual of the vehicle to make sure that the voltage supplied is 12Vdc DC.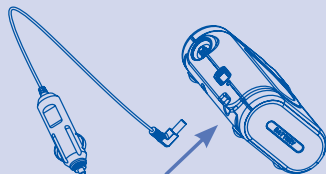


Insert the "L" jack of the cigarette lighter cable in the DC plug of the device.

## CHARGING SEQUENCE

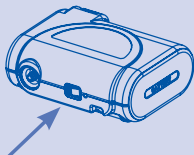Make sure that the switch is OFF, in position (0).

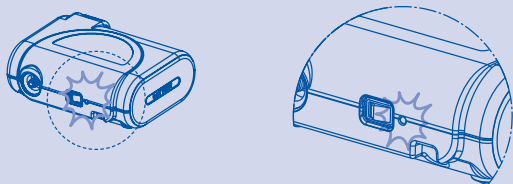Flashing light (\*) = battery charging

Light off= battery charged, device ready for use

Red light = battery discharged

(\*) blue light for basic version / green light for plus version

**The device features the auto-off function, meaning that after 20 minutes of uninterrupted operation, the device switches off to save battery life and the luminous LED switches from green to flashing red.**

**Please be reminded that the device is approved for operation 30 minutes ON - 30 minutes OFF.**

## CLEANING, SANITIZATION AND DISINFECTION

Turn off the unit before any cleaning and disconnect the power cord from the mains.

### CLEANING -UNIT AND OUTER SURFACE OF THE TUBE

Only use a cloth dampened with antibacterial detergent (non-abrasive and free of solvents of any kind).

### NEBULIZER AND ACCESSORIES

Open the nebuliser by turning the upper part (G1) anticlockwise, remove the nozzle (G2) from the upper part (G3) by pressing as shown by the 2 arrows in the “Connection diagram” in point G. Then proceed according to the following instructions.

#### SANITISATION

Before and after each use, sanitise the nebuliser cup and the accessories, choosing one of the methods described below.

**method A:** Sanitise accessories **G1-G2-G3-G4-H-I-L-M-F** under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

**method B:** Sanitise accessories **G1-G2-G3-G4-H-I-L-M-F** in the dishwasher with a hot cycle.

**method C:** Sanitise accessories **G1-G2-G3-G4-H-I-L-M-F** by immersing them in a solution of 50% water and 50% white vinegar, then rinse thoroughly under potable hot water (approximately 40°C).

If you want to also perform the cleaning for DISINFECTION, jump to the DISINFECTION paragraph.

After having sanitised the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).

#### DISINFECTION

After sanitising the nebuliser cup and the accessories, disinfect them choosing one of the methods described below.

**method A:** Accessories **G1-G2-G3-G4-H-I-L-M-F** can be disinfected.

The disinfectant must be an electrolytic chloroxidizer (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

Implementation:

- Fill a container big enough to hold all of the parts to disinfect with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse abundantly with warm potable water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

**method B:** Sanitise the accessories **G1-G2-G3-G4-H-I-F** by boiling them in water for 10 minutes; use demineralised or distilled water to prevent calcium deposits.

**method C:** Sanitise the accessories **G1-G2-G3-G4-H-I-F** with a hot steam steriliser for baby-bottle (not the microwave type). Perform the process faithfully following the instructions of the steriliser. To ensure that the disinfection is effective, choose a steriliser with an operating cycle of at least 6 minutes.


After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

At the end of each use store the device complete with accessories in a dry place away from dust.


## FILTER REPLACEMENT

- Under normal conditions of use, the filter should be replaced after every 100 applications, or at least once a year.
- You must check the filter on a regular basis (every 10-15 applications).
- Replace the filter if it is dirty, a brownish or grayish color, or clogged.
- Replace the filter if it is damp or wet.
- Do not wash or dry the filter.
- Do not replace the filter during use.


**Only use original accessories and spare parts by Flaem, we disclaim any liability in the event of using non original spare parts or accessories.**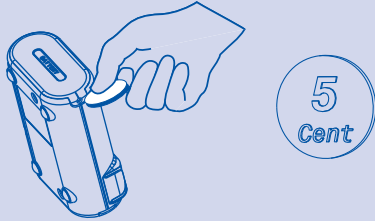

To remove the “**AIR FILTER**” cover, insert a small coin in the rear slot and gently pry open.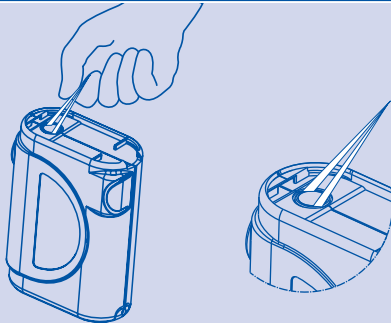

Remove the dirty filter using household tweezers and insert a new filter making sure to position it all the way back in its seat.

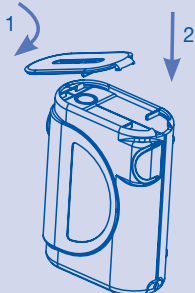Replace the “**AIR FILTER**” cover by 1 positioning the front part in the seat and 2 gently pressing on the rear part until it snaps securely into place.

## TROUBLESHOOTING

<p><b><u>If the device does not work</u></b> when operated using power adaptor or cigarette lighter cable)</p>	<ul style="list-style-type: none"> <li>• Check that the power adaptor plug is properly plugged in. Check that the electrical receptacle functions by plugging in a different device to see if it works.</li> <li>• Check if the power cord and/or plug are faulty or damaged. In this case do not use the device.</li> <li>• Check the plug for connecting the cable to the 12 V DC cigarette lighter receptacle is properly inserted and that the fuse, found in the connection plug is not blown.</li> </ul>
<p><b><u>If the device does not work</u></b> (when battery operated)</p>	<ul style="list-style-type: none"> <li>• Check that battery charging operations have been performed correctly (see section 1.4)</li> </ul>
<p><b><u>If the device provides low mist</u></b></p>	<ul style="list-style-type: none"> <li>• Check that the medicine is not too cold; it should be close to room temperature.</li> <li>• Check that there is at least 2ml of medicine in the nebulizer.</li> <li>• When the device is in operation, do not disconnect the power adaptor from the electrical receptacle without first disconnecting the “L” jack of the device, as this could cause a malfunction.</li> </ul>
<p><b><u>If the device provides no mist</u></b></p>	<ul style="list-style-type: none"> <li>• Check for medicine in the nebulizer.</li> <li>• Check that the nebulizer nozzle is not clogged.</li> <li>• Check that the air inlet of the device and the accessories are properly connected (see connection diagram in fig. 1).</li> <li>• If you are using the device with the power adaptor, make sure that the device is connected to the power adaptor and that the power adaptor is connected to the receptacle.</li> </ul>
<p><b><u>The nebulizer seems inefficient</u></b></p>	<ul style="list-style-type: none"> <li>• Failure to clean the nebulizer from medicine deposits compromises the efficiency and functioning of the device.</li> <li>• Carefully read and follow the instructions included in the <b>CLEANING, SANITIZATION AND DISINFECTION</b> section.</li> </ul>

If after checking the conditions listed above the nebulizing function of the device still does not work contact your local dealer or authorized service center for assistance.

## SPARE PARTS AVAILABLE

Description	Code
RF7 PLUS nebulizer kit composed of: nebulizer, mouthpiece with valve and nasal prong	ACO436P
1m connection tube	ACO35
Adult mask	ACO437P
Pediatric mask	ACO438P
Air filter replacement kit for device (2 pcs)	ACO439P
Removable Lithium-ion battery	ACO377P
Multi-volt universal power adaptor	ACO458P
	ACO460P with USA plug
	ACO214P
Power cord for 12V DC cigarette lighter receptacle	ACO 211P

## TECHNICAL SPECIFICATIONS

Device mod. P0905EM	
Power supply:	12V DC
Absorption:	1.25 A
Max Pressure:	approx. 1.7 bar
Compressor air output:	approx. 14 l/min
Noise level (at 1 m):	approx. 52 dB (A)
Operation:	30 minutes ON – 30 minutes OFF
Operating conditions:	Temperature: min 10°C; max 40°C Air humidity: min 10%; max 95% Atmospheric pressure: min 69KPa; max 106KPa
Storage conditions:	Temperature: min -25°C; max 70°C Air humidity: min 10%; max 95% Atmospheric pressure: min 69KPa; max 106KPa
In conformity with Directive 93/42 EEC	 0051
Safety approvals:	
Dimensions (W)x(D)x(H)	13x9.5x5.5 cm
Weight	0.545 Kg

## APPLIED PARTS

Type BF applied parts are:	patient accessories (H, I, L, M)
----------------------------	----------------------------------

<b>RF7 PLUS Nebulizer</b>	
Medication minimum capacity:	2 ml
Medication maximum capacity:	8 ml
Operating pressure (with nebulizer):	approx. 0.7 bar
(1) Delivery:	Min With valve 0.18 ml/min
	Max: approx. 0.42 ml/min
(2) MMAD:	Min With valve 2.95 µm
	Max: 2.44 µm
(2) Breathable fractions < 5 µm (FPF):	Min With valve 74.7%
	Max: 80.7%
(1) data detected according to Flaem I29-P07.5 internal procedure	
(2) In vitro characterization certified by TÜV Rheinland LGA Products GmbH - Germany in compliance with the new European Standard for aerosol therapy units, Standard EN 13544-1, ANNEX CC. Further details are available on request.	
Use only power adaptor mod. UME318-1215 (cod. 13014)	
Primary:	100/240V~ 50/60Hz
Secondary:	12V DC 1.5 A
<b>Universal power adaptor (USA alternative)</b>	
Use only power adaptor mod. UML318-1215 (cod. 13373)	
Primary:	100/240V~ 50/60Hz
Secondary:	12V DC 1.5 A
<b>Power cord for 12V DC cigarette lighter receptacle</b>	
If the fuse inside the connection plug blows, replace with 3A.	
<b>Disposal of device</b>	
In conformity with Directive 2012/19/EC, the symbol shown on the device to be disposed of indicates that it is considered as waste and is therefore subject to "sorted waste collection". The user must therefore take (or have taken) the above waste to a pre-sorted waste collection centre set up by the local authorities, or else give it back to the dealer when purchasing a new appliance of the same type. Pre-sorted waste collection and the subsequent treatment, recovery and disposal operations favor the production of appliances made of recycled materials and limit the negative effects of any incorrect waste management on the environment and public health. The unlawful disposal of the product by the user could result in administrative fines as provided by the laws transposing Directive 2012/19/EC of the European member state or of the country in which the product is disposed of.	

Removable battery (optional)	
Type:	Li-ion 7.4V 2200mAh
Autonomy with fully charged new battery:	about 45 minutes (at a temperature of about 25°C)
Possibility to recharge:	with power adaptor or cable with plug for cigarette lighter receptacle included with the Wi.Neb device
Minimum charge time:	about 2 hours
Operating conditions:	Temperature: min 10°C; max 40°C RH Air humidity: min 10%; max 75% Operating atmospheric pressure: P min 86KPa; max 106KPa
Storage conditions: (3 months max)	Charge the battery 100% before putting the device in storage Recharge the battery once after every three months of inactivity. Temperature: min -5°C; max 35°C. RH Air humidity: min 10%; max 75%. Storage atmospheric pressure: P min 86KPa; max 106KPa
Weight	0,140 Kg

## SYMBOLS

	Class II device	<b>IP21</b>	<b>Enclosure protection rating: IP21.</b> (Protected against solid bodies over 12 mm. Protected against access with a finger; Protected against vertically falling water drops.)
	Type BF applied part		Alternate current
	Important: check the operating instructions		Power adaptor for household use
	Direct current		Plug for safety extra low voltage
	Switch "ON"		CE mark for medical devices ref. Dir. 93/42 EEC and subsequent amendments
	Switch "OFF" When you shut off the device, the switch will interrupt the compressor function only on one of the two power phases		
	Outlet for safety extra low voltage		TUV approval ref. EN 60601-1 3° Ed. - EN 60601-1-11
	Outlet for safety extra low voltage		Serial number of device
	In compliance with European standard EN 10993-1 Biological evaluation of medical devices and 93/42/EEC Medical Devices Directive		Manufacturer

## ELECTROMAGNET COMPATIBILITY

This device has been designed to meet the current requirements for electromagnetic compatibility (EN 60601-1-2:2015). For additional information please visit [www.flaemnuova.it](http://www.flaemnuova.it). The Device may be subject to electromagnetic interference if other devices are used for specific diagnosis or treatments.

The Manufacturer reserves the right to make technical and functional changes to the product without notice.

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© Copyright by FLAEM NUOVA S.p.A., 2017 11845C0-0 rev.05/2018 (TUV A1)