Rhino Clear

by **FLAEM**

SINCE 1966
MADE IN ITALY

MANUALE ISTRUZIONI D'USO DELLA DOCCIA NASALE
NASAL SHOWER USER INSTRUCTION MANUAL
MODE D'EMPLOI DE LA DOUCHE NASALE
BEDIENUNGSANLEITUNG NASENSPÜLUNG
MANUAL DE INSTRUCCIONES DE USO DEL NEBULIZADOR NASAL
MANUAL DE INSTRUÇÕES DE USO DA DUCHA NASAL
РУКОВОДСТВО ПОЛЬЗОВАТЕЛЯ НАЗАЛЬНОГО ДУША
INSTRUKCJA OBSŁUGI IRYGATORA DO NOSA
BRUGSANVISNING TIL NÆSEDOUCHE
NENÄSUIHKUN KÄYTTÖOPAS
BRUKSINSTRUKSJONER FOR NESEDUSJ
NEUSDOUCHE GEBRUIKERSHANDLEIDING

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NASAL WASH

“The pleasure of breathing well”

Thank you for the purchase you made. Our goal is to ensure satisfaction to all our consumers by offering them cutting-edge products for the prevention and treatment of respiratory tract. Visit the website

www.flaem.it. Please read carefully these instructions in order to successfully use the nasal shower.

In addition, we recommend you to keep this manual carefully for any further consultations.

Rhino Clear is a medical device for home use.

What is it?

RHINO CLEAR® is a nasal shower specifically designed to wash and treat both adults and children nasal cavities, either using saline solutions or aerosol solutions for administering drugs with a nebulizer, according to medical prescription.

How does it work?

RHINO CLEAR® connected to a FLAEM compressor, nebulizes solutions for washing the nasal cavities, nebulization is activated by a push button on the handle. On the body of the nasal shower are two adjacent chambers, one for the solution to be nebulized and the other for the recovery solution. At the top of the nasal shower are inserted the nasal adapters available in 3 different sizes, depending on the size of the nostril of the user.

When to use it?

RHINO CLEAR® can be:

- 1. Used for physiological cleaning of nasal cavities** (saline solutions or thermal water). Useful to help child with proper nasal hygiene. Used in adults to favour physiological cleaning of the nose and can be used before the aerosol therapy for a more effective drug distribution in the respiratory tract.
- 2. Used for therapeutic cleaning:** drugs administration through **RHINO CLEAR®** must be made on the advice of a physician or a pharmacist, who defines drug selection, dosage, timing of administration and duration of treatment.
- 3. Used to help children to breathe well**, in fact, up to 1 year old the child only breathes through the nose, then, if the latter is blocked (snuffles), the child breathes with difficulty, he wakes up in the night, struggles to eat and cries often. Considering that child can't usually blow his own nose until about the age of 3, the mucus with trapped bacteria, viruses and dust stagnates in the nasal passages. To clear a blocked nose two are the best-known and recommended methods: the nasal cleaning and the removal of mucus; to clean we suggest **RHINO CLEAR®** and to remove the mucus the baby nasal aspirator included in the Baby FLAEM products line.

DEVICE COMPOSITION:

- 1) Nasal transparent adapter "A", suitable for children up to 3 years of age
Nasal white adapter "B", suitable for children 3 to 12 years old
Nasal blue adapter "C", suitable for children 12 years old and up
- 2) Separator
- 3) Atomizer
- 4) Shower body with ergonomic handle
 - 4.1) 10 ml nebulization chamber
 - 4.2) Collection chamber
- 5) Dispensing button
 - 5.1) Button
 - 5.2) Rubber cap
- 6) Nasal shower-tube connect fitting
- 7) Connection tube to the compressor (not included in the package)

IMPORTANT NOTICES

- This is a medical device (Compliant with dir. 93/42/CEE) and must be used with medications prescribed or recommended by your doctor. Before using the accessories, consult the device operating instructions manual. It is important for the patient to read and understand the information for use. Contact your retailer or service centre if you have any questions.
- Children and people who are not self-sufficient must use the device under the close supervision of an adult who has read this manual.
- Some parts of the device are small enough to be swallowed by children; therefore, keep the accessory out of the reach of children.
- Interactions: the materials used in contact with medication have been tested with a vast range of medications. However, in view of the variety and continuous evolution of pharmaceuticals, interactions cannot be ruled out. We recommend using the medication as soon as possible once it has been opened and preventing prolonged exposure in the nasal wash system.

INSTRUCTIONS FOR USE

Before each utilisation, wash your hands accurately and clean the NASAL WASH SYSTEM and the accessories as described in the paragraph "CLEANING, SANITISATION, DISINFECTION AND STERILISATION".

The NASAL WASH SYSTEM and accessories are for personal use only to avoid potential risks of transmitting infection.

During application, it is advisable to suitably protect yourself from any dripping.

1. Insert the atomiser (3) into the nebulisation chamber (4.1).
2. Put the solution (max 10 ml) into the nebulisation chamber (4.1)(fig. A).
3. Apply the separator (2) verifying that the arrow on the shower body is aligned with the separator (Pic. B). Then, insert the correct adapter for your age, or the most appropriate to fit the size of your nostrils (1A/1B/1C).
4. Connect the nasal douche system to the device using the tubing (7) and the fitting (6).
5. Switch on the device (see the user manual).

6. Bring it to your nose, placing the hole of the adapter against one nostril.
7. During treatment, it is better to close with a finger the nostril where the NASAL WASH SYSTEM is not applied.
8. Activate the NASAL WASH by holding down the delivery button (5) while inhaling and release the button before exhaling; to help expel the mucous, blow gently with the nostril being treated directly into the NASAL WASH SYSTEM.
9. Repeat steps 6. through 8. for the other nostril, trying to distribute the solution evenly between the two nostrils.
10. For children or anyone unable to correctly coordinate the various parts of step 8., we recommend alternating the application between the nostrils every 20 seconds, leaving suitable pauses to allow the mucous to be expelled.
11. The steps described above are useful for nasal irrigation using saline solutions. It is advisable to carry out medicated treatments after an initial irrigation with saline solution for a more efficient distribution of the medication on the nasal mucous membranes.
12. Once the application is complete, switch off the device and clean it, following the instructions provided in the "CLEANING, SANITISATION, DISINFECTION STERILISATION" paragraph.

CLEANING, SANITISATION, DISINFECTION, STERILISATION

Switch off the device before any cleaning procedure and unplug the power cable from the socket.

DEVICE AND TUBING EXTERIOR

Use only a damp cloth with antibacterial soap (non-abrasive and with no solvents of any sort).

ACCESSORIES

Disassemble the NASAL WASH SYSTEM as illustrated in the connection diagram. Then proceed according to the instructions below.

CLEANING AT HOME - SANITISATION AND DISINFECTION

SANITISATION

Before and after each use, sanitise the nebuliser cup and the accessories, choosing one of the methods described below.

method A: Sanitise accessories 1A-1B-1C-2-3-4-5-6 under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

method B: Sanitise accessories 1A-1B-1C-2-3-4-5-6 in the dishwasher with a hot cycle.

method C: Sanitise accessories 1A-1B-1C-2-3-4-5-6 by immersing them in a solution with 50% water and 50% white vinegar, then rinse thoroughly under potable hot water (approximately 40°C).

If you also wish to perform cleaning by DISINFECTION, jump to the DISINFECTION paragraph.

After having sanitised the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).

DISINFECTION

After sanitising the nebuliser cup and the accessories, disinfect them choosing one of

the methods described below.

method A: The disinfectable accessories are 1A-1B-1C-2-3-4-5-6.

The disinfectant must be an electrolytic chlorine oxidiser (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

Directions:

- Fill a container of suitable size to hold all of the parts to be disinfected with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse with plenty of warm potable water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

method B: Sanitise the accessories 1A-1B-1C-2-3-4-5-6 by boiling them in water for 10 minutes; use demineralised or distilled water to prevent calcium deposits.

method C: Sanitise the accessories 1A-1B-1C-2-3-4-5-6 with a hot steam steriliser for baby-bottle (not the microwave type). Perform the process faithfully following the instructions of the steriliser. To ensure that the disinfection is effective, choose a steriliser with an operating cycle of at least 6 minutes.

After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

CLEANING IN A CLINICAL OR HOSPITAL SETTING - DISINFECTION AND STERILISATION

Before disinfection or sterilisation, sanitise the nebuliser cup and the accessories, choosing one of the methods described below.

method A: Sanitise accessories 1A-1B-1C-2-3-4-5-6 under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

method B: Sanitise accessories 1A-1B-1C-2-3-4-5-6 in the dishwasher with a hot cycle.

DISINFECTION

The disinfectable accessories are 1A-1B-1C-2-3-4-5-6.

The disinfectant must be an electrolytic chlorine oxidiser (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

Directions:

- Fill a container of suitable size to hold all of the parts to be disinfected with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.

- Remove the disinfected parts and rinse with plenty of warm potable water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

If you want to also perform the STERILISATION, jump to the STERILISATION paragraph

After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

STERILISATION

The accessories that can be sterilised are 1A-1B-1C-2-3-4-5.

Equipment: Fractionated vacuum overpressure steam steriliser in accordance with EN 13060.

Directions: Wrap every single part to be treated with a sterile barrier system or packaging in accordance with Norm EN 11607. Place the packed components in the steam steriliser. Run the sterilisation cycle according to the operating instructions of the device by selecting a temperature of 121°C and a time of 10 minutes first.

Storage: Store the sterilised parts as per the instructions for use of either the sterile barrier system or packaging.

The sterilisation procedure is validated in its conformity to ISO 17665-1.

NASAL WASH SYSTEM TECHNICAL FEATURES

Mod.: Rhino Clear®

Operating Conditions:

Temperature: min 10°C; max 40°C

Relative humidity: min 10%; max 95%

Atmospheric pressure: min 69KPa; max 106KPa

Storage conditions:

Temperature: min -25°C; max 70°C

Relative humidity: min 10%; max 95%

Atmospheric pressure: min 69KPa; max 106 KPa

Capacity: medication maximum 10 ml

Type BF applied parts are: patient accessories (1A-1B-1C)

Compliant with Directive 93/42 EEC-

| | |
|-----------------------------|------------------------------------|
| Family of FLAEM compressors | 10 ml Delivery Time ⁽¹⁾ |
| F400 | 2 minutes (approx) |
| F700/F1000 | 2 minutes (approx) |
| F2000 | 2 minutes (approx) |

⁽¹⁾ Tests performed with 0.9% NaCl saline solution, according to the Flaem internal procedure

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© Copyright by FLAEM NUOVA S.p.A.,
FLAEM NUOVA S.p.A. - Via Colli Storici, 221