**MISURATORE DI PRESSIONE AUTOMATICO
AUTOMATIC BLOOD PRESSURE MONITOR
TENSIO MÈTRE AUTOMATIQUE
AUTOMATISCHE SFYGMOMANOMETER
AUTOMATISCHES BLUTDRUCKMESSGERÄT
TENSIO MÈTRO AUTOMÁTICO**

**Superior
accuracy**
DK
Technology

- I** MANUALE ISTRUZIONI D'USO
- GB** INSTRUCTIONS FOR USE MANUAL
- F** MODE D'EMPLOI
- NL** GEBRUIKSAANWIJZING
- D** BEDIENUNGSANLEITUNG
- E** MANUAL DE INSTRUCCIONES

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Congratulations for your purchase and thank you for choosing our unit. Carefully read these instructions and keep them in a safe place for future reference.

Use the unit only as described in this instruction manual.

For specific information on your own blood pressure, contact your physician.

This Blood Pressure Monitor bears the CE mark "CE 0051" and is manufactured in compliance with the essential requirements of Directive 93/42/EEC concerning medical devices, as well as the EMC directive 89/336/EEC:

EN 1060-1: 1995/A1: 2002 Non-invasive sphygmomanometers - Part 1: General requirements

EN 1060-3: 1997/A1: 2005 Non-invasive sphygmomanometers - Part 2: Supplementary requirements for electromechanical blood pressure measuring systems

EN 1060-4: 2004 Non-invasive sphygmomanometers - Part 4: Test Procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers.

Blood pressure measurements determined with this automatic blood pressure monitor are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers. This monitor is to be used by adult consumers in a home environment.

1. Digital Korotkoff Sound (DK) Technology

The "Digital Korotkoff Sound" (DK) technology is a combination between the comfortable oscillometric method and the competence of the clinically tested Korotkoff Sound method. A peculiar feature of this appliance is its advanced system fitted with a pair of sensors for pressure and sound, whose detections are then processed by a sophisticated algorithm which determines the result of the measurement and clearly shows it on the large display of the appliance.

2. About Blood Pressure

What is Blood Pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

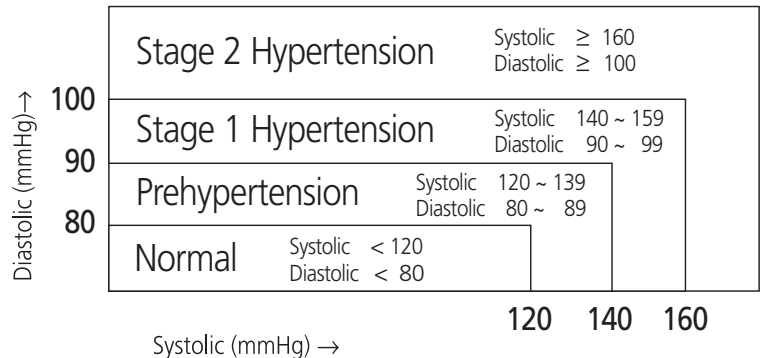
Why Measure Your Blood Pressure?

Among the various health problems afflicting people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.

3. Blood Pressure Standard

The World Health Organization (WHO) and National High Blood Pressure Education Program Coordinating Committee have developed a blood pressure standard, according to which areas of low- and high-risk blood pressure are identified. This standard, however, is a general guideline as individual's blood pressure varies among different people and different age groups...etc.


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.


4. Blood Pressure Fluctuation

Blood pressure fluctuates all the time!

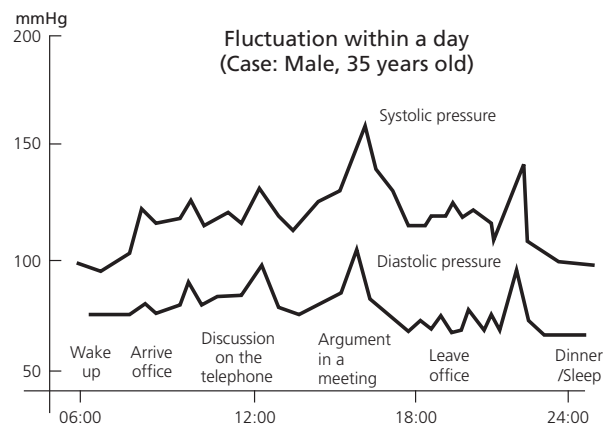You should not be over worried if you encountered two or three measurements at high levels. Blood pressure changes over the month and even throughout the day. It is also influenced by season and temperature.

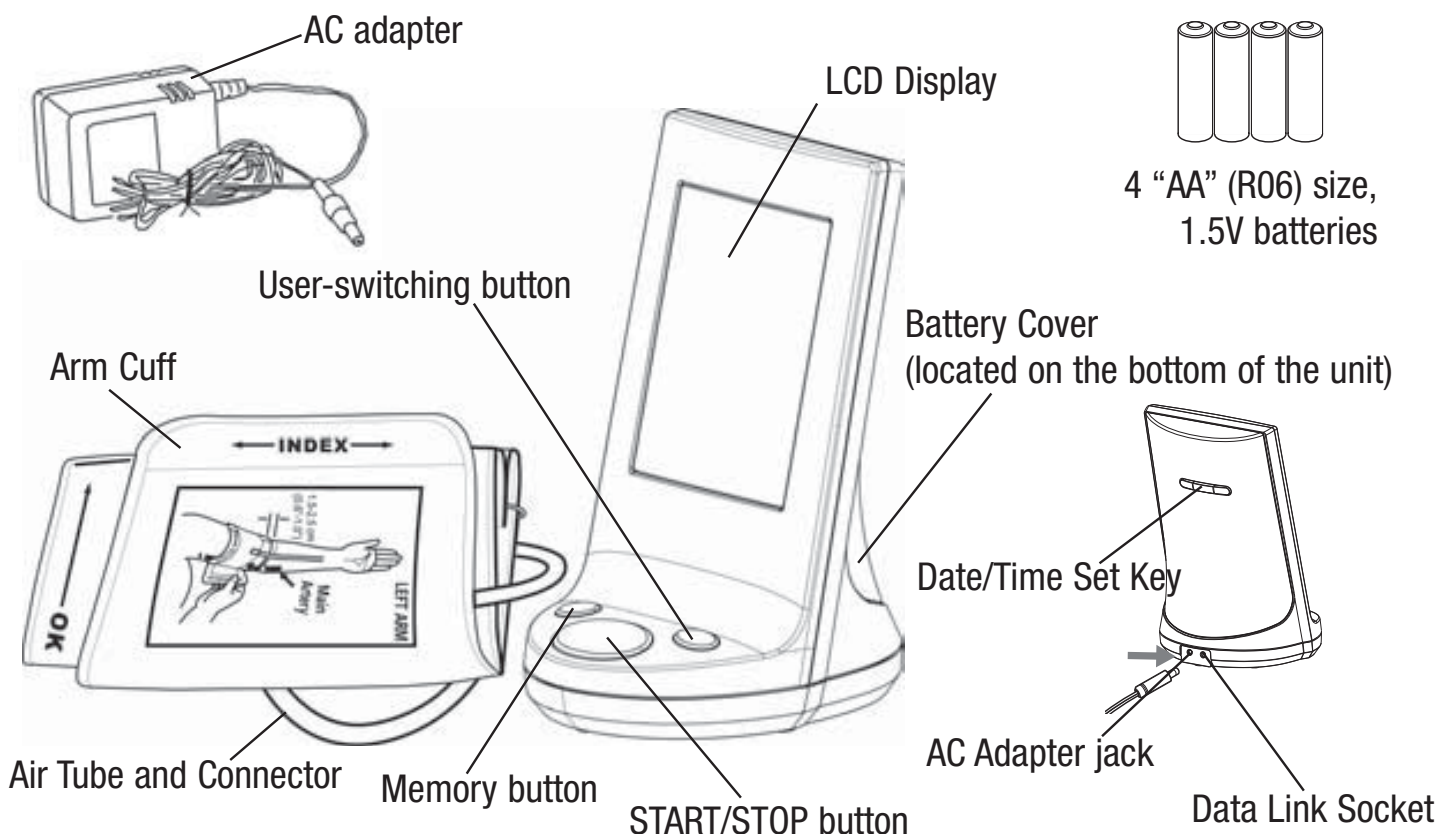
5. Name/Function of Each Part

6. Display Explanations

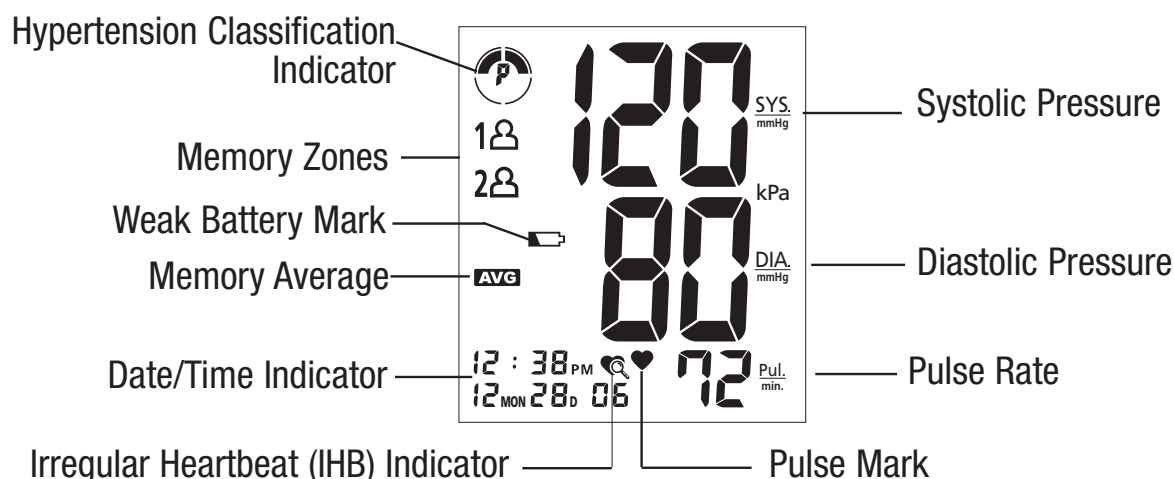Display:

Icons:

1 **Memory Zone 1:** Appears when the monitor is operated in memory zone 1

2 **Memory Zone 2:** Appears when the monitor is operated in memory zone 2

Weak Battery Mark: Appears when batteries should be replaced

Pulse Mark: Shows the pulse rate per minute

Irregular heartbeat icon: see section 16. at page 18

AVG. **Memory Average:** Displays average of last 3 memories

Hypertension Classification Indicator: see paragraph 7. page 14

EE **Measurement Error:** Make sure the L-plug is securely connected to the air socket and measure again quietly. Wrap the cuff correctly and keep arm steady during measurement. If error keeps occurring, return the device to your dealer or an authorised service centre.

E 1 **Air Circuit Abnormality:** Make sure the L-plug is securely connected to the air socket and measure again quietly. If error keeps occurring, return the device to your dealer or an authorised service centre.

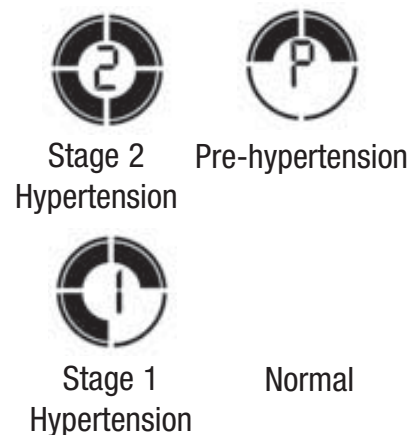
EE **Pressure Exceeding 300 mmHg:** Switch the unit off and measure again quietly. If error keeps occurring, return the device to your dealer or an authorised service centre.

E3 **Data Error:** Remove and reload the batteries. If error keeps occurring, return the device to your dealer or an authorised service centre.

E r **The measurement exceeds the acceptable interval:** Repeat the measurement, remaining still and calm. If the problem continues, return the appliance to your dealer or to an authorised service centre.


7. Hypertension Classification Indicator

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard, classifying blood pressure ranges into 4 stages. This unit is equipped with innovative blood pressure classification indicator, which visually indicates the assumed risk level (prehypertension / stage 1 hypertension / stage 2 hypertension) of the result after each measurement.

8. Installing Batteries

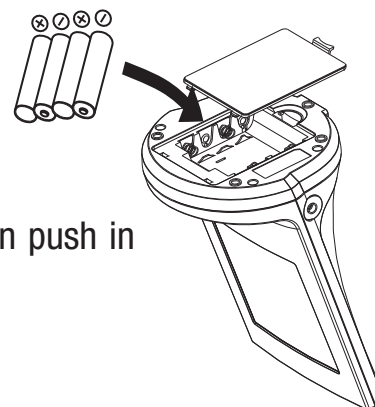1. Press down and lift the battery cover in the direction of the arrow to open the battery compartment.
2. Install or replace 4 “AA” sized batteries in the battery compartment according to the indications inside the compartment.
3. Replace the battery cover by clicking in the bottom hooks first, then push in the top end of the battery cover.

You need to replace the batteries when:

1. low battery icon appears on display.
2. the ON/OFF/START button is pushed and nothing appears on display.

Replace the batteries in pairs. Remove batteries when unit is not in use for extended periods of time.

Warning: The removal of even one battery causes cancellation of the data contained in the 2 memory channels (1 and 2).

Batteries disposal: to safeguard the environment, when the batteries have reached the end of life, dispose them to an appropriate collection site.

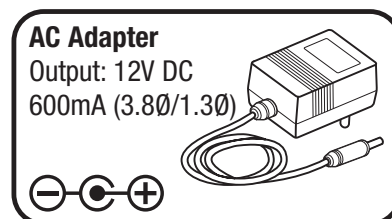
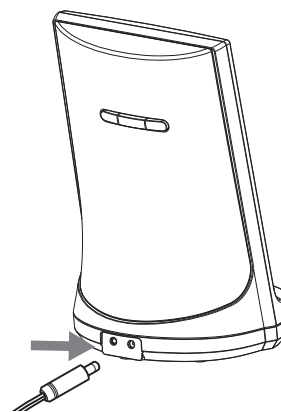
9. Using the AC adapter (included with the unit)

1. Connect the AC adapter with the AC adapter jack in the back of the unit.
2. Plug the AC adapter into the socket.

The power unit provided has been specifically designed for use with the AD701K unit. Never use the power unit with other appliances or for uses other than those indicated in this manual and never use AD701K with other power units. Never use adapters for mains voltages different from those shown on the supply unit (plate details). Keep the cord away from heated surfaces.


Caution:

1. Please unload the batteries when operating with the AC mode for a longer period of time. Leaving the batteries in the compartment for a long time may cause leakage, which may lead to damage of the unit.
2. No batteries are needed when operating with the AC mode.
3. The disconnection of the power supply unit from the power mains causes cancellation of the data contained in the 2 memory channels (1 and 2).

10. Applying the cuff

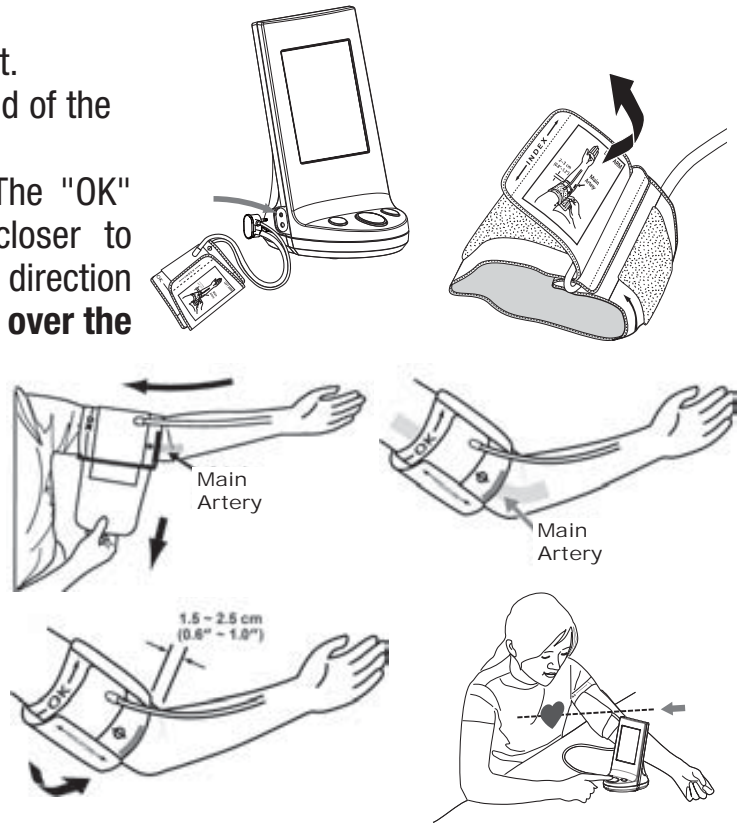1. Plug in the cuff connecting tube into the unit.
2. Unwrap the arm cuff, leaving the "Index" end of the cuff through the D-ring of the cuff
3. Put your left arm through the cuff loop. The "OK" range indication should be positioned closer to yourself with the rubber tube pointing in the direction of your arm. **Position the artery mark (ϕ) over the main arteries (on the inside of your arm) in the upper arm.**
4. Turn your left palm upward and place the edge of the arm cuff at approximately 1.5 to 2.5 cm above the inner side of the elbow joint. Tighten the cuff by pulling the "Index" end of the cuff.
5. **If the Index line falls within the OK range, indicated at the edge of the cuff, this cuff is suitable for your use.** If the Index line falls out of the OK range, you may need a cuff with other circumferences.



11. Measurement Procedures

1. Press the **User-Switching** key to select Memory Zone 1 or Memory Zone 2.

To confirm your choice, now switch off the appliance by pressing the START / STOP key. At this point, the subsequent measurement will be automatically stored in the selected channel.

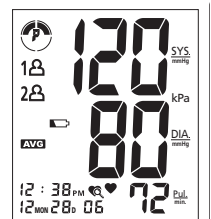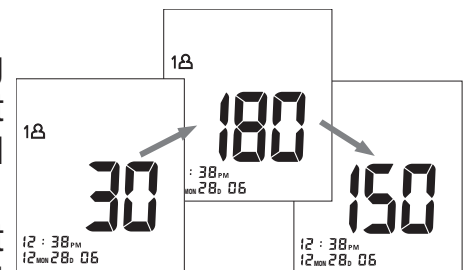2. To start measuring, press key START / STOP. All digits will light up, checking the display functions. The checking procedure will be completed after about 2 seconds.
3. After all symbols appear, the display will show a blinking "0". The monitor is **"Ready to Measure"** and will **automatically** inflate the cuff slowly to start measurement.
4. When the measurement is completed, the cuff will exhaust the inside pressure. Systolic pressure, diastolic pressure and pulse will be shown simultaneously on the LCD screen. **The measurement is then automatically stored into the pre-designated memory zone.**

This monitor will re-inflate automatically to approximately 220 mmHg if the system detects that your body needs more pressure to measure your blood pressure.

Note:

1. This monitor automatically switches off approximately **1 minute** after last key operation.
2. To interrupt the measurement, simply press the **M** or START/STOP key. The cuff will deflate immediately after a key is pressed.
3. During the measurement, do not talk or move your arm or hand muscles.

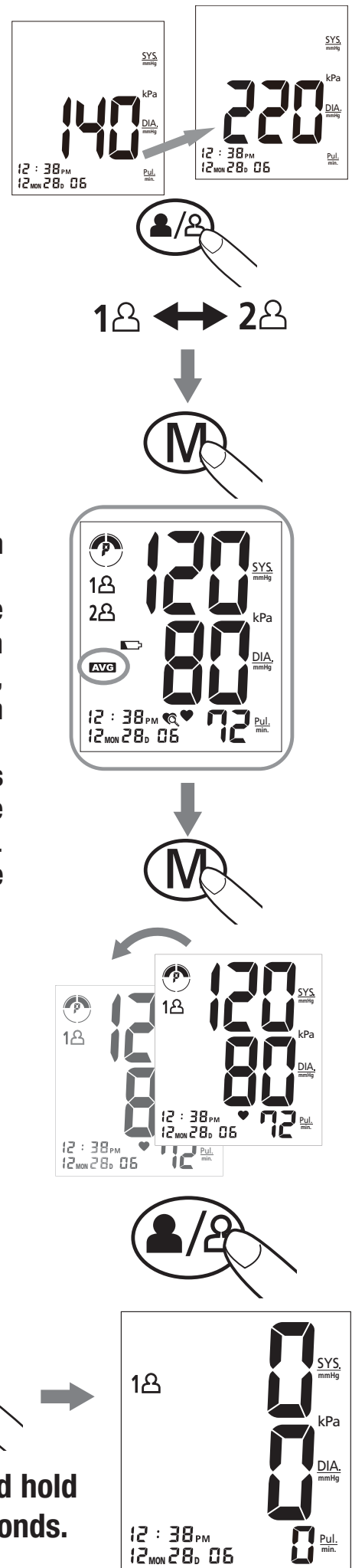
12. Recalling Values from Memory

1. The Monitor has two Memory Zones (1 and 2). **Each zone can store up to 60 measurements.**
2. To read memory values from a selected Memory Zone, use the **User-Switching Key** to select a Memory Zone (1 or 2) from which you want to recall values from. Then, press the **M key**, and the values will be shown on the LCD display with an assigned memory sequence number.
3. **The monitor will display an average (AVG) of the previous measurements first.** Every new press of the **M key** calls for one set of prior values stored. Latest memory will be recalled first. **The calculated average will be applied to a maximum of the last 3 memories.**

13. Clearing Values from Memory

1. Press the **User-Switching key** to select the memory zone 1 or memory zone 2.
2. Continue to press and hold the **M key** for approximately 5 seconds, then the data in the pre-designated memory zone can be erased automatically.


Note: the removal of even one of the batteries or the disconnection of the supply unit from the power mains causes the cancellation of the data contained in the 2 memory channels (1 and 2). We therefore suggest transcribing the values in the “Blood Pressure Log (see page 61 and 62) before replacing or removing the batteries, or disconnecting the supply unit from the power mains.

Press and hold for 5 seconds.

14. Setting the date/hour

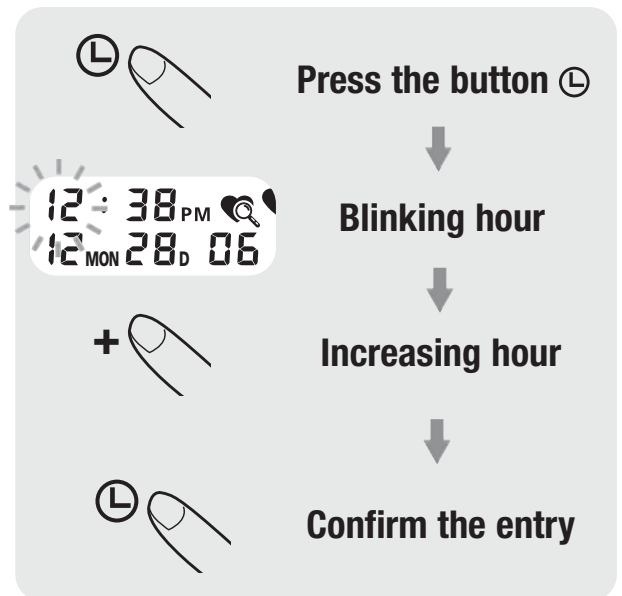1. To adjust the date/ time in the monitor, press the ⌚ key. The display will show a blinking number showing the hour.
2. Change the hour by pressing the "+" key. Each press will increase the number by one in a cycling manner. Press the ⌚ key again to confirm the entry, and the screen will show a blinking number representing the minute.
3. Change the minute, and date as described in Step 2 above, using the "+" key to change and the ⌚ key to confirm the entries.
4. "0" will reappear as the Blood Pressure Monitor is ready for measurement again.


Repeat process to set time and date

15. Data transfer to PC

This accessory can be bought upon request

This unit is equipped with a Data Link Socket; through the suitable interface it can be connected to the PC. Moreover, a software for the following operations is included:

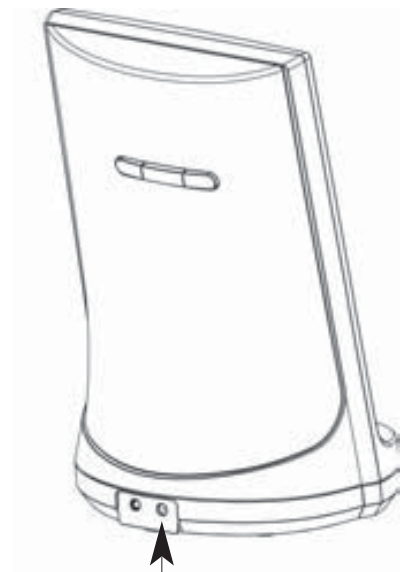- Data interpretation
- Patient health records
- Measurements of blood pressure values
- Average feature for blood pressure
- Analysis of diagram, histogram and distributed drawings


Data Link Socket

16. Irregular Heartbeat (IHB) Indicator

This unit is equipped with an Irregular Heartbeat (IHB) Indicator which allows those who have an irregular heartbeat to obtain accurate measurements alerting the user of the presence of an irregular heart beat during the measurement.

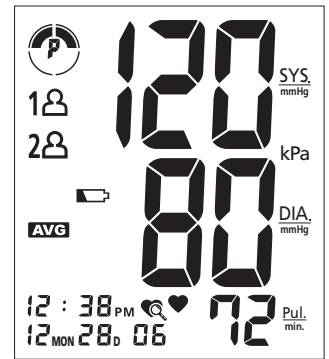You are strongly advised to contact your doctor if this icon  appears often (sign of an irregular heartbeat).

17. Helpful Tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal.
- To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not be physically tired or exhausted while taking a measurement.
- It is important that you relax during measurement. Try to take 15-minute rest before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- **Wait about 5 minutes before taking the next pressure measurement.**

Disposal: In conformity with Directive 2002/96/EC on waste electrical and electronic equipment (WEEE), at the end of its life the unit should be taken to an electrical appliance collection point for its environmentally friendly disposal.

18. Troubleshooting

If any abnormality should arise during use, please check the following points.

Symptoms	Check Points	Correction
No display when the START/STOP key is pressed	Have the batteries run down?	Replace them with four new batteries.
	Have the batteries' polarities been positioned incorrectly?	Re-insert the batteries in the correct positions.
EE mark shown on display or the blood pressure value is displayed excessively low (high)	Is the cuff placed correctly?	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement?	Repeat the measurement, wait 5 minutes after the previous measurement remaining still and calm.
	Did you vigorously shake the cuff during measurement?	

Note: If the unit still does not work, return it to your dealer or an authorised service centre. Under no circumstance should you disassemble and repair the unit by yourself.

To ensure accurate measurements, it is recommended that the performance should be checked every 2 years.

19. Cautionary Notes:

- The appliance contains high precision components. Avoid exposing it to extreme temperatures, humidity, dust and direct sunlight. Do not drop the appliance and protect it against violent knocks. Should this occur, contact your nearest dealer or an authorised after-sales centre to check the appliance.
- Please note that this is a home healthcare product only and it is not intended to serve as a substitute for the advice of a physician or medical professional.
- Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
- The unit should not be operated by children so to avoid hazardous situations.
- If the unit is stored near freezing, allow it to acclimate at room temperature before use.
- Repairs should be performed by authorised personnel only. Unauthorised repairs will invalidate the warranty and may represent a hazard to the user. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact your dealer or an authorised service centre.
- For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use.
- Do not use this device for diagnosis or treatment of any health problem or disease. Measurement results are for reference only. Consult a healthcare professional for interpretation of pressure measurements. Contact your physician if you have or suspect any medical problem. Do not change your medications without the advice of your physician or healthcare professional.
- Do not operate this blood pressure monitor close to strong electrical field, such as cellular phone, microwave ovens, X-rays, etc.
- Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
- To stop operation at any time, press the START/STOP key, and the air in the cuff will be rapidly exhausted.
- Once the inflation reaches 300 mmHg, the unit will start deflating rapidly for safety reason.

20. Specifications:

Mod.	AD701K
Measurement Method:	Digit Arithmetic Waveform of Korotkoff Method
Measurement Range:	Pressure: 40~250mmHg; Pulse: 40~199 beats/minute
Double pressure sensor:	Semi-conductor / acoustic
Accuracy:	Pressure: ± 3 mmHg;
Inflation:	Pump Driven
Deflation:	Automatic Pressure Release Valve
Memory capacity:	120 (60 memories for each zone x 2 zones)
Auto-shut-off:	1 minute after last button operation
Operating conditions:	Temperature: min 10°C; max 40°C Air humidity: min 40%; max 85%
Storage conditions:	 Temperature: min -10°C; max 60°C RH Air humidity: min 10%; max 90%
DC Power Source:	6V DC, four LR06 (AA) Batteries
AC Power Source:	12V DC 600mA, (Plug size: out(-) is 3.8Ø, inner(+) is 1.3Ø)
Dimensions:	159 (L) X 120 (W) X 177 (H) mm
Weight (without batteries):	420g (with arm cuff)
Arm circumference:	Adult: 24 ~36 cm

 Type BF

 Important: check the operating instructions

- Appliance not suitable for use together with inflammable anaesthetic mix with air, or with oxygen or with nitrous oxide.
- Device and cuff are designed to provide special protection against electrical shocks.

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 0051

Made in China

The Manufacturer reserves the right to make technical and functional changes to this product without prior notice.

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