# Indice

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## Table of Contents

|   |    |
|---|----|
| INTRODUCTION.....   | 26 |
| • General Description   |    |
| • Indications for Use   |    |
| • Measurement Principle   |    |
| • Safety Information  |    |
| • LCD Display Signal  |    |
| • Monitor Components  |    |
| • List  |    |
| BEFORE YOU START.....   | 31 |
| • The Choice of Power Supply  |    |
| • Installing and Replacing the Batteries                                    |    |
| • Setting Date, Time and Measurement Unit                                   |    |
| MEASUREMENT.....  | 35 |
| • Tie the Cuff  |    |
| • Start the Measurement   |    |
| DATA MANAGEMENT.....  | 37 |
| • Recall the Records  |    |
| • Delete the Records  |    |
| INFORMATION FOR USER.....   | 39 |
| • Tips for measurement  |    |
| • Maintenances  |    |
| ABOUT BLOOD PRESSURE.....   | 41 |
| • What are systolic pressure and diastolic pressure?                        |    |
| • What is the standard blood pressure classification?                       |    |
| • Irregular Heartbeat Detector  |    |
| • Why does my blood pressure fluctuate throughout the day?                  |    |
| • Why do I get a different blood pressure at home compared to the hospital? |    |
| • Is the result the same if measuring on the right arm?                     |    |
| TROUBLESHOOTING.....  | 43 |
| SPECIFICATIONS.....   | 44 |
| AUTHORIZED COMPONENT.....   | 45 |
| CONTACT INFORMATION.....  | 45 |
| COMPLIED EUROPEAN STANDARDS LIST.....                                       | 46 |
| EMC GUIDANCE.....   | 47 |

## ♥ General Description

Thank you for selecting Sfigmolife Pro arm type blood pressure monitor (SFG62). The monitor features blood pressure measurement, pulse rate measurement and the result storage. The design provides you with two years of reliable service.

Readings taken by the SFG62 are equivalent to those obtained by a trained observer using the cuff and stethoscope auscultation method.

This manual contains important safety and care information, and provides step by step instructions for using the product.

Read the manual thoroughly before using the product.

Features:

- Approx.60mm×92 mm Digital LCD with blue backlight display
- Maximum 250 records per each user
- 3rd technology: Measuring during inflation  
(The updated technology in the world)

## ♥ Indications for Use

The Sfigmolife Pro Blood Pressure Monitor is digital monitors intended for use in measuring blood pressure and heartbeat rate with arm circumference ranging from 22 cm to 32 cm ( about 8¾"-12½" ).

It is intended for adult indoor use only.

## ♥ Measurement Principle

This product uses the Oscillometric Measuring method to detect blood pressure.

Before every measurement, the unit establishes a "zero pressure" equivalent to the air pressure. Then it starts inflating the arm cuff, meanwhile, the unit detects pressure oscillations generated by beat-to-beat pulsatile, which is used to determine the systolic and diastolic pressure, and also pulse rate.

The device also compares the longest and the shortest time intervals of detected pulse waves to mean time interval then calculates standard deviation. The device will display a warning signal with the reading to indicate the detection of irregular heartbeat when the difference of the time intervals is over 25%.

## ♥ Safety Information

The signs below might be in the user manual, labeling or other component. They are the requirement of standard and using.

|   |   |   |  |
|---|---|---|--|
|  | Symbol for "THE OPERATION GUIDE MUST BE READ"         |  | Symbol for "TYPE BF APPLIED PARTS"   |
|  | Symbol for "COMPLIES WITH MDD 93/42/EEC REQUIREMENTS" |  | Symbol for "ENVIRONMENT PROTECTION - Electrical waste products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice". In compliance with the Directive 2012/19/EC |
|  | Symbol for "MANUFACTURER"                             |   |  |
|  | Symbol for "SERIAL NUMBER"                            |   |  |
|  | Symbol for "DIRECT CURRENT"                           |  | Symbol for "Authorised Representative in the European Community"   |
|  | Symbol for "MANUFACTURE DATE"                         |  | Caution: These notes must be observed to prevent any damage to the device.   |

## ⚠ CAUTION

This device is intended for adult use only.

This device is intended for no-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on extremities other than the arm or for functions other than obtaining a blood pressure measurement.

Do not confuse self-monitoring with self-diagnosis. This unit allows you to monitor your blood pressure. Do not begin or end medical treatment without asking a physician for treatment advice. If you are taking medication, consult your physician to determine the most appropriate time to measure your blood pressure. Never change a prescribed medication without consulting your Physician.

When the device was used to measure patients who have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, the best result may occur with deviation. Please consult your physician about the result.

If the cuff pressure reaches 40 kPa (300 mmHg), the unit will automatically deflate. Should the cuff not deflate when pressures reaches 40 kPa (300 mmHg), detach the cuff from the arm and press the START-STOP button to stop inflation.

The equipment is not AP/APG equipment and not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.

The operator shall not touch output of batteries /adapter and the patient simultaneously.

To avoid measurement errors, please avoid the condition of strong electromagnetic field radiated interference signal or electrical fast transient/burst signal.

The user must check that the equipment functions safely and see that it is in proper working condition before being used.

This device is contraindicated for any female who may be suspected of, or is pregnant. Besides providing inaccurate readings, the affects of this device on the fetus are unknown.

Manufacturer will make available on request circuit diagrams, component parts list etc.

This unit is not suitable for continuous monitoring during medical emergencies or operations.

Otherwise, the patient's arm and fingers will become anaesthetic, swollen and even purple due to a lack of blood.

Please use the device under the environment which was provided in the user manual. Otherwise, the performance and lifetime of the device will be impacted and reduced.

During use, the patient will be in contact with the cuff. The materials of the cuff have been tested and found to comply with requirements of ISO 10993-5:2009 and ISO 10993-10:2010. It will not cause any potential sensitization or irritation reaction.

Please use ACCESSORIES and detachable partes specified/ authorised by MANUFACTURE.

Otherwise, it may cause damage to the unit or danger to the user/patients.

The device doesn't need to be calibrated within the two years of reliable service.


Please dispose of ACCESSORIES, detachable parts, and the ME EQUIPMENT according to the local guidelines.

If you have any problems with this device, such as setting up, maintaining or using, please contact the SERVICE PERSONNEL of Flaem. Don't open or repair the device by yourself.

Please report to Transtek if any unexpected operation or events occur.

Please use the soft cloth to clean the whole unit. Don't use any abrasive or volatile cleaners.

## ♥ LCD display signal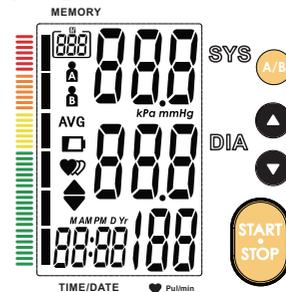


| SYMBOL      | DESCRIPTION                    | EXPLANATION  |
|-------------|--------------------------------|--|
| <b>SYS</b>  | Systolic blood pressure        | High blood pressure  |
| <b>DIA</b>  | Diastolic blood pressure       | Low blood pressure   |
| Pul/min     | Pulse display                  | Pulse in beats per minute  |
| ▼           | Deflation symbol               | The cuff is deflating.   |
| 000         | Memory                         | Indicate it is in the memory mode and which group of memory it is.             |
| <b>kPa</b>  | kPa                            | Measurement Unit of the blood pressure   |
| <b>mmHg</b> | mmHg                           | Measurement Unit of the blood pressure   |
| Lo +        | Low battery                    | Batteries are low and need to be replaced                                      |
|             | Irregular heartbeat            | Blood pressure monitor is detecting an irregular heartbeat during measurement. |
|             | Blood pressure level indicator | Indicate the blood pressure level  |
|             | Current Time                   | Year/Month/Day, Hour/Minute  |
|             | User A                         | Start measurement for User A   |
|             | User B                         | Start measurement for User B   |
|             | Heartbeat                      | Blood pressure monitor is detecting a heartbeat during measurement.            |
| <b>AVG</b>  | The average value              | The average value of the latest three records                                  |

## ♥ Monitor Components

Component list of  
pressure measuring  
system

- 1 Cuff
- 2 Air pipe
- 3 PCBA
- 4 Pump
- 5 Valve

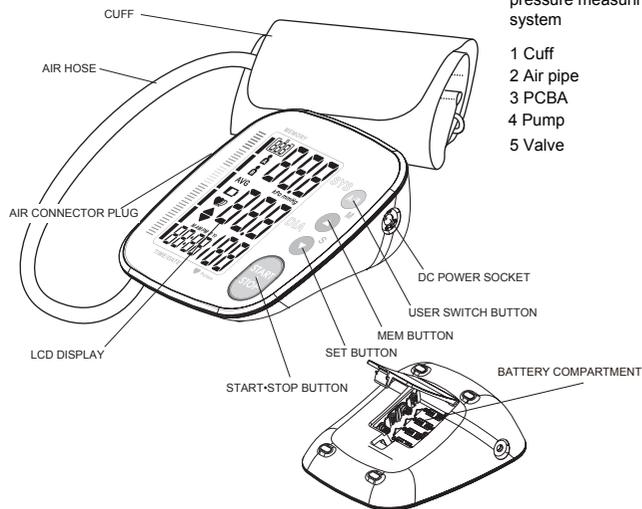## ♥ List

1. Blood Pressure Monitor  
(TMB-1490-C)

3. 4×AAA batteries

2. Cuff (Type BF applied part) (22cm~32cm)

(Please use Flaem  
authorized cuff. The size  
of the actual cuff please  
refer to the label on the  
attached cuff.)

4. User manual


5. Storage bag

## ♥ The Choice of Power Supply

1. Battery powered mode:  
6VDC 4×AAA batteries
2. AC adaptor powered mode:  
6V  $\bar{\bar{}}$  1A  
(Please only use the recommended AC  
adaptor model). (Not Included)

Please unplug the adaptor to depart from  
the using utility power.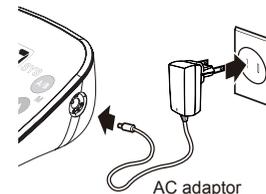


### ⚠ CAUTION

In order to get the best effect and protect your monitor, please use the  
the right batteries and special power adaptor which complies with CE  
safety standard.

## ♥ Installing and Replacing the Batteries

- Open the battery cover.
- Install the batteries by matching  
the correct polarity, as shown.
- Replace the cover.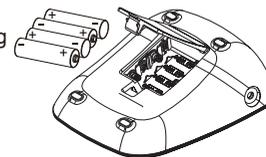

Replace the batteries whenever the below happen

- The  $\text{LO} + \square$  shows
- The display is dim
- The display does not light up


### ⚠ CAUTION

- Do not use new and used batteries together.
- Do not use different types of batteries together.
- Do not dispose the batteries in fire. Batteries may explode or leak.
- Remove batteries if the device is not likely to be used for some time.
- Worn batteries are harmful to the environment. Do not dispose with daily garbage.
- Remove the old batteries from the device following your local recycling guidelines.


## ♥ Setting Date, Time and Measurement Unit

It is important to set the clock before using your blood pressure monitor, so that a time stamp can be assigned to each record that is stored in the memory. (The setting range of the year :2014—2054 time format:12 H/24H)


- When the monitor is off, hold pressing "SET" for 3 seconds to enter the mode for year setting.  
Or when the monitor is off, press "SET" button shortly, it will display the time. Then hold pressing "SET" button to enter the mode for year setting.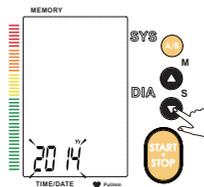


- Press the "MEM" to change the [YEAR]. Each press will increase the numeral by one in a cycling manner.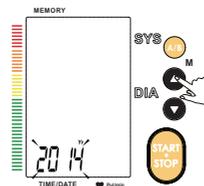



- When you get the right year, press "SET" to set down and turn to next step.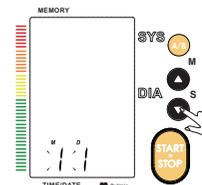


- Repeat steps 2 and 3 to set the [MONTH] and [DAY].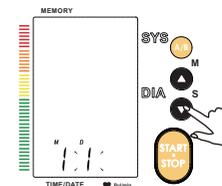


- Repeat steps 2 and 3 to set the [TIME FORMAT] between 12h and 24h.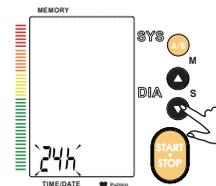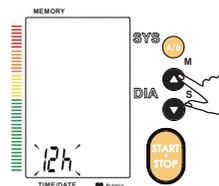


6. Repeat steps 2 and 3 to set the [HOUR] and [MINUTE].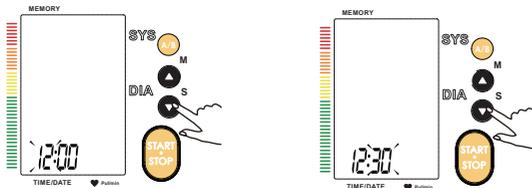

7. Repeat steps 2 and 3 to set the [UNIT].

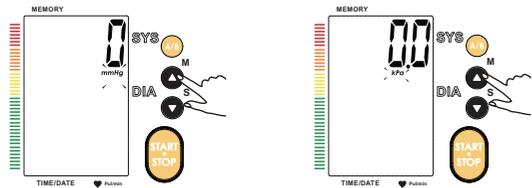
8. After the unit is set, the LCD will display "dOnE" first, then display all the settings you have done and then it will turn off.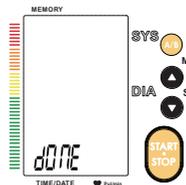

## ♥ Tie the cuff

1. Tie the cuff on your upper arm, then position the tube off-center toward the inner side of arm in line with the little finger. Or position the artery mark  $\phi$  over the main artery (on the inside of your arm). Note: Locate the main artery by pressing with 2 fingers approximately 2 cm above the bend of your elbow on the inside of your left arm. Identify where the pulse can be felt the strongest. This is your main artery.

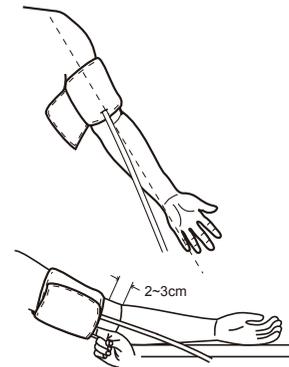2. The cuff should be snug but not too tight. You should be able to insert one finger between the cuff and your arm.

3. Sit comfortably with your tested arm resting on a flat surface.

4. Patients with Hypertension: The middle of the cuff should be at the level of the right atrium of the heart; Before starting measurement, please sit comfortably with legs uncrossed, feet flat on the floor, back and arm supported.

- Rest for 5 minutes before measuring.
- Wait at least 3 minutes between measurements. This allows your blood circulation to recover.
- For a meaningful comparison, try to measure under similar conditions. For example, take daily measurements at approximately the same time, position of upper arm, or as directed by a physician.

## ♥ Start the Measurement


- When the monitor is off, press "START-STOP" button to turn on the monitor, and it will finish the whole measurement and save the measurement data for the desired user. (Take User A for example.)

Notes: you can select user A or B by pressing User button during the process of measurement or when it is in the memory mode.

LCD display

Inflating and measuring.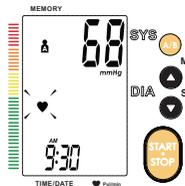


- Press the "START-STOP" to power off, otherwise it will turn off within 1 minute.


Tips: Maximum 250 records are both for User A and User B.

Adjust the zero.
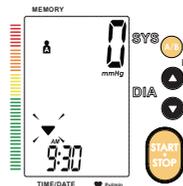



Display and save the measurement result.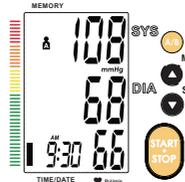

## ♥ Recall the Records

- When the monitor is off, please press the "MEM", it will display the latest record first when the records are less than three groups. When there are more than three groups, it will display the average value of the latest three records first.
- Press the "MEM" or "SET" to get the record you want. Press and hold the "MEM" button to look over ten groups of the historical records quickly.

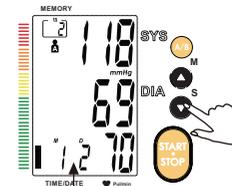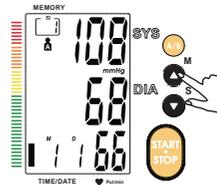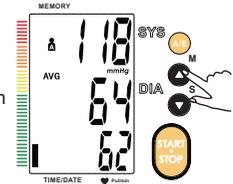The date and time of the record will be shown alternately.

- If you want to look over another user's data, press the User button to select the desired user. Then you can look over its historical records.


### ⚠ CAUTION

The most recent record (1) is shown first. Each new measurement is assigned to the first (1) record. All other records are pushed back one digit (e.g., 2 becomes 3, and so on), and the last record (250) is dropped from the list.

## ♥ Delete the Records

If you did not get the correct measurement, you can delete all results for the selected user by following steps below .

1. Hold pressing "SET" button for 3 seconds when the monitor is in the memory recall mode ,the flash display "dEL ALL" will show.

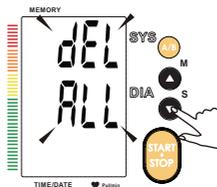
2. Press "SET" button to confirm deleting and the monitor will turn off.

Note: To exit out of delete mode without deleting any records, press "START-STOP" button before pressing "SET" button to confirm any delete commands.

3. If there is no record, press "MEM" button, the right display will show.

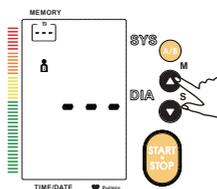## ♥ Tips for Measurement

Measurements may be inaccurate if taken in the following circumstances.

Within 1 hour after dinner or drinking

Immediate measurement after tea, coffee, smoking

Within 20 minutes after taking a bath

When talking or moving your fingers

In a very cold environment

When you want to discharge urine

## ♥ Maintenance

In order to get the best performance, please follow the instructions below.

Put in a dry place and avoid the sunshine

Avoid intense shaking and collisions

Using wet cloths to remove the dirt

Avoid touching water, clean it with a dry cloth in case.


Avoid dusty and unstable temperature environment

Do not attempt to clean the reusable cuff with water and never immerse the cuff in water.


## ♥ What are systolic pressure and diastolic pressure?

When ventricles contract and pump blood out of the heart, the blood pressure reaches its maximum value in the cycle, which is called systolic pressure. When the ventricles relax, the blood pressure reaches its minimum value in the cycle, which is called diastolic pressure.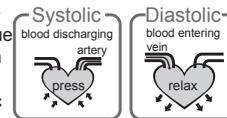

## ♥ What is the standard blood pressure classification?

The blood pressure classification published by World Health Organization (WHO) and International Society of Hypertension (ISH) in 1999 is as follows:

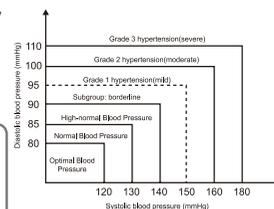### ⚠ CAUTION

Only a physician can tell your normal BP range. Please contact a physician if your measuring result falls out of the range.

Please note that only a physician can tell whether your blood pressure value has reached a dangerous point.

| Level | Optimal | Normal  | High-normal | Mild    | Moderate | Severe |
|-------|---------|---------|-------------|---------|----------|--------|
| SYS   | <120    | 120-129 | 130-139     | 140-159 | 160-179  | ≥180   |
| DIA   | <80     | 80-84   | 85-89       | 90-99   | 100-109  | ≥110   |

## ♥ Irregular Heartbeat Detector

An irregular heartbeat (IHB) is detected when a heartbeat rhythm varies while the unit is measuring the systolic and diastolic blood pressure. During each measurement, the monitor records the heartbeat intervals and calculates the average. If any average is larger than or equal to 25%, the irregular heartbeat symbol appears on the display when the measurement results appear.

### ⚠ CAUTION

The appearance of the IHB icon indicates that a pulse irregularity consistent with an irregular heart-beat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

### ♥ Why does my blood pressure fluctuate throughout the day?

1. Individual blood pressure varies multiple times everyday. It is also affected by the way you tie your cuff and your measurement position, so please take the measurement under the same conditions.
2. If the person takes medicine, the pressure will vary more.
3. Wait at least 3 minutes for another measurement.

### ♥ Why do I get a different blood pressure at home compared to the hospital?

The blood pressure is different even throughout the day due to weather, emotion, exercise etc. Also, there is the "white coat" effect, which means blood pressure usually increases in clinical settings.

What you need to pay attention to when you measure your blood pressure at home:

- If the cuff is tied properly.
- If the cuff is too tight or too loose.
- If the cuff is tied on the upper arm.
- If you feel anxious.
- Taking 2-3 deep breaths before beginning will be better for measuring.
- Advice: Relax yourself for 4-5 minutes until you calm down.

### ♥ Is the result the same if measuring on the right arm?

It is ok for both arms, but there will be some different results for different people. We suggest you measure the same arm every time.

This section includes a list of error messages and frequently asked questions for problems you may encounter with your blood pressure monitor. If the products not operating as you think it should, check here before arranging for servicing.

| PROBLEM                    | SYMPTOM   | CHECK THIS   | REMEDY  |
|----------------------------|---|--|---|
| <b>No power</b>            | Display will not light up.  | Batteries are exhausted.   | Replace with new batteries  |
|                            |   | Batteries are inserted incorrectly.  | Insert the batteries correctly  |
|                            |   | AC adaptor is inserted incorrectly.  | Insert the AC adaptor tightly   |
| <b>Low batteries</b>       | Display is dim or show  | Batteries are low.   | Replace with new batteries  |
| <b>Error message</b>       | E 1 shows   | The cuff is not secure.  | Refasten the cuff and then measure again.                             |
|                            | E 2 shows   | The cuff is very tight   | Readjust the cuff ,not too loose or too tight and then measure again. |
|                            | E 3 shows   | The pressure of the cuff is excess.  | Relax for a moment and then measure again.                            |
|                            | E10 or E11 shows  | The monitor detected motion,talking or the pulse is too poor while measuring.  | Relax for a moment and then measure again.                            |
|                            | E20 shows   | The measurement process does not detect the pulse signal.  | Loosen the clothing on the arm and then measure again                 |
|                            | E21 shows   | The treatment of the measurement failed.   | Relax for a moment and then measure again.                            |
| EExx,shows on the display. | A calibration error occurred (XX can be some numeric characters,such as 01, 02 and so on .                | Retake the measurement. If the problem persists, contact the retailer or our customer service department for further assistance.Refer to the warranty for contact information and return instructions. |   |

|   |  |
|---|--|
| <b>Power supply</b>                           | Battery powered mode: 6VDC 4×AAA batteries<br>AC adaptor powered mode: 6V $\bar{\text{~}}$ 1A<br>(Please only use the recommended AC adaptor model.)(Not Included) |
| <b>Display mode</b>                           | Digital LCD With Blue Backlight V.A.:Approx.60mm×92mm  |
| <b>Measurement mode</b>                       | Oscillographic testing mode  |
| <b>Measurement range</b>                      | Rated cuff pressure:<br>0kPa~39.9kPa (0mmHg~299mmHg)<br>Measurement pressure: 5.3kPa~30.7kPa<br>(40mmHg~230mmHg)<br>Pulse value: (40-199) beat/minute              |
| <b>Accuracy</b>                               | Pressure:<br>5°C~40°C within±0.4kPa(3mmHg)<br>Pulse value:±5%  |
| <b>Normal working condition</b>               | Temperature:5°C to 40°C<br>Relative humidity: ≤85%RH<br>Atmospheric pressure: 86kPa to 106kPa  |
| <b>Storage &amp; transportation condition</b> | Temperature:-20°C ~60°C<br>Relative Humidity: 10%RH~93%RH<br>Atmospheric Pressure: 50kPa~106 kPa   |
| <b>Measurement perimeter of the upper arm</b> | About 22cm~32cm  |
| <b>Net Weight</b>                             | Approx.250g(Excluding the dry cells)   |
| <b>External dimensions</b>                    | Approx.140mm×130mm×49.7mm  |
| <b>Attachment</b>                             | 4×AAA batteries,user manual,storage bag  |
| <b>Mode of operation</b>                      | Continuous operation   |
| <b>Degree of protection</b>                   | Type BF applied part   |
| <b>Protection against ingress of water</b>    | IP21 It means the device could protected against solid foreign objects of 12.5mm and greater, and protect against vertically falling water drops.                  |
| <b>Device Classification</b>                  | Battery Powered Mode: Internally Powered ME Equipment<br>AC Adaptor Powered Mode: Class II ME Equipment  |
| <b>Software Version</b>                       | V01  |

WARNING: No modification of this equipment is allowed.

## ♥ Authorized Component

1. please use the Flaem authorized adapter.(Not Included)

Adapter  
Type: UE08WCP-060100SPA  
Input: 100~240V, 50~60Hz,400mA  
Output: 6V  $\bar{\text{~}}$  1A  
(Conforms to UL certificate)

2. Storage bag

## ♥ Contact Information

For more information about our products, please visit [www.transtek.cn](http://www.transtek.cn).you can get customer service, usual problems and customer download, transtek will serve you anytime.

**Manufactured by:** Guangdong Transtek Medical Electronics Co., Ltd.  
**Company:** Guangdong Transtek Medical Electronics Co., Ltd.  
**Address:** Zone A, No.105 ,Dongli Road, Torch Development District, Zhongshan,528437,Guangdong,China

**Authorized European Representative:**  
**Company:** MDSS - Medical Device Safety Service GmbH  
**Address:** Schiffgraben 41, 30175 Hannover, Germany

## ♥ Complied European Standards List

|                                 |   |
|---------------------------------|---|
| Risk management                 | EN ISO 14971:2012 Medical devices - Application of risk management to medical devices   |
| Labeling                        | EN 980:2008 Symbols for use in the labelling of medical devices   |
| User manual                     | EN 1041:2008 Information supplied by the manufacturer of medical devices  |
| General Requirements for Safety | EN 60601-1:2006 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance<br>EN 60601-1-11:2010 Medical electrical equipment - Part 1-11: General requirements for basic safety and essential performance - Collateral standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment<br>IEC 80601-2-30:2009 Medical electrical equipment - Part 2-30: Particular requirements for the basic safety and essential performance of automated noninvasive sphygmomanometers |
| Electromagnetic compatibility   | EN 60601-1-2:2007 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic compatibility - Requirements and tests  |
| Performance requirements        | EN ISO 81060-1:2012 Non-invasive sphygmomanometers - Part 1: Requirements and test methods for non-automated measurement type<br>EN 1060-3:1997+A2:2009 Non-invasive sphygmomanometers - Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems   |
| Clinical investigation          | EN 1060-4:2004 Non-invasive sphygmomanometers - Part 4: Test procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers  |
| Usability                       | EN 60601-1-6:2010 Medical electrical equipment - Part 1-6: General requirements for basic safety and essential performance - Collateral standard: Usability<br>EN 62366:2008 Medical devices - Application of usability engineering to medical devices  |
| Software life-cycle processes   | EN 62304:2006/AC: 2008 Medical device software - Software life cycle processes  |

## ♥ EMC Guidance

1) This equipment needs to be installed and put into service in accordance with the information provided in the user manual;

2) Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be kept at least a distance  $d=3,3m$  away from the equipment.

(Note: As indicated in Table 6 of IEC 60601-1-2:2007 for ME EQUIPMENT, a typical cell phone with a maximum output power of 2 W yields  $d=3, 3m$  at an IMMUNITY LEVEL of 3V/m)