FLAEM® **Olly Vap** Warm Steam Vaporizer

Medical Devices

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We are pleased you have purchased our product and we thank you for your trust in us. Please note that the full range of Flaem products is available on the website www.flaemnuova.it. Read these instructions carefully before using the device, as they provide important information on the use, safety and maintenance of the humidifier. Store carefully for future reference. This household device is used to humidify and perfume a domestic environment. Use the device only for the purposes indicated in these instructions. Comply with the important safety warnings. Suitably remove the packaging.

NOMENCLATURE

Refer to the page instructions 1-2

- | | |
|--|---|
| <ul style="list-style-type: none"> 1 - Tank <ul style="list-style-type: none"> 1.1 - MAX. level indicator 1.2 - IN-OUT slits 1.3 - Water drain hole | <ul style="list-style-type: none"> 2 - Heating unit <ul style="list-style-type: none"> 2.1 - Power cable 2.2 - Essence container 2.3 - Flow shutter 2.4 - IN-OUT fins 2.5 - Heating unit cap 2.6 - Resistance 2.7 - Night light - ON/OFF |
|--|---|

IMPORTANT NOTICES

This household device is intended **ONLY** for domestic use. It is not intended for commercial or industrial use and it must not be used outdoors. Use the device in compliance with the instructions contained in this manual. Any other uses may generate fire or electric shock.

- Before using the product for the first time, and periodically during its life-time, check the integrity of the device structure and of the power cable to make sure there is no damage. In the event of damage, do not plug in the cable and immediately take the product to an authorised FLAEM service centre or to your trusted dealer.
- If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.
- **CAUTION:** In order to avoid a hazard due to inadvertent resetting of the thermal cut-out, this appliance must not be supplied through an external switching device, such as timer, or connected to a circuit that is regularly switched on and off by the utility.
- In order to prevent burns, use the humidifier in the presence of an adult who has read and is familiar with the instructions. Do not use near children, non-self-sufficient individuals, and pets to prevent the risk of burns. **ATTENTION: HOT STEAM!**
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the

hazards involved. Children shall not play with the appliance cleaning and user maintenance shall not be made by children without supervision.

- Keep the packaging material, such as plastic bags, out of the reach of children.
- Do not drag or pull the device by the power cable. Do not wind the power cable around the device. Do not touch the power cable with wet hands. Do not move and switch the device on if you are barefoot.
- Do not use the device near bathtubs, sinks, large water containers, or expose it to humidity or rain. Do not entirely immerse the heating unit (2) in water. In the event of immersion, immediately disconnect the plug and then remove the device from the water. Do not use it again until it has been inspected or repaired at an authorised service centre.
- Never use adapters for power supply values other than those indicated on the bottom of the device (plate data). Do not power the device with direct current (DC).
- Position the device on a flat, stable, and water-resistant surface. Do not place it directly on furniture, carpets, or parquet. Do not expose the device to direct sunlight or external weather agents. Do not direct the steam jet directly towards furniture, walls, or floors. Do not move or manoeuvre the device when it is running. Pay attention not to trip over it.
- Do not obstruct or block the air vents. Keep the ventilation inlets free from dust. Ensure that no object falls or is inserted into the vents.
- Devices equipped with a water tank may be subject to water leaks. Before each use, make sure that the tank is not damaged or cracked.
- Only use RUNNING WATER.
- Do not add medicines to the water tank or to the aromatic essence container. Never add aromatic oils to the water tank.
- All the parts of the device must be installed as indicated in the first start-up description.
- Unplug the appliance during filling and cleaning.
- Wait for the device to cool down and disconnect the plug from the socket before cleaning or refilling the tank with water. Throw out the residual water and rinse before refilling the tank.
- Make sure that there is no water left in the device and that it is preferably dry before storage.
- Periodically perform "Maintenance and Cleaning" operations.
- The Manufacturer, the Vendor and the Importer shall be held responsible for safety, reliability and performance only if: a) the device is used in compliance with the instructions for use b) the wiring where the device is being used is in compliance with current laws.
- Repairs must be carried out by an authorised service centre. Unauthorised repairs void and null the warranty and may pose a potential electric risk.

- Be aware that high humidity levels may encourage the growth of biological organisms in the environment.
- Do not permit the area around the humidifier to become damp or wet. If dampness occurs, turn the output of the humidifier down. If the humidifier output volume cannot be turned down, use the humidifier intermittently. Do not allow absorbent materials, such as carpeting, curtains, drapes, or tablecloths, to become damp.
- Unplug the appliance during filling and cleaning.
- Never leave water in the reservoir when the appliance is not in use.
- Empty and clean the humidifier before storage. Clean the humidifier before next use.
- Keep the appliance out of the reach of children. Steam and boiling water may cause burns.

This text is intended for users outside the European Union:

- This device is not intended to be used by individuals (including children) with impaired physical, sensory or mental capacities or lacking experience and/or knowledge, unless they are monitored or they have received instructions on using the device by the person responsible for their safety.
- Children must be monitored to make sure they do not play with the device.

INSTRUCTIONS FOR USE

1. Always disconnect the power cable (2.1) before topping up the device.
2. To remove the heating unit (2) see (fig. B), press it downwards (B1), simultaneously turning it anti-clockwise (B2) and then remove it (B3) from the tank (1).
3. Fill the tank (1) with running water up to the Maximum level indicator on the tank (1.1) (Max. 4.5 litres).
4. Insert heating unit 2 inside tank 1.
 - 4.1 Ensure that the fins (2.4) on the heating unit correspond with the slits (1.2) on the tank.
 - 4.2 Rotate clockwise until it comes to a complete stop.
5. **WARNING: to prevent damage to the humidifier, DO NOT POUR the aromatic essences directly into the water tank (1).** Pour the aromatic essences into the essence container (2.2) (fig. E). Pay attention to prevent over flows, and fill the container up to the level indicated (2.2).
6. Insert the plug into the socket corresponding to the device voltage in order to switch the night light on (2.7).
7. Adjust the steam flow by moving the steam flow shutter (2.3) forwards and backwards (fig. F).
8. When there is no water in the tank, the ON/OFF night light (2.7) switches off. Unplug it from the socket and wait 10 minutes for it to cool down before performing a new operation from (point 1).

NOTES: During operation, you may hear a gentle bubbling sound.

- **The humidifier also includes a safety switch that switches the unit off when the tank runs out of water or if the unit runs in dry operation.**
- **To restart the humidifier, disconnect the plug and wait 5 minutes for it to cool down. Top up with water and restart from (point 4).**

MAINTENANCE AND CLEANING

Always disconnect the power cable (2.1) and wait 30 minutes for the device to cool down, before cleaning. For ex-tended periods of non-use, store the device, only when it is clean and dry, inside its box.

ATTENTION: Presence of excessive scale may be found on the resistance due to lack of or poor maintenance and cleaning operations. This may be the cause of malfunctions on the ON/OFF night light (2.7) even when the tank is full.

Daily cleaning/maintenance:

- 1- Remove the heating unit (2) by rotating it and pulling it upwards (Fig. B).
- 2- Fully drain the tank. Ensure that water drains from the hole (1.3) (Fig. G).
- 3- Before refilling the tank, clean and rinse a couple of times with abundant water. If you do not use it immediately, drain and dry with a cloth.

Warning: DO NOT use abrasive detergents or brushes. DO NOT leave water in the water tank when the device is not running. This prevents bacteria from forming in the tank.

Weekly cleaning/ maintenance:

- 1- On a weekly basis, clean the humidifier as described in "Daily cleaning".
- 2- Clean the heating unit. Immerse the heating unit into 10 cm of solution containing 50% water and 50% white vinegar for 10 minutes. Lastly, rinse abundantly with hot water (approximately 40 °C) see (Fig. H).

End of season cleaning/maintenance:

- 1- Clean the humidifier as described in "Weekly cleaning".
- 2- Descale the resistance (2.6), by removing the cap of the heating unit (2.5) rotating it anti-clockwise (fig. I). Use a regular sponge to remove the scale (fig. L).

TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
The ON/OFF night light (2.7) fails to switch on and the humidifier fails to produce steam.	The device is not connected to the socket.	Insert the plug into the socket.
	No power supply reaches the socket.	Check the operation of the socket with another device.
	No water inside the tank.	Fill the tank with water.
	Presence of mineral deposits on the resistance.	Perform weekly and "end of season" cleaning/maintenance.
	Circuit breaker intervention.	Unplug and wait 5 minutes for it to cool down.
Water leaks from the tank.	Excessive water inside the tank.	Drain and lower the water to the MAX. level (1.1).

TECHNICAL SPECIFICATIONS

Model:	P0413ED
The device is suitable for spaces up to	25 m ²
Rated voltage:	220-240V~ 50Hz 240-260W
Operating temperature:	10 to 40 °C (50 °F - 104 °F)
Dimensions (cm):	27 (W) x 29 (D) x 22 (H)
Weight (dry):	1.3 Kg
Tank capacity:	approx. 4.5 litres
Autonomy:	up to 14 hours

Marking:

Approvals:

SYMBOLS

Read the operating instructions

CAUTION:
Hot water vapour

Alternating current

In accordance with directive 2014/35/EU and subsequent modifications

ELECTROMAGNETIC COMPATIBILITY:

The Room Steam Vaporiser is designed to meet the requirements currently established for electromagnetic compatibility. However, if you suspect that the operation of this appliance is interfering with the normal operation of your television set, radio or other electrical appliance, try placing it in a different position until the interference stops, or connect the appliance to a different power socket.

DISPOSAL

This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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FLAEM NUOVA S.p.A. non risponde di ulteriori estensioni del periodo di garanzia assicurate da terzi.
FLAEM NUOVA S.p.A.