over 40 years  
MADE IN ITALY

# Tiralatte

by **FLAEM**®

**I** MANUALE ISTRUZIONI D'USO  
**GB** INSTRUCTIONS FOR USE MANUAL  
**F** MODE D'EMPLOI  
**NL** GEBRUIKSAANWIJZING

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**GB**

### SUCTION PUMP

*We compliment you on your purchase and thank you for your confidence in us.  
Visit our Internet site [www.flaem.it](http://www.flaem.it) to view the whole range of Flaem products.*

This **TIRALATTE** (5) is an appliance featuring special characteristics for milk suction. The unit is equipped with a vacuum regulation knob (4), a vacuum gauge (2) and an anti-spill vessel (6) with a protection device preventing liquid from entering the TIRALATTE through a float shut-off located in the vessel lid.

This unit is handy, easy to use, reliable, resistant and silent and does not need lubrication.

The **TIRALATTE** is supplied with the following FLAEM accessories: Anti-spill vessel (6) with protection device, Short connection tube (12), Antibacterial filter (where provided) (13), Long connection tube (10), 250 ml feeding bottle (8), Breast shield (9) and Bottle cap suitable for milk preservation.

**N.B.: Use only FLAEM original accessories**

#### **IMPORTANT SAFEGUARDS**

**For home use only.**

**It must be operated as indicated in the instruction manual. It is important that the user reads and understands the information provided for use and care of the unit. Please contact your dealer with any questions.**

The manufacturer takes every step necessary to ensure utmost product quality and safety. However, as for all electrical appliances, you should follow some basic safety rules to avoid creating a hazard for people or property. Before using the appliance for the first time, and periodically throughout its life, check the power supply cord for any damage. Should you detect any damage, immediately unplug the appliance and take it to an authorised service centre or your dealer.

- Persons with disabilities must always use the device under the close supervision of an adult who has read this manual.
- Never use adaptors for power supplies different from the one indicated on the bottom of the appliance. Keep the cord away from heated surfaces.
-  Do not handle the plug with wet hands. Never use the appliance while having a bath or shower. Never immerse the appliance in water: should this occur accidentally, immediately pull out the plug. Do not remove or touch the appliance under water until you have pulled out the plug. Do not reuse the appliance after taking it out of the water (send it immediately to an authorised service centre).
- The casing of this appliance is not waterproof.
- Always unplug the appliance immediately after use.
- A safety fuse located inside the appliance can easily be inspected in case of fault. Always unplug the appliance before this operation. Servicing may be performed by authorised personnel only. Any unauthorised repairs will invalidate the warranty.
- Make sure that all connections are properly made and that the anti-spill vessel and the bottle are properly closed in order to prevent suction leaks.
- Be careful not to knock the anti-spill vessel over while connected to the operating appliance as liquid might be aspirated back into the appliance and damage the pump. Should this occur, immediately turn the appliance off, empty and clean the vessel and the bottle (send it to an authorised service center).
- Do not use the overflow container (6) to extract and store milk. Use only a 250 ml bottle (8) for these purposes.
- When the protection device triggers, suction is stopped; empty the vessel and carry out the cleaning procedures.
- The disposable antibacterial filter (where provided) must be replaced within 24 hours of first use

## **OPERATING INSTRUCTIONS**

**Before each use, the accessories should be carefully checked so as to make sure that there are no dust, deposits, lumps or liquid substances inside the connection tubes, the anti-spill vessel, the bottle and their lids. Accurately clean or disinfect these parts strictly following the instructions for "CLEANING AND DISINFECTION".**

1. Connect the antibacterial filter (13) and/or the air inlet (3) of the appliance to the "A" outlet (OUT) (11) of the anti-spill vessel by means of the short tube (12). By means of the long tube (10) connect the "B" inlet (IN) (7) of the anti-spill vessel (6) to the breast shield (9) screwed on the bottle.
2. Plug the power cord into a power socket corresponding to the voltage of the appliance.
3. Place the appliance on a level surface making sure not to obstruct the air slots.
4. Operate the appliance by turning the switch on (1).
5. The desired level of vacuum (bar) can be set by using the vacuum regulation knob (4). A higher vacuum level is obtained by turning the knob in the (+) direction; a lower vacuum level is obtained by turning the knob in the (-) direction. These values can be read on the vacuum gauge (2).
6. Place the breast shield (9) over your breast. Place your finger on the hole C of the breast shield (9) and start operation. By means of pressing discontinuously with your finger, pulsed suction can be achieved.
7. When the operation is finished, turn off the appliance. Before restarting the appliance make sure that it has cooled down to room temperature.
8. Unplug the appliance; empty and clean the feeding bottle and the connection tubes.

## **CLEANING AND DISINFECTION**

- Disconnect the long tube (10) from both the bottle and the anti-spill vessel. Unscrew the breast shield; empty and clean the bottle. Disconnect the short tube (12) from both the anti-spill vessel and the appliance (or the antibacterial filter where provided). In order to clean the accessories immediately after each use, it is sufficient to rinse the disassembled parts under running water. To remove any impurities use the bottle brush (19) and the teat brush (18).

As required by Regulation (EU) 321/11 of the European Commission, all components of this Kit, made for contact with milk, are made of materials that DO NOT contain BISPHENOL A.

Disinfection of these components must only be performed with common chloro-oxidising disinfectants available on the market (Milton, Amuchina, etc.) in accordance with the directions indicated on the packaging of the disinfectant itself.

**Do not disinfect or subject components to: boiling, steaming or sources of heat in general.**

- The disposable antibacterial filter (where provided) must be replaced within 24 hours of first use.
- Never wash the appliance under running water or soak in water; wipe it only with a damp cloth using mild detergents. Do not use abrasive cleaners.
- Reassemble the bottle, the anti-spill vessel and the connection tubes by following the "INSTRUCTIONS FOR USE".

## **ACCESSORIES - TECHNICAL FEATURES**

- 125 ml anti-spill vessel
- Ø 4 X 6 mm connection tube, length 160 mm
- Ø 4 X 6 mm connection tube, length 1000 mm
- 250 ml feeding bottle
- Breast shield
- Ring for bottle teat
- Ring cap
- Teat brush
- Bottle cap
- Teat
- Ring lid

## **Microbial contamination**

The accessories, bottle, anti-spill vessel and connection tubes are for single person use only, to prevent cross infections.

**Cross reactions**

The materials used in manufacture of the appliance that come into contact with body-secretions are thermoplastic polymers. These polymers have been selected due to their excellent chemical stability and resistance. The possibility of a chemical interaction, however, cannot be excluded. The following precautions are therefore recommended:

- a) Only use a 250 ml bottle for milk extraction and storage.
- b) Always avoid prolonged contact of liquid with bottles or tubes; wash or disinfected immediately after use.
- c) If unexpected situations occur, for example, the accessories crack or become soft, stop operation immediately and substitute the accessories used. Contact an authorised dealer for service specifying how suctioning was being performed.

**N.B.: Use only FLAEM original accessories**

**APPLIANCE TECHNICAL FEATURES**

Piston-driven rotary compressor with thermal cut-out, lubrication-free resin.

Voltage:	230V- 50Hz 140VA
Fuse:	T 2A - 250V
Adjustable vacuum level:	from 0 to -0.4 bar
Compressor air output:	approx. 11 l/min
Dimensions:	approx. 22x17x17 cm
Weight:	2.300 kg
Sound level (at 1 m):	60 dB (A)
Continuous use	
Marking:	

**SYMBOLS**

-  Class II appliance
-  Important: check the operating instructions
-  Switch "OFF"
-  Switch "ON"
-  Alternate current
-  Never use the appliance while having a bath or shower
-  Compliant with Directive 2006/95 CE 2004/108 CE - EN 14350-2
-  Latex Free

-  -The Manufacturer, the Retailer and the Importer are responsible for safety, reliability and performance only if: a) the appliance is used in accordance with the operating instructions; b) the electrical system of the premises where the appliance is used is in conformity with current laws.
- Servicing may be performed by authorised personnel only. Unauthorised repairs will invalidate the warranty and may represent a hazard to the user.

 **Disposal:** In conformity with Directive 2002/96/EC, the symbol shown on the appliance to be disposed of indicates that this is considered as waste and must therefore undergo "sorted waste collection". The user will therefore have to take the above waste (or have it taken) to pre-sorted waste collection centres organised by the local authorities, or else return it to the dealer at the time of purchasing a new appliance of the same type. The pre-sorted collection of the waste and subsequent treatment, recovery and disposal operations favour the production of appliances made from recycled materials and limit negative effects on the environment and on health caused by any incorrect waste management. The unlawful disposal of the product by the user could result in administrative fines as provided by article 50 and subsequent modifications of the Italian Legislative Decree no. 22/1997.

**Electromagnetic compatibility:** this aspirator is designed to meet the current requirements for electromagnetic compatibility. However if you suspect that the appliance is interfering with the operation of your TV, radio or other electrical appliance, try relocating the appliance until the interference ceases. Alternatively try plugging the appliance into a different power socket.

The Manufacturer reserves the right to make technical and functional changes to this product without prior notice.

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© Copyright by FLAEM NUOVA S.p.A., 2010
- FLAEM NUOVA S.p.A. non risponde di ulteriori estensioni del periodo di garanzia assicurate da terzi.
Fabbricante / Manufacturer: FLAEM NUOVA S.p.A. Elettromedicali - Elettrodomestici