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HEAT PAD INSTRUCTIONS FOR USE

Dear customer, thanks you for purchasing this product. We are certain you will appreciate the quality and reliability of this appliance, designed and manufactured with customer satisfaction in mind. These instructions for use have been drawn up in accordance with European standard EN 82079.

IMPORTANT INSTRUCTIONS KEEP FOR FUTURE USE.

ATTENTION!

Instructions and warnings for a safe use.

Please read the operating instructions and safety warnings in particular before using the appliance and follow these at all times. Keep this manual and illustrative guide with the appliance for future consultation. Should you pass the appliance on to another user, make sure to also include this documentation.

i Should certain parts of this user manual be difficult to understand or should doubts arise, contact the company before using the product, at the address indicated on the last page.

CONTENTS

SAFETY WARNINGS	page 6	CLEANING AND MAINTENANCE	page 9
SYMBOLS	page 8	DISPOSAL	page 9
DESCRIPTION OF THE APPLIANCE	page 8	ASSISTANCE AND WARRANTY	page 9
USING THE HEAT PAD	page 8	ILLUSTRATIVE GUIDE	page 1
USE	page 8	TECHNICAL DATA	page 1

SAFETY WARNINGS

- After having removed the appliance from its packaging, verify its integrity based on the drawing and check for any damage caused during transport. If in doubt, do not use the appliance and contact the authorised service centre.
- Do not let children play with the packaging! Keep the plastic wrapping out of the reach of children; risk of suffocation!
- Before connecting the appliance, check that the mains voltage data reported on the identification plate corresponds to that of the available electric mains. The identification plate is attached to the appliance.
- This appliance is intended exclusively as a heat pad for home

- use. Any other use is improper and potentially dangerous.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
 - Do not use the appliance for ill people, for infants or for persons who are insensitive to heat.
 - This appliance must not be used by children aged from 3 years and above if the control devices have not been set beforehand by a parent or guardian or if the child has not been properly instructed on how to use the device safely.
 -  **WARNING** – NOT suitable for children under the age of 3. Children under the age of 3 must not use the appliance as they are unable to react in the event of overheating.
 -  **DO NOT** activate the temperature control with wet hands.
 -  **DO NOT** pull the power cord or the appliance itself to remove the plug from the socket.
 - Always disconnect the appliance from the mains before cleaning or performing maintenance and when the appliance is not in use.
 - In case of a fault or malfunctioning of the appliance, switch it off and do not tamper with it. If the appliance needs to be repaired, contact an authorised technical service centre.
 - If the power cord is damaged, it must be replaced by an authorised technical service centre, so as to prevent all risks.
 - The appliance must be used solely with the types of control indicated on the label.
 - Inspect the appliance frequently to check for signs of wear and tear. If such signs are found or if the appliance has been used incorrectly, contact an authorised technical assistance centre before continuing to use it.
 - Persons with pacemakers must not use the appliance.
 - This appliance is not intended for medical use inside of hospitals.
 -  **DO NOT** use the appliance if it is wet.
 -  **DO NOT** use the appliance to keep animals warm.
 -  **DO NOT** leave the appliance running while you sleep.
 -  **DO NOT INSERT PINS OR NEEDLES.**
 - Application of the appliance for a long time at high temperature can burn the skin.

SYMBOLS

	Protection class II.		Prohibited actions.
	Warning.		Temperature selector.
	ON/OFF selector.		STAND-BY.

DESCRIPTION OF THE APPLIANCE

- | | | |
|------------------------|-------------------------|-------------------|
| 1 Temperature display. | 3 Temperature selector. | 5 Control socket. |
| 2 Stand-by Display. | 4 ON/OFF selector. | 6 Plug. |

USING THE HEATING PAD

The heat pad is designed to warm the human body. It allows for targeted application of heat.

The heat pad can be placed on parts of the body, such as: neck, shoulders, back, abdomen and joints, depending on the relative model.

USE

CONNECTING AND REMOVING THE TEMPERATURE ADJUSTMENT CONTROL

ATTENTION!

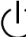
The heat pad can only be used with the control supplied. The relative model is specified on the label sewn on the panel.

- Insert the control socket (5) into the socket (6) to connect the temperature adjustment control to the heat pad.
- Insert the plug in the socket.
- Place the heat pad on the part of the body that is to be heated.

SWITCHING THE APPLIANCE ON AND SELECTING THE TEMPERATURE

- Press the ON/OFF  (4) selector to switch the heat pad on.
- The display (1) will light up and the temperature will be automatically set to position 5.
- Select the desired temperature (from 1 to 5) by pressing the temperature selector  (3) until the relative number lights up on the temperature display (1).
- Select temperature from 1 (min) to 3 (max) for continuous use.

AUTOMATIC SHUTDOWN

- The heat pad is provided with an automatic switch-off system. The stand-by display  (2) lights up after 3 operating hours to indicate that the product has switched off automatically.

- Switch the appliance back on by pressing the ON/OFF selector (4) and select the desired temperature once again.

SWITCH-OFF

- Switch the heat pad off by pressing the ON/OFF selector (4) when it is no longer used.
- Remove the plug from the socket.

CLEANING AND MAINTENANCE

The heat pad is washable. Refer to the washing symbols on the heat pad label for the washing instructions.

 Wash at 30°C.	 Do not bleach.
 Tumble-dry at less than 40°C.	 Do not iron.
 Do not dry-clean.	

WASHING INSTRUCTIONS

Always disconnect the plug and the control socket (5) from the socket (6) before cleaning.

MACHINE WASHING

- The heat pad can be machine washed at 30°C with a normal cycle.
- Only the gentle spin cycle is to be used. DO NOT set an additional spin cycle.

DRYING

- Set the dryer at a low temperature.
- Do not apply pegs on the electrical parts when hanging the heat pad to dry in order not to damage them.
- DO NOT use the heat pad until it is completely dry.

DISPOSAL

The product is packaged in recyclable materials. Dispose of it in compliance with the environmental protection standards.

The product must be disposed of appropriately, in compliance with European standard 2002/96/EC. The recyclable materials contained in the appliance must be recovered so as to prevent environmental pollution. For further information, contact the local disposal body or the appliance dealer.

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CELLULAR HOSPITAL S.A.S.