IT Aspiratore Nasale per Bambini  
EN Nasal Aspirator for Children  
FR Aspirateur nasal pour les enfants  
NL Nasale Aspirator voor kinderen  
DE Nasensauger für Kinder  
ES Aspirador nasal para niños  
RU Носовой аспиратор для детей  
PL Aspirator do nosa dla dzieci  
KO 영유아용 콧물흡인기- 베이비 코푸리  
EL Ρινική αναρρόφησης για τα παιδιά  
TR Çocuklar için Burun Aspiratörü  
ZH 儿童鼻吸气

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## EN NASAL ASPIRATOR FOR CHILDREN

We are grateful for your purchase.

**INTENDED USE:** Medical device for manual mucus suction.

The **BABY NASAL ASPIRATOR Baby Flaem** is a patented device, practical, easy to use and hygienic. It actually only requires the simple action of one hand and does not require any mouth aspiration by mum as is the usual case with other aspirators found on the market, thus avoiding possible microbiological contamination.

**INDICATIONS FOR USE:** Prevention of diseases caused by poor nasal hygiene. Mucus is sucked out of the patient's nose through mechanical action.

### WHY SHOULD EXCESS MUCUS BE ASPIRATED?

Good breathing is the first defense which a baby has available to protect its wellbeing and curb infections. As a matter of fact, **the babies** under 1 year old, **breathe exclusively via the nose**. So, if the nose is blocked, it is difficult for the baby to breathe, having problems sleeping and eating; it can also at times refuse to suckle and starts crying very often. Considering that **up to when it is 3 years old, the child is not able to blow its nose on its own**, mucus and entrapped bacteria, viruses and dust particles get clogged in the nasal cavities. There are two methods which are mostly advised to unblock the nose: **a good wash and the removal of mucus**; as for washing, we advise you to use the Flaem nasal rinses and for mucus removal, the baby nasal aspirator.

**INTENDED USERS:** Patients, health professionals. Adults responsible for the safety of patients with conditions described in the paragraph "Target patient group".

### IMPORTANT WARNINGS

**TARGET PATIENT GROUP:** Adults, children of all ages. Read the user manual carefully before using the device. For safety reasons, make sure an adult is present when the device is used by children of any age or persons with limited abilities (e.g. physical, mental or sensory). The evaluation of the use of the device on particular types of patients such as newborns, incapacitated subjects or with limited physical abilities is referred to the medical staff.

**OPERATING ENVIRONMENT:** This is a Medical device for home use. Follow the cleaning instructions carefully before each use. Read these instructions carefully and keep for future reference. Only use the device as described in this leaflet. It is important to read and understand the instructions for use. Contact your vendor or support centre for further clarification.

**Lifespan:** Expected Device Lifetime is 1 year. Use the device only as described in the intended use, any different uses are to be considered improper and can be dangerous. The manufacturer is not responsible for any improper use.

**Chokingrisk:** - Some components of the appliance are so small that they can be swallowed by children. Keep the device out of the reach of children. The manufacturer must be contacted to communicate problems and / or unexpected events relating to operation and, if necessary, for explanations relating to use and / or maintenance / cleaning.

### NOMENCLATURE (ref. Fig. C)

1. Soft Ergonomically-shaped Tip (2 per pack)

2. Body with external Fins (2A) allowing an easy and safe grip, and Mucus collection chamber (2B)

3. Spring

4. Plunger

5. Sealing ring

### CLEANING AND DISINFECTION

Before each use, wash your hands thoroughly and clean the **Aspirator** as follows: open the **Nasal Aspirator** by turning the **Sealing Ring** (5) (Picture D), paying attention to the **Spring** (3) thrust.

**Cleaning:** Wash components with warm drinking water (approximately 40°) with non-abrasive mild dish detergent or directly in the dishwasher with hot cycle.

**Disinfection:** The device can be treated with commonly

marketed cold disinfectant, used according to their specific instructions.

**Drying:** After having washed or sanitized all components, shake them vigorously, and finally place them on a paper towel, or alternatively dry with a jet of hot air (for example using a hairdryer).

**Assemble the device as shown on the relevant diagram (Picture C), after having lubricated with baby moisturizing oil the seal of the plunger (4).**

### INSTRUCTIONS FOR USE

Personal use is recommended, as well as frequent replacement of the nasal **tip (1)** to avoid any risk of infection.

**a**-Hold the **Aspirator** as if it were a syringe.

**b**-Place the **Nasal tip (1)** of the **Aspirator** inside the nostril of the child (Picture A).

**c**-Press with the thumb on the bottom of the **Plunger (4)** (Picture A); the air is expelled through the slots at the base of the device **Body (2)**, thus avoiding the creation of a pressure towards the nasal orifice of the child. This phase can also be performed before placing the **Aspirator** at the nostril of the child.

**d**-Gradually release the **Plunger (4)** (Picture B), thus creating a vacuum that gently sucks the mucus from the nose of the child.

**e**-Repeat for the other nostril from point **b**.

### ENVIRONMENTAL CONDITIONS

#### Operating conditions:

Ambient temperature Between + 10 °C and + 40 °C

Relative air humidity Between 10% and 95%

Atmospheric pressure Between 69KPa and 106 KPa

#### Storage and transport conditions:

Ambient temperature Between -5 °C and + 70 °C

Relative air humidity Between 10% and 95%

Atmospheric pressure Between 69KPa and 106 KPa

### SYMBOLS

	Production date		Manufacturer
	Medical device		Warning
	Unique Device Identifier	#	Model number
	Lot code		Temperature limits
	Atmospheric pressure limits		Humidity limits
	Read the instructions for use		
	Medical CE marking ref. regulation 2017/745 EU and subsequent updates		

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