# Rhino Clear®

by **FLAEM®**

SINCE 1966  
MADE IN ITALY

**PHTHALATES  
& BPA FREE**

**Ergo  
Plus**

MANUALE ISTRUZIONI D'USO DELLA DOCCIA NASALE  
NASAL SHOWER USER INSTRUCTION MANUAL  
MODE D'EMPLOI DE LA DOUCHE NASALE  
BEDIENUNGSANLEITUNG NASENSPÜLUNG  
MANUAL DE INSTRUCCIONES DE USO DEL NEBULIZADOR NASAL  
MANUAL DE INSTRUÇÕES DE USO DA DUCHA NASAL  
РУКОВОДСТВО ПОЛЬЗОВАТЕЛЯ НАЗАЛЬНОГО ДУША  
INSTRUKCJA OBSŁUGI IRYGATORA DO NOSA  
BRUGSANVISNING TIL NÆSEDOUCHE  
NENÄSUIHKUN KÄYTTÖOPAS  
BRUKSINSTRUKSJONER FOR NESEDUSJ  
NEUSDUCHE GEBRUIKERSHANDLEIDING

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## NASAL WASH

### “The pleasure of breathing well”

Thank you for your purchase. Our goal is our customers' complete satisfaction, offering them cutting edge products to prevent and treat respiratory disorders.

Visit the website

[www.flaem.it](http://www.flaem.it). **Read these instructions carefully in order to correctly use the NASAL WASH SYSTEM. In addition, we advise you to keep this manual with care should you need it for further consultation.**

**RHINO CLEAR® Ergo Plus is a medical device for home use.**

#### What is it?

**RHINO CLEAR® Ergo Plus** is a NASAL WASH SYSTEM specifically designed to irrigate and treat adult and child nasal cavities, both with saline solution and with medicated solutions for aerosol use, depending on the medical prescription.

#### How does it work?

**RHINO CLEAR® Ergo Plus** connected to a FLAEM compressor nebulises solutions to irrigate the nasal cavities; nebulisation is activated by a button on the handle. On the body of the NASAL WASH SYSTEM there are two adjacent chambers: one for the solution to nebulise and the other for the recovery solution. Nasal adapters are inserted on the upper part of the NASAL WASH SYSTEM. They are available in 3 different sizes depending on the size of the user's nostrils.

#### When do I use it?

**RHINO CLEAR® Ergo Plus can be used for:**

1. **Nasal cavity physiological irrigation** (saline solution or thermal water). It is useful to help with nasal hygiene in children. In adults, it favours physiological irrigation of the nose and can be used before aerosol treatments in the respiratory tract in order to help distribute the medication.
2. **Therapeutic washing:** medicinal administration through **RHINO CLEAR® Ergo Plus** must take place on recommendations from a doctor or pharmacist, who defines the type of medication, the doses and the duration of the treatment.
3. **Help children breathe well:** in fact, up to 1 year of age, children breathe exclusively through their nose. If their nose is clogged, they have trouble breathing, wake up during the night, have trouble eating and cry often. Considering that up to 3 years of age children are not able to blow their nose by themselves, the mucous containing bacteria, viruses and dust stagnates in the nasal cavities. There are two most recommended ways to flush out the nose well: nasal washing and mucous removal; for washing we recommend **RHINO CLEAR® Ergo Plus** and for the removal of mucous the nasal aspirator for children of the FLAEM Baby Line.

## DEVICE COMPOSITION:

---

- 1 - Nasal adapter "A", recommended for children up to 3 years  
Nasal adapter "B", recommended for children from 3 to 12 years  
Nasal adapter "C", recommended for anyone over the age of 12
- 2 - Separator
- 3 - Atomiser
- 4 - Wash system body with ergonomic handle
  - 4.1) 10 ml nebulisation chamber
  - 4.2) Collection chamber
- 5 - Delivery button
- 6 - Nasal wash system - TUBING CONNECTION FITTING
- 7 - Tubing to connect to compressor (not included in the package)

## IMPORTANT NOTICES

---

- This is a medical device (Compliant with dir. 93/42/CEE) and must be used with medications prescribed or recommended by your doctor. Before using the accessories, consult the device operating instructions manual. It is important for the patient to read and understand the information for use. Contact your retailer or service centre if you have any questions.
- Children and people who are not self-sufficient must use the device under the close supervision of an adult who has read this manual.
- Some parts of the device are small enough to be swallowed by children; therefore, keep the accessory out of the reach of children.
- Interactions: the materials used in contact with medication have been tested with a vast range of medications. However, in view of the variety and continuous evolution of pharmaceuticals, interactions cannot be ruled out. We recommend using the medication as soon as possible once it has been opened and preventing prolonged exposure in the nasal wash system.

## INSTRUCTIONS FOR USE

---

**Before each utilisation, wash your hands accurately and clean the NASAL WASH SYSTEM and the accessories as described in the paragraph "CLEANING, SANITISATION, DISINFECTION AND STERILISATION".**

**The NASAL WASH SYSTEM and accessories are for personal use only to avoid potential risks of transmitting infection.**

During application, it is advisable to suitably protect yourself from any dripping.

1. Insert the atomiser (3) into the nebulisation chamber (4.1).
2. Put the solution (max 10 ml) into the nebulisation chamber (4.1)(fig. A).
3. Attach the separator (2). Insert the correct nasal adapter for your age or the one most suitable for the size of your nostrils (1A/1B/1C).
4. Connect the nasal douche system to the device using the tubing (7) and the fitting (6).
5. Switch on the device (see the user manual).

- Bring it to your nose, placing the hole of the adapter against one nostril.
- During treatment, it is better to close with a finger the nostril where the NASAL WASH SYSTEM is not applied.
- Activate the NASAL WASH by holding down the delivery button (5) while inhaling and release the button before exhaling; to help expel the mucous, blow gently with the nostril being treated directly into the NASAL WASH SYSTEM.
- Repeat steps 6. through 8. for the other nostril, trying to distribute the solution evenly between the two nostrils.
- For children or anyone unable to correctly coordinate the various parts of step 8., we recommend alternating the application between the nostrils every 20 seconds, leaving suitable pauses to allow the mucous to be expelled.
- The steps described above are useful for nasal irrigation using saline solutions. It is advisable to carry out medicated treatments after an initial irrigation with saline solution for a more efficient distribution of the medication on the nasal mucous membranes.
- Once the application is complete, switch off the device and clean it, following the instructions provided in the "CLEANING, SANITISATION, DISINFECTION STERILISATION" paragraph.

## **CLEANING, SANITISATION, DISINFECTION, STERILISATION**

*Switch off the device before any cleaning procedure and unplug the power cable from the socket.*

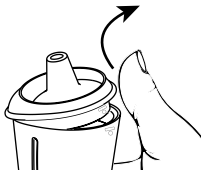
### **DEVICE AND TUBING EXTERIOR**

Use only a damp cloth with antibacterial soap (non-abrasive and with no solvents of any sort).

### **ACCESSORIES**

Disassemble the NASAL WASH SYSTEM as illustrated in the connection diagram.

As the separator (2) is fitted in the body (4), it is necessary to exert some force to separate them.

Then proceed according to the instructions below.

## **CLEANING AT HOME - SANITISATION AND DISINFECTION**

### **SANITISATION**

Before and after each use, sanitise the nebuliser cup and the accessories, choosing one of the methods described below.

**method A:** Sanitise accessories 1A-1B-1C-2-3-4-5-6 under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

**method B:** Sanitise accessories 1A-1B-1C-2-3-4-5-6 in the dishwasher with a hot cycle.

**method C:** Sanitise accessories 1A-1B-1C-2-3-4-5-6 by immersing them in a solution with 50% water and 50% white vinegar, then rinse thoroughly under potable hot water (approximately 40°C).

If you also wish to perform cleaning by DISINFECTION, jump to the DISINFECTION paragraph.

After having sanitised the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).

#### DISINFECTION

After sanitising the nebuliser cup and the accessories, disinfect them choosing one of the methods described below.

**method A:** The disinfectable accessories are 1A-1B-1C-2-3-4-5-6.

The disinfectant must be an electrolytic chlorine oxidiser (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

#### **Directions:**

- Fill a container of suitable size to hold all of the parts to be disinfected with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse with plenty of warm potable water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

**method B:** Sanitise the accessories 1A-1B-1C-2-3-4-5-6 by boiling them in water for 10 minutes; use demineralised or distilled water to prevent calcium deposits.

**method C:** Sanitise the accessories 1A-1B-1C-2-3-4-5-6 with a hot steam steriliser for baby-bottle (not the microwave type). Perform the process faithfully following the instructions of the steriliser. To ensure that the disinfection is effective, choose a steriliser with an operating cycle of at least 6 minutes.

After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

### **CLEANING IN A CLINICAL OR HOSPITAL SETTING - DISINFECTION AND STERILISATION**

Before disinfection or sterilisation, sanitise the nebuliser cup and the accessories, choosing one of the methods described below.

**method A:** Sanitise accessories 1A-1B-1C-2-3-4-5-6 under potable hot water (approximately 40°C) with a gentle, non abrasive dish detergent.

**method B:** Sanitise accessories 1A-1B-1C-2-3-4-5-6 in the dishwasher with a hot cycle.

#### DISINFECTION

The disinfectable accessories are 1A-1B-1C-2-3-4-5-6.

The disinfectant must be an electrolytic chlorine oxidiser (active principle: sodium hypochlorite) specific for disinfecting, which is available in any pharmacy.

#### **Directions:**

- Fill a container of suitable size to hold all of the parts to be disinfected with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse with plenty of warm potable water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

If you want to also perform the STERILISATION, jump to the STERILISATION paragraph

After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, from a hair dryer).

### STERILISATION

The accessories that can be sterilised are 1A-1B-1C-2-3-4-5.

Equipment: Fractionated vacuum overpressure steam steriliser in accordance with EN 13060.

**Directions:** Wrap every single part to be treated with a sterile barrier system or packaging in accordance with Norm EN 11607. Place the packed components in the steam steriliser. Run the sterilisation cycle according to the operating instructions of the device by selecting a temperature of 121°C and a time of 10 minutes first.

**Storage:** Store the sterilised parts as per the instructions for use of either the sterile barrier system or packaging.

The sterilisation procedure is validated in its conformity to ISO 17665-1.

## NASAL WASH SYSTEM TECHNICAL FEATURES

### Mod.: Rhino Clear® Ergo Plus

Operating Conditions:

Temperature: min 10°C; max 40°C

Relative humidity: min 10%; max 95%

Atmospheric pressure: min 69KPa; max 106KPa

Storage conditions:

Temperature: min -25°C; max 70°C

Relative humidity: min 10%; max 95%

Atmospheric pressure: min 69KPa; max 106 KPa

Capacity: medication maximum 10 ml

Type BF applied parts are: patient accessories (1A-1B-1C)

Compliant with Directive 93/42 EEC-

Family of FLAEM compressors	10 ml Delivery Time <sup>(1)</sup>
F400	2 minutes (approx)
F700/F1000	2 minutes (approx)
F2000	2 minutes (approx)

<sup>(1)</sup> Tests performed with 0.9% NaCl saline solution, according to the Flaem internal procedure

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© Copyright by FLAEM NUOVA S.p.A.,
FLAEM NUOVA S.p.A. - Via Colli Storici, 221