# FLAEM®

Medical Devices

## Aspira Go

IT > MANUALE ISTRUZIONI  
EN > USER MANUAL  
DE > BETRIEBSANLEITUNG  
FR > MODE D'EMPLOI  
NL > GEBRUIKSAANWIJZING  
ES > INSTRUCCIONES DE USO  
ΕΛ > ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ  
PL > INSTRUKCJA OBSŁUGI  
CZ > NÁVOD K POUŽITÍ  
RU > ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ  
DA > BRUGSANVISNING  
إرشادات الاستخدام < AR

Aspiratore chirurgico portatile a batterie / Portable battery-operated surgical aspirator / Tragbares batteriebetriebenes chirurgisches Absauggerät / Aspirateur chirurgical portable fonctionnant sur batterie / Draagbaar chirurgisch afzuigapparaat op batterijen / Aspirador quirúrgico portátil a pilas / Φορητός χειρουργικός αναρροφητήρας με μπαταρία / Przenośny ssak chirurgiczny na baterie / Přenosná chirurgická odsávačka na baterie / Værbar batteridrevet kirurgisk sugekop til kirurgisk brug / Портативный хирургический аспиратор с питанием от аккумулятора / Værbar batteridrevet kirurgisk sugekop til kirurgisk brug / شفاط جراحي محمول يعمل بالبطارية

ITALIANO

ENGLISH

DEUTSCH

FRANÇAIS

NEDERLANDS

ESPAÑOL

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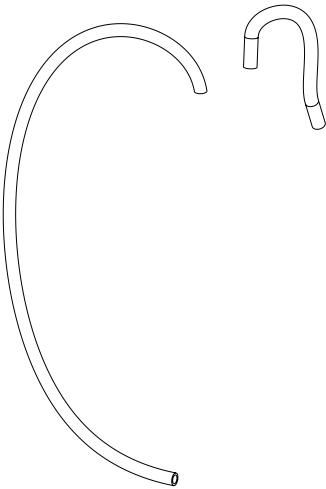
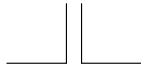
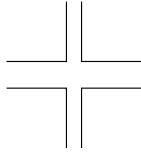
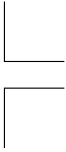
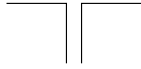
POLSKI

ČESTINA

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
DANSK

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
# A005

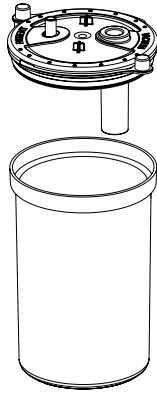
# ASP-2









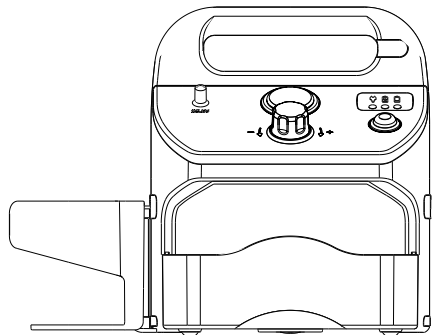
# P161EM-20
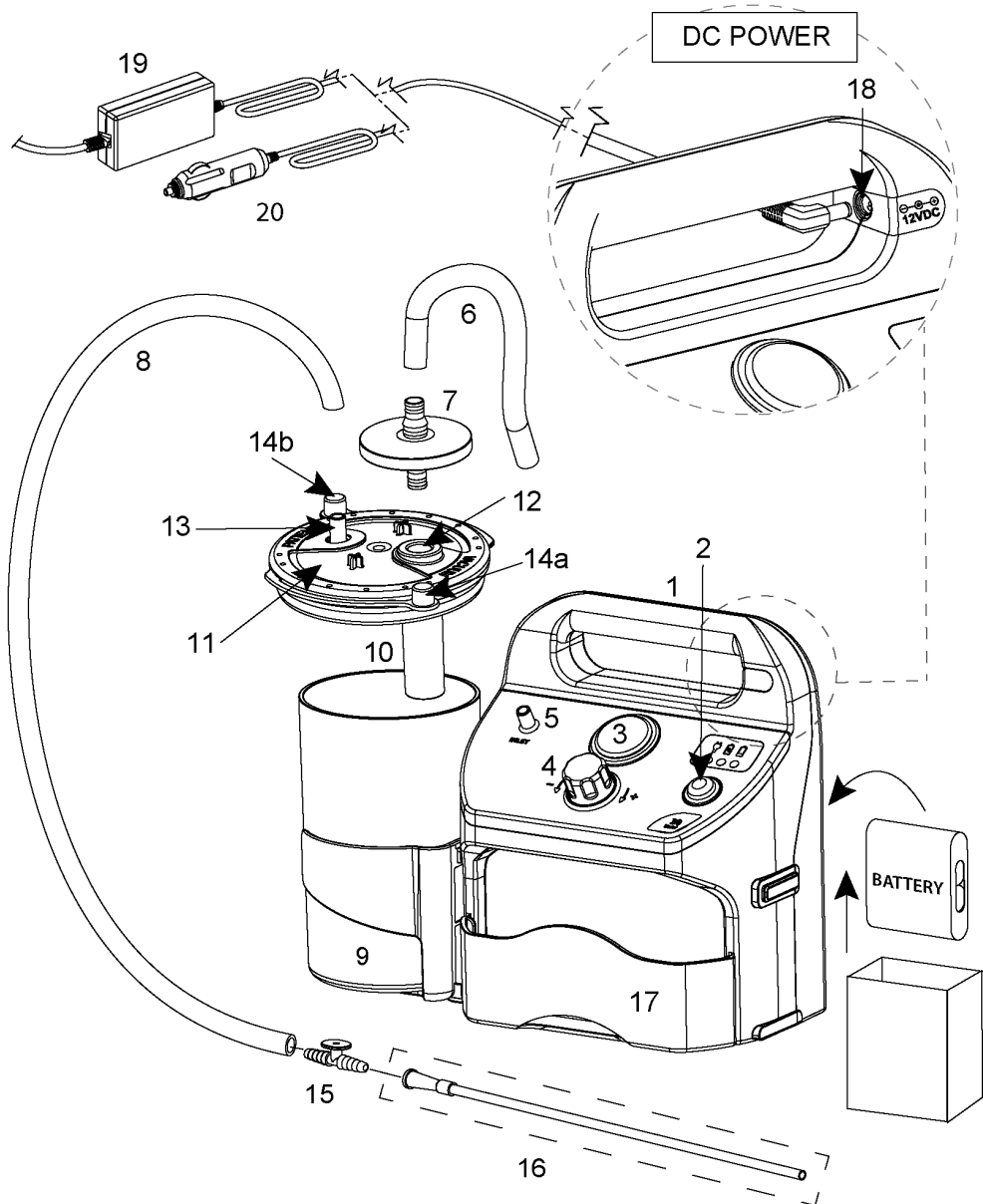
# P161EM-30


Schema di collegamento per nomenclatura - Connection diagram for nomenclature - Anschlussplan für die Nomenklatur - Schéma de raccordement pour nomenclature - Aansluitschema voor nomenclatuur - Esquema de conexión para nomenclatura - Διάγραμμα σύνδεσης για την ονοματολογία - Schemat połączeń dla nomenklatury - Schéma zapojení pro nomenklaturu - Схема подключения по перечню - Tilslutningsdiagram til nomenklatur - مخطط توصيل لمجموعة المصطلحات

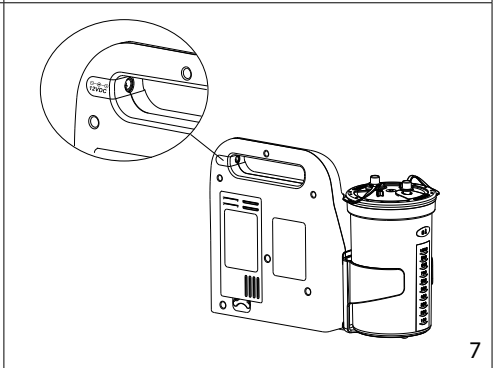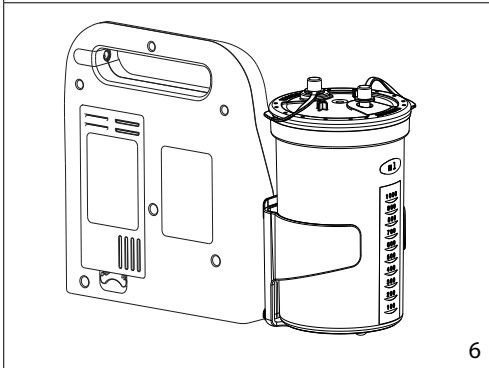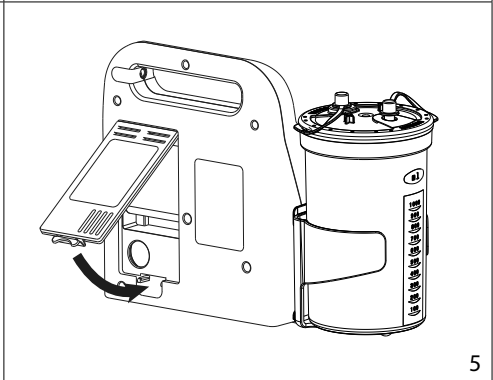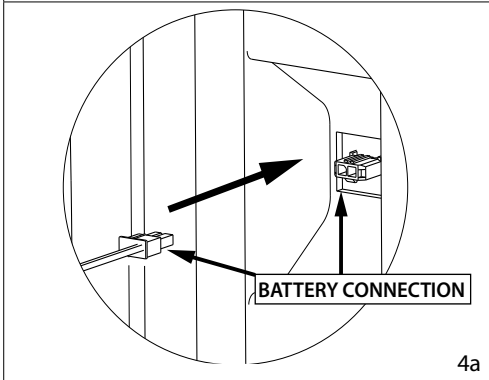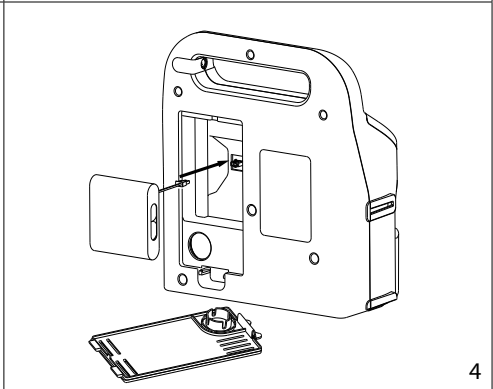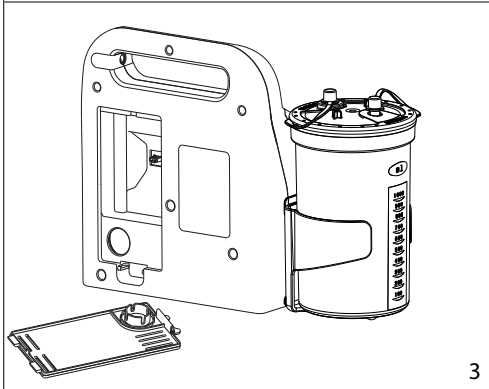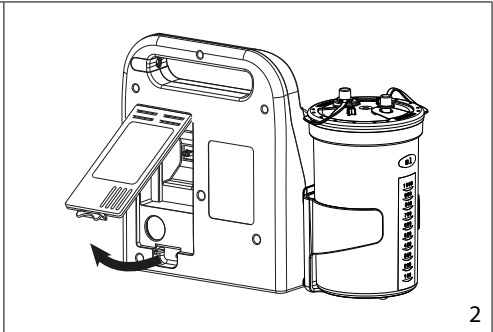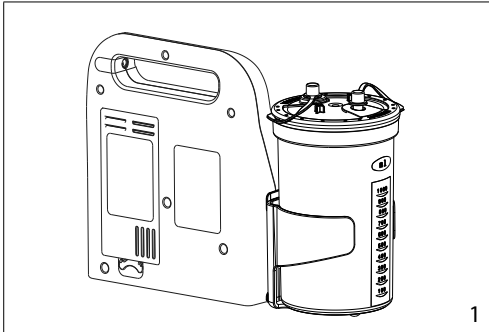
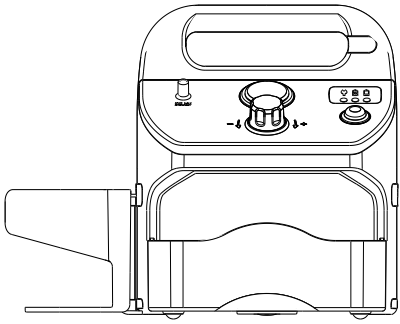
8

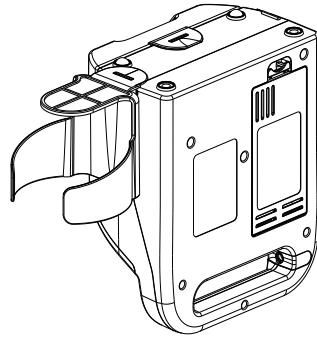
9

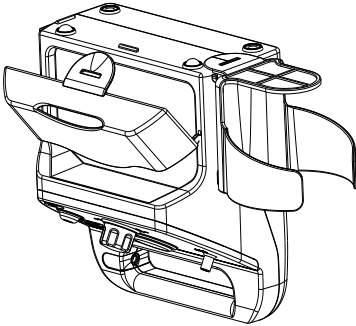
10

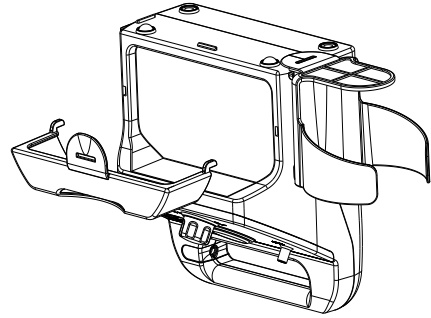
11

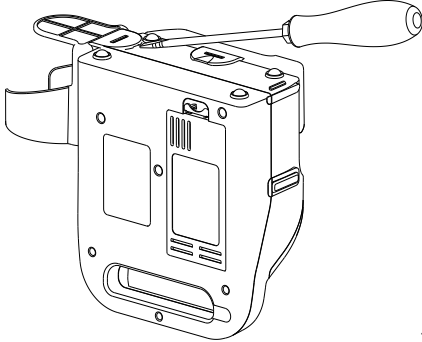
12

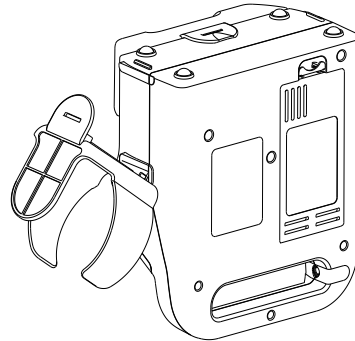13

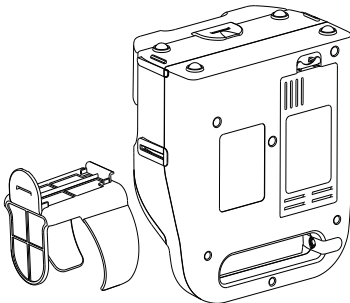14

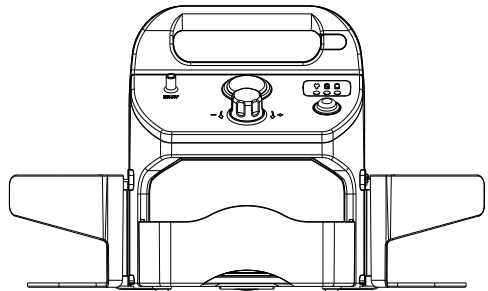15

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**These operating instructions are provided for the FLAEM aspirator consisting of a depressor unit model P1611EM-20 or P1611EM-30 and the secretion collection vessel model ASP-2 and piping model A005 accessories.**

**INTENDED USE.** Medical Device, aspirator for medical and surgical use, for the aspiration of bodily fluids, pharyngeal aspiration and permanent tracheotomy. Its use must be prescribed by a doctor.

#### **INDICATIONS FOR USE**

Treatment of pathologies requiring the aspiration of body fluids (saliva, secretions, post tracheotomy fluids, etc.).

**⚠️ CONTRAINDICATIONS.** The device must NOT be used in operating theatres, for chest drainage or on transport vehicles such as ambulances or emergency vehicles.

**INTENDED USERS.** The devices are intended for use by legally authorised medical personnel/health workers (doctors, nurses, therapists, etc.). The device can be used directly by the patient for home use, but under the guidance of medical personnel.

**⚠️ TARGET GROUP OF PATIENTS.** Adults, children of all ages. Before using the device, the user manual must be read carefully and an adult responsible for safety must be present if the device is used by children of any age or persons with limited abilities (e.g. physical, mental or sensory). The evaluation of the use of the device on particular types of patients such as pregnant women, breastfeeding women, newborns, incapacitated subjects or those with limited physical capabilities is referred to medical personnel. All this must take place under the guidance of medical personnel.

It is up to the medical personnel to assess the patient's condition and capabilities in order to determine, when prescribing the device, whether the patient is able to operate the aspirator safely on their own or whether the therapy should be administered by a responsible person.

**OPERATING ENVIRONMENT.** All devices can be used in healthcare facilities, such as hospitals, outpatient clinics, etc., or even at home.

**⚠️ WARNINGS CONCERNING POSSIBLE MALFUNCTIONS.** • Should your appliance fail to perform, please contact the authorised service centre for clarification. • The manufacturer should be contacted to report problems and/or unexpected events relating to operation and if necessary for clarification of use and/or hygienic preparation. • Please also refer to the case history of faults and their resolution.

**⚠️ WARNINGS.** • For correct operation and to prolong the life of the device, the operating instructions must be strictly followed.

- If the packaging is damaged or opened, contact the distributor or service centre. • The manufacturer, the seller and the importer are only responsible for safety, reliability and performance if: a) the appliance is used in accordance with the operating instructions, b) the electrical installation in the room where the appliance is used is in accordance with the regulations and complies with the laws in force.
- The 12V DC power cable for cigarette lighter socket is equipped with a safety fuse that can be inspected in the event of a fault.
- If the aspirator is not used for a long period (6 months), it is advisable to remove the battery before storing it.
- It is forbidden to access the device opening in any way. Repairs may only be carried out by authorised FLAEM personnel following the information provided by the manufacturer. Unauthorised repairs invalidate the warranty and may pose a danger to the user.
- The operations of preparation of the collection vessel must be carried out on a stable support surface.

**Suffocation risk:** Some components of the appliance are small enough to be swallowed by children, so keep the appliance out of the reach of children.

**Strangulation risk:** Do not use the supplied pipes and cables outside of their intended use, they could cause a strangulation hazard, take special care for children and people with special difficulties, often these people are not able to correctly assess the dangers.

**Fire risk:** - The use of batteries other than those supplied with the device is prohibited.

- Do not expose the appliance to temperatures other than those indicated in the chapter "technical specifications"

- The aspirator is only intended for the collection of NON-flammable fluids, not suitable for use in the presence of flammable anaesthetic mixture with air, or with oxygen or nitrous oxide.

**Risk of electrocution:** - Before first use, and periodically during the life of the product, check the integrity of the structure of the appliance, the supplied power supply unit, the power cord to ensure that there is no damage; if it is damaged, do not plug it in and immediately take the product to an authorised service centre or your dealer.

- Do not carry out any hygienic preparation when the device is in use on a patient.

- In the presence of children and care-dependent persons, the appliance must be used under the close supervision of an adult who has read these operating instructions.

- Keep the power cable of the power supply unit away from animals (e.g. rodents), otherwise such animals could damage the insulation of the power cable.

- Always keep the power supply unit and the power cable away from hot surfaces.

- Do not immerse the appliance in water; if this happens, pull the plug immediately. Do not pull out or touch the appliance while immersed in water, unplug it first. Take it immediately to an authorised FLAEM service centre or your dealer.

- Do not wash the appliance under running water or by immersion and keep it away from splashes of water or other liquids.

- The provided power supply unit is specifically designed for use with this appliance. Never use the power supply unit with other appliances or for purposes other than those stipulated in these operating instructions.

**• Risk of ineffectiveness of therapy:**

- Only use original Flaem accessories or spare parts, no liability is accepted if non-original parts or accessories are used.

- Ensure that the connections and closing of the collection vessel are carried out carefully to avoid aspiration leaks.

- With the intervention of the protection device (10) aspiration ceases, switch off the appliance, empty the vessel (9) and carry out hygienic preparation.

**• Risk of infection:**

- Personal use of the accessories, collection vessels and connecting tubes is recommended to avoid risks of infection.

- The manual aspirated flow control (15) is a sterile disposable product and must be replaced each time it is used.

- Check the expiry date on the original packaging of the manual aspirated flow control (15) or on the label on the packaging of the device and check the integrity of the sterile packaging, if the packaging is not intact please contact your Flaem authorised service centre

or your dealer. FLAEM NUOVA will not be held liable for damage to the patient related to the deterioration of the above-mentioned sterile packaging due to manipulations carried out by third parties on the original packaging of the entire device.

- The hydrophobic/antiviral and antibacterial filter supplied with Flaem equipment is a disposable device and must be replaced after each use or if the filter becomes saturated.

Check the expiry date on the original filter packaging or on the label on the aspirator packaging.

- Follow hygienic preparation before each use.

- Do not tip the vessel while it is connected to the aspirator in operation, as liquid can be sucked into the aspirator and thus damage the pump. If this happens, switch the aspirator off immediately and empty and clean the vessel (send it to an authorised FLAEM service centre).

• **Risk of injury:**

- Do not place the medical device on a soft supporting surface such as a sofa, a bed or a tablecloth.

- Always operate it on a hard surface clear of any objects.

#### WARNINGS ON INTERFERENCE RISKS DURING USE IN DIAGNOSTIC INVESTIGATIONS

• This device is designed to meet the current requirements for electromagnetic compatibility. As far as EMC requirements are concerned, electromedical devices require special care when being installed and used, and are therefore required to be installed and/or used in accordance with the manufacturer's specifications. Risk of potential electromagnetic interference with other devices. Mobile or portable RF radio and telecommunication devices (mobile phones or wireless connections) may interfere with the operation of electromedical devices. The device may be susceptible to electromagnetic interference in the presence of other devices used for specific diagnosis or treatment. For more information visit [www.flaemnuova.it](http://www.flaemnuova.it).

• The Medical device is not suitable for use in MRI (Magnetic Resonance Imaging) environments.

#### CASE HISTORY OF FAULTS AND THEIR RESOLUTION

Before performing any operation, switch off the appliance and unplug the mains cable from the socket.

PROBLEM	CAUSE	REMEDY
The device is not working	Low battery	Recharge the battery
	Battery not inserted correctly	Check the correct insertion of the battery connector
	Power cable not correctly plugged into the appliance socket or power socket - Faulty power supply unit	Replace the power supply unit
No aspiration	Collection vessel lid not properly inserted in the vessel	Insert the lid correctly into the collection vessel
Aspiration failure caused by fluid spillage	Clogged filter	Replace the filter
Blocked float	Fouling on the float	Remove the lid from the vessel, dismantle the protective device and remove the float. Then carry out cleaning operations as per the paragraph on hygienic preparation
Low and/or no vacuum power	Vacuum regulator all open	Close the regulator completely and check vacuum power
	Protection filter blocked	Replace the filter
	Obstructed, kinked or disconnected filter and device connection pipes	Check the condition of the pipes, replace them if they are obstructed and connect them correctly as per the 'CONNECTION DIAGRAM' in these operating instructions
	Vessel lid overflow valve closed or blocked	Unlock the overflow valve, hold the device upright
	Dirty or obstructed or damaged pump	Take the device to your dealer or an authorised FLAEM service centre

If, after checking the conditions described above, the device still is not working properly, we recommend that you contact your trusted dealer or an authorised FLAEM service centre nearest to you. You can find a list of all Service Centres at <http://www.flaemnuova.it/it/info/assistenza>

## DISPOSAL

### Aspiration unit

 In accordance with Directive 2012/19/EC, the symbol on the equipment indicates that the equipment to be disposed of (excluding accessories) is considered as waste and must therefore be subject to 'separate collection'. Therefore, the user must deliver this waste (or have it delivered) to the separate collection centres set up by the local authorities, or hand it over to the retailer when purchasing a new appliance of an equivalent type. Separate waste collection and subsequent treatment, recovery and disposal operations promote the production of equipment from recycled materials and limit the negative environmental and health effects caused by improper waste management. Unauthorised disposal of the product by the user entails the application of the administrative sanctions provided for in the laws transposing Directive 2012/19/EC of the member state or country where the product is disposed of.

### Accessories (Secretion collection vessel and piping)

Accessories to be disposed of as general waste after a sanitisation cycle.

### Packaging

Product box

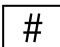
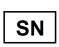
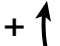
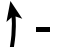
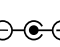
Product packaging bag

**REPORTING OF SERIOUS INCIDENTS.** Please report any serious incidents that occur in relation to this device to the manufacturer and the competent authority of the Member State in which you reside. An event is considered serious if it causes or may cause, directly or indirectly, death or an unexpected and serious deterioration in the state of health of a person.



### INFORMATION ON RESTRICTIONS OR INCOMPATIBILITIES WITH CERTAIN SUBSTANCES.

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Interactions: The materials used for contact with secretions are thermoplastic polymers with high stability and chemical resistance (PP, PC, SI). However, interactions cannot be excluded. Therefore, it is recommended to: a) Always avoid prolonged contact of the liquid with vessels or tubes, sanitise immediately after use. b) If abnormal situations occur, e.g. softening or cracking of accessories, quickly terminate operation and replace the materials used. Contact the authorised service centre specifying the method of use.

### SYMBOLS ON DEVICE OR PACKAGING

 Medical CE marking ref. regulation 2017/745 EU and subsequent updates	 Model number	 Serial number of the device	 Moisture limits
 Class II device	 Temperature limits	 Manufacturer	 Unique device identifier
 Before use: Caution check instructions for use	 Atmospheric pressure limits	 Type BF applied part	 Greater vacuum
 Switched 'ON' When switching off the unit, the switch stops compressor operation on only one of the two power phases.	 Batch Code	 Alternating current	 Less vacuum
 Switched 'OFF'	 Direct Current	 Production date	 Medical device
 Positive polarity	 Socket for safety extra-low voltage 12VDC	 Attention	
 Negative polarity	 Quality mark	 See instructions for use	
<b>IP22</b> Degree of protection of the casing: IP22. (Protected against the entry of solid objects larger than 12mm in diameter. Protected against dripping water at an angle within $\pm 15^\circ$ ).			

## ASPIRATOR TECHNICAL SPECIFICATIONS

Model	P1611EM-20	P1611EM-30
<b>Voltage</b>	12V 	12V 
<b>Internal battery included:</b>	14.4V; 2200/2500/2600/2750/ 2800/3000/3120 mAh	14.4V; 2200/2500/2600/2750/ 2800/3000/3120 mAh
<b>Battery autonomy:</b>	Approximately 50 minutes	Approximately 40 minutes
<b>Appliance dimensions:</b>	36(L) x 14(D) x 27(H) cm	36(L) x 14(D) x 27(H) cm
<b>Weight:</b>	2.5 kg	2.5 kg
<b>Usage:</b>	30 minutes ON/ 30 minutes OFF	30 minutes ON/ 30 minutes OFF
<b>APPLIED PARTS</b> Type BF applied parts are:	If necessary, a sterile cannula can be attached to the connection of the MANUAL ASPIRATED FLOW CONTROL (15).	
<b>Aspiration:</b>	high vacuum/high flow	high vacuum/high flow
<b>Adjustable vacuum level:</b>	-0.10 to -0.80 bar (approx.) (accuracy class 2.5)	-0.10 to -0.85 bar (approx.) (accuracy class 2.5)
<b>Max. air flow:</b>	20 l/min (approx)	30 l/min (approx)
<b>Noise level (at 1 m)</b>	63 dB (A) (approx)	62 dB (A) (approx)
<b>Battery charging time</b>	7 hours (minimum)	4 hours (minimum)
<b>Ambient temperature for battery charging</b>	10° C to 35° C	10° C to 35° C

## ENVIRONMENTAL CONDITIONS

### Operating conditions:

Ambient temperature	Between 0°C and +35°C
Relative air humidity	Between 10% and 95%
Atmospheric pressure	Between 69 KPa and 106 KPa

### Storage and transport conditions:

Ambient temperature	Between -5°C and +45°C
Relative air humidity	Between 10% and 95%
Atmospheric pressure	Between 69 KPa and 106 KPa

## DURATION

### Model: 1611EM-20 and 1611EM-30

(Vacuum unit)

Service life 550 hours

### Model: ASP-2 and A005 (Accessories)

Average life expectancy is 1 year

APPLIANCE AND MATERIAL INFORMATION		
The equipment includes:	Model	Information on materials
1) ASPIRATOR	P1611EM-20	
2) SWITCH		
3) VACUUM GAUGE		
4) VACUUM ADJUSTMENT KNOB		
5) AIR INLET	P1611EM-30	
6) CONNECTION PIPE - Ø 13 x 7.5 mm L 1300 mm		
8) CONNECTION PIPE - Ø 13 x 7.5 mm L 280 mm	A005	Silicone
7) HYDROPHOBIAL ANTIVIRAL/ ANTIBACTERIAL FILTER (SINGLE USE) (device not manufactured by FLAEM)		
9) COLLECTION VESSEL -with complete cap	ASP-2	Polycarbonate
10) PROTECTIVE DEVICE		Polypropylene+ Silicone
11) VESSEL CLOSURE CAP		Polypropylene + thermoplastic elastomers
12) VESSEL 'VACUUM' SOCKET		
13) VESSEL 'PATIENT' SOCKET		
14) LID PROTECTION CAPS		
15) MANUAL ASPIRATED FLOW CONTROL - (disposable) (device not manufactured by FLAEM)		
16) ASPIRATOR CANNULA. (NOT SUPPLIED WITH THE DEVICE). The cannula is not supplied and is not available as a spare part for the medical device. Any cannulae to be used with our aspirators must be of a shape adaptable to the 'MANUAL ASPIRATED FLOW CONTROL' connection supplied with the aspirator.		
17) ACCESSORY COMPARTMENT		
18) EXTERNAL POWER SOCKET		
19) MULTI-VOLTAGE POWER SUPPLY		
20) CIGARETTE LIGHTER CABLE		
IMPORTANT NOTE: There is an identification label on the packaging, remove it and affix it in the spaces provided on page 2. When replacing the Nebuliser and accessories, perform the same procedure.		

## INSTRUCTIONS FOR USE

**Before each use, the accessories must be carefully inspected, making sure that there is no dust, encrustation, lumps or liquid inside the connection pipe, the vessel and its cap. They must also be cleaned strictly in accordance with the instructions given in the section 'CLEANING, SANITISATION, DISINFECTION AND STERILISATION'. Personal use of the accessories, collection vessels and connecting tubes is recommended to avoid the risk of infection.**

### 1. Internal battery operation:

**1.1.** The device is supplied with a partially charged battery and it is advisable to recharge it before use. Insert the battery as shown on page 1. Unhook and lift up and remove the battery compartment cover at the rear of the unit (draw.1,2,3); insert the battery by inserting the connector correctly into place (draw.4,4a,); replace and close the battery compartment cover (draw.5,6). Proceed with recharging following the instructions in section 2.3.1.

**1.2.** Start up the appliance by actuating the switch (2).

**1.3.** At the end of each suction treatment, always set the switch (2) to position 0 (even when the battery is empty) to save battery power. If, during use, the red LED(1c) flashes and the buzzer sounds, this means that the appliance's autonomy is coming to an end; in this condition, charge the appliance as soon as possible.

**⚠ WARNING:** In order to avoid having to use the device in an emergency and not having the support of the internal power source, it is recommended to charge the battery at least once every 3 months.

### 2. Operation and charging via vehicle cigarette lighter cable (12V DC), or with multi-voltage power supply:

#### 2.1. Cigarette lighter cable (12V DC) (20):

**2.1.1.** Use the cigarette lighter cable (20) to connect the external socket (18) of the appliance with the cigarette lighter socket. Check the charge status of the vehicle or boat battery before use with the cigarette lighter cable.

**2.1.2.** Start up the device by actuating the switch (2); operation is indicated by the green LED (1a) lighting up. The device is intended for intermittent use 30 min ON/30 min OFF.

#### 2.2. Multi-voltage (switching) power supply (19).

**2.2.1.** Connect the connector to the external socket (18) of the appliance. Plug the power cord into the mains socket corresponding to the voltage of the appliance, at the end of the use of the appliance pull the plug out of the socket, it must be positioned so that it is not difficult to disconnect from the mains. If the plug of the power cable is different from the mains socket, please contact your dealer or authorised service centre.

**2.2.2.** Repeat the instructions from point 2.1.2

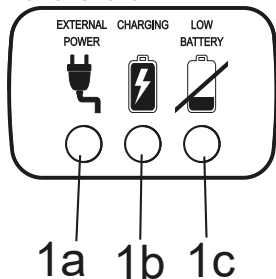
#### 2.3. Recharging the internal battery by the vehicle cigarette lighter cable, or with multi-voltage power supply:

Check the charge status of the vehicle or boat battery before use with the cigarette lighter cable.

**2.3.1.** The battery is recharged by leaving the cigarette lighter cable or the multi-voltage power supply unit (19) connected to the external socket (18) of the appliance. The lighting up of the green LED (1a) and the yellow LED (1b) indicates that the internal battery is being recharged.

Charge the appliance for 24 hours the first time; thereafter, 7 hours (minimum) for model P1611EM-20 and 4 hours (minimum) for model P1611EM-30 are sufficient.

## LED FUNCTIONS

**GREEN LED (1a):** when lit it means there is an external power supply (12VDC), i.e., the switching power supply was plugged into the socket (18) of the appliance.

**YELLOW LED (1b):** when lit it means that the battery is charging, when off it means that the battery is charged and ready for use\*.

**Only for model P1611EM-30:** if the YELLOW LED (1b) flashes, it indicates that the battery is not connected and/or is not properly connected to the circuit (see draw. 4, 4a, 4b).

**RED LED (1c):** flashing and acoustic alarm active, both indicating that the battery is running low and must be recharged as soon as possible.

**\*IMPORTANT:** To recharge the battery insert the plug of the power supply unit (12VDC) into the socket (18) of the appliance.

## ⚠ BATTERY REPLACEMENT

To replace the battery, follow the instructions on page 1. Unhook, lift and remove the battery compartment cover at the rear of the appliance (draw.1,2,3); remove the battery to be replaced and place the new one, inserting the connector correctly into place (draw.4,4a); fit and close the battery compartment cover (draw.5,6). Then proceed with recharging following the instructions in section 2.3.1.

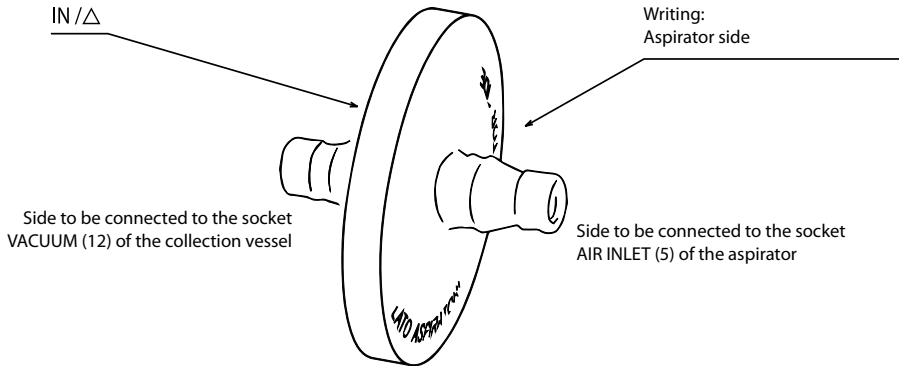
The flat battery must be disposed of through the appropriate collection containers or by contacting a suitable waste disposal centre. The use of batteries other than those supplied with the medical device is prohibited. If necessary, please contact an authorised FLAEM service centre.

### 3. Instructions for aspiration operation on the patient:

**3.1.** Connect the accessories, referring to the 'CONNECTION DIAGRAM' on the cover.

**⚠ ATTENTION:** the filter is hydrophobic and in addition to being antibacterial/antiviral, it also acts as a secondary protection and stops fluids that the primary protection device in the lid of the collection vessel accidentally fails to stop.

**Follow the instructions below for the correct assembly of the filter:**

**3.2.** Operate the appliance according to the indications of the selected operation. Always use the appliance on a horizontal plane; this will allow the liquid ingress protection device in the aspiration pump to function properly.

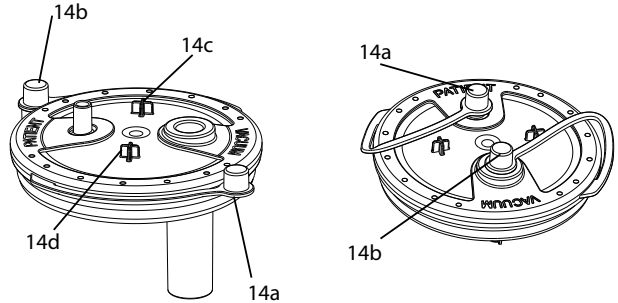
**3.3.** For more comfortable aspiration, set the desired vacuum value (bar) using the vacuum regulator (4). Turning the knob clockwise gives a higher vacuum value and turning it counterclockwise gives a lower vacuum value; these values can be read on the "vacuum gauge" (3).

**3.4.** Place your finger on the hole of the manual aspirated flow control (15) and, in pulsed mode, start aspiration on the patient through the cannula (NOT INCLUDED WITH THE DEVICE).

**3.5.** When the application is finished, switch off the appliance.

**3.6.** Disconnect the connecting pipes and if necessary close the 'PATIENT' and 'VACUUM' outlets with plugs 14a -14b. During aspiration, place caps 14a and 14b on fixing anchors 14d and 14c.

**3.7.** Empty and clean the vessel and connecting pipes.

**HYGIENIC PREPARATION**

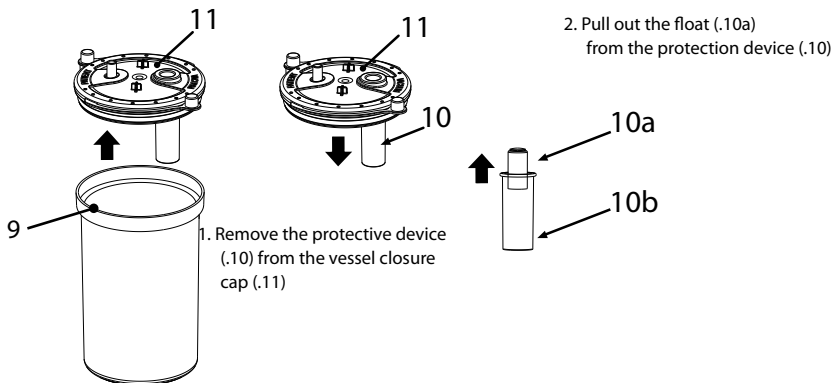
Switch off the appliance before each cleaning operation and unplug the mains cable from the socket.

**CLEANING THE APPLIANCE**

Use only a cloth moistened with antibacterial detergent (non-abrasive and free of solvents of any kind).

**COLLECTION VESSEL AND CONNECTION PIPES**

• Detach the cannula (16) (NOT INCLUDED WITH THE DEVICE), the manual aspirated flow control (15) and the tube (8) from the vessel (9). Detach the filter (7) from the vessel (9), disconnect the pipe (6) from both the filter (7) and the air inlet of the device (5), take the vessel from its stand in an upright position and empty it (at home in the toilet, in a hospital in the biological material container) and clean the vessel, disassembling it as shown below:

**Sanitisation:** Before and after each use, sanitise the collection vessel and connection pipes by choosing one of the methods as described below.

**(method A):** rinse individual components in hot drinking water (approx. 40°C) with mild dishwashing detergent (non-abrasive) or in a dishwasher on a hot cycle.

**(method B):** soak the individual components in a solution of 50% water and 50% white vinegar. Finally, rinse them thoroughly with warm (about 40°C) drinking water.

**(method C):** boil the individual components in water for 10 minutes; demineralised or distilled water is preferable to avoid lime deposits.

After sanitising the accessories, shake them vigorously and place them on a paper towel, or alternatively dry them with a hot air jet (e.g. hair dryer).

**Disinfection:** The disinfection procedure described in this section is to be carried out before using the accessories and is only effective on the components to be treated if every point is followed and only if the components to be treated are sanitised beforehand.

The disinfectant must be of the electrolytic chloride type (active ingredient: sodium hypochlorite), specific for disinfection and available in all pharmacies.

**Procedure:** - Fill a container of a suitable size to hold all the individual components to be disinfected with a solution of drinking water and disinfectant, observing the proportions indicated on the packaging of the disinfectant. - Completely immerse each individual component in the solution, taking care to avoid the formation of air bubbles in contact with the components. Leave the components immersed for the period of time indicated on the disinfectant packaging, and associated with the concentration chosen for the preparation of the solution. - Recover the disinfected components and rinse them thoroughly with lukewarm drinking water.

- After disinfecting the accessories, shake them vigorously and place them on a paper towel, or alternatively dry them with a hot air jet (e.g. hair dryer). - Dispose of the solution according to the disinfectant manufacturer's instructions.

**Sterilisation:** The sterilisation procedure described in this section is only effective on treated components if it is adhered to in all points and only if the components to be treated are sanitised beforehand, and is validated in accordance with ISO 17665-1.

Equipment: Steam steriliser with fractionated vacuum and overpressure compliant with EN 13060.

**Procedure: Package each individual component to be treated in sterile barrier system or packaging in accordance with EN 11607. Insert the packed components into the steam steriliser, making sure to keep the vessel (9) upright.**

**Perform the sterilisation cycle in accordance with the instructions for use of the equipment by selecting a temperature of:**

**(method A):** 134°C and a time of 10 minutes first. **(method B):** 121°C and a time of 20 minutes first.

**Storage:** After sanitising, disinfection or sterilisation, reassemble the vessel and connecting pipes according to the instructions given in the 'CONNECTION DIAGRAM'.

**At the end of each use, store the medical device complete with accessories in a dry and dust-free place.**

- The manual aspirated flow control (15) is a sterile disposable product and must be replaced each time it is used.
- The ANTIBACTERIAL HYDROPHOBIC FILTER (7) is a single-use product and must be replaced each time it is used.

#### ACCESSORY COMPARTMENT

The accessory compartment (17) can be easily removed for more thorough cleaning. See the disassembly sequence on page 1 in drawings 8-9-10-11.

#### VESSEL HOLDER

For ease of use, the vessel holder can be disassembled and reassembled on the side (RIGHT or LEFT) considered most suitable. With the use of a tool (draw.12) disengage the vessel holder from the bottom of the appliance (draw.13) and rotate it so that it disengages (draw.14) Re-engage it, proceeding with the reverse operation, on the chosen side. Make sure that the vessel holder is properly hooked to the bottom of the appliance. This system allows up to two vessel holders to be connected. (draw.15).

Table of planned methods / patient accessories						
	6	8	9	10a	10b	11
<b>HYGIENIC PREPARATION IN THE HOME</b>						
<b>Sanitisation</b>						
method A	✓	✓	✓	✓	✓	✓
method B	✓	✓	✓	✓	✓	✓
method C	✓	✓	✓	✓	✓	✓
<b>Disinfection</b>						
	300	300	300	300	300	300
<b>HYGIENIC PREPARATION IN A CLINIC OR HOSPITAL</b>						
<b>Sterilisation</b>						
method A	50	50	\	\	\	\
method B	\	\	50	50	50	50
✓: planned \: not planned 300:✓ MAX 300 TIMES 50:✓ MAX 50 TIMES						

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© 2024 FLAEM NUOVA®
**FLAEM NUOVA S.p.A. non risponde di ulteriori estensioni del periodo di garanzia assicurate da terzi.**
FLAEM NUOVA S.p.A.