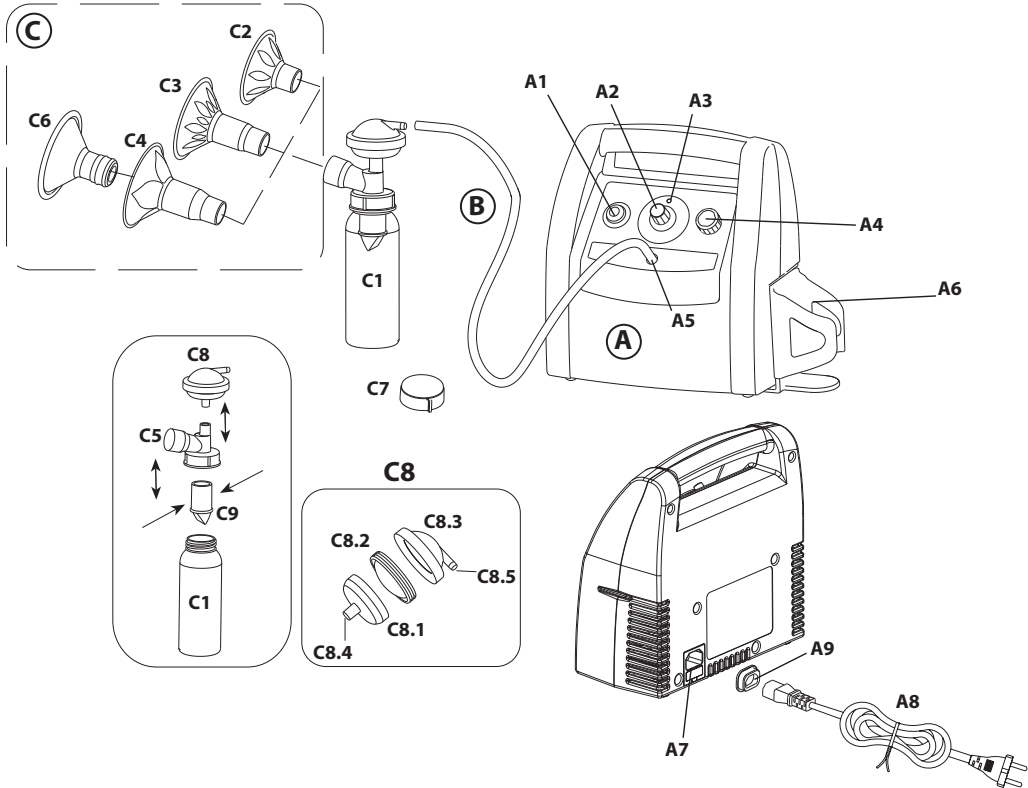
Latty

50

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Latty

Mod. P0117EM

We are pleased you have purchased our product and we thank you for your trust in us. Please read these instructions carefully and retain them for future reference. Only use the device as described in this manual. Please note that the full range of Flaem products can be viewed on the website www.flaem.it

Intended use and indications

The Latty electrical breast pump is a medical device intended for the expression of breast milk at home or in the hospital; it is recommended for relief from symptoms of breast congestion during breastfeeding, and relief from fissures or cracked and painful nipples. In case of mastitis, breast pumps contribute to the healing process by extracting the breast milk from the inflamed breast.

In addition, it allows mothers to offer newborns their own breast milk, also in case this is physically impossible to do directly for various reasons that include but are not limited to premature birth, cleft palate, or attachment problems.

Target users

Women, young teenagers or adults, during breastfeeding.

Breast milk contains all the nutrients that are necessary to meet the nutritional needs of newborns in the first months of their lives: using the breast pump allows breast milk to be expressed and stored if there is a surplus of milk or if the mother needs to be away from the newborn for any reason, for example if she is going back to work.

Contraindications

There are no known contraindications to the use of the Latty breast pump.

Product description

The Latty breast pump is fitted with two regulators that allow the suction frequency and intensity to be changed. Frequency adjustment allows the suction intervals to be adjusted from stimulation mode to the actual suction mode. In stimulation mode, the interval between suction is very brief, the suction force is minimal and a low quantity of milk is expressed: this stage has the purpose of stimulating milk production by the breast glands and simulates the fast suction of a newborn at the start of a feed. In suction mode, on the other hand, the sucking frequency decreases, while the force with which the milk is expressed increases.

Device equipment

A - Main unit

- A1 - Switch
- A2 - Frequency adjustment knob
- A3 - Suction indication LED
- A4 - Intensity adjustment knob
- A5 - Site of connecting tube plug
- A6 - Bottle holder
- A7 - Power supply socket
- A8 - Power cable
- A9 - Protective gasket

B - Suction connection tube

C - Accessories

- C1 - Milk bottle 250 ml
- C2 - Shield, Small
- C3 - Shield, Medium
- C4 - Shield, Large
- C5 - Shield cap
- C6 - Silicone adaptor for Large Shield
- C7 - Airtight cap for storage
- C8 - Diaphragm system
 - C8.1 - Lower diaphragm
 - C8.2 - Diaphragm membrane
 - C8.3 - Upper diaphragm
 - C8.4 - Diaphragm plug
 - C8.5 - Diaphragm tube connection
- C9 - Expression valve

Optional accessories

- Full Twins kit (code ACO0021)
- Bag for transport (cod. 15663A0)

For other accessories/spare parts contact the retailer

Symbols

The warning symbol identifies all instructions that are important for safety. Failure to comply with these instructions may cause harm to persons or damage the breast pump.

Warning: check the instructions for use before using the medical device.

Class II device

production batch

Device serial number

The device must not be disposed of in unsorted household waste (EU only).

Manufacturer

Type BF applied part

Alternating current

"ON"

When the device is switched off, the on/off switch stops compressor operation in only one phase of the two-phase power supply.

"OFF"

Medical CE Marking ref. Dir. 93/42 EEC as amended

Warning!

In compliance with: European Standard EN 10993-1 "Biological Evaluation of medical devices" and the European Directive 93/42/EEC "Medical Devices". Phthalate-free. In compliance with: Reg. (EC) no. 1907/2006

Minimum and maximum room temperature

Minimum and maximum air moisture

Minimum and maximum atmospheric pressure

Contains products intended to come into contact with food, pursuant to Regulation 1935/2004.

IP21

Protection rating of the casing: IP21. (Protected against solid foreign objects larger than 12mm. Protected against access with a finger; Protected against vertically dripping water.)

⚠ IMPORTANT WARNINGS

The manufacturer makes every effort to ensure that every product is of the highest quality and safety; however, as for any electrical device, basic safety rules must always be followed in order to prevent harming persons and damaging property.

- Carefully follow the instructions in this manual to assure proper operation and to extend the service life of the device.
- Do not expose the device and its components to direct sunlight, rain and weathering.
- Before using the product for the first time, and periodically during its service life, check the integrity of the device structure and of the power cable to make sure there is no damage. In the event of damage, do not plug in the cable and immediately take the product to an authorised service centre or to your trusted dealer.
- The supplied cable is fitted with a protective gasket (A9) to prevent the penetration of liquids into the device. Do not separate this gasket from the cable.
- Before each use, check that the diaphragm membrane (C8.2) is intact: it must never be torn or cut. In case the membrane is torn or cut, do not use the product.
- Should your device fail to perform as expected, contact the authorised service centre for clarifications.
- In the presence of children and non self-sufficient persons, the device must be used under the close supervision of an adult who has read this manual.
- Before use, remove and dispose of any plastic bags and all items that are part of the product's packaging and keep them out of the reach of children; they may be dangerous.
- Some parts of the device are small enough to be swallowed by children; therefore, keep the device out of the reach of children.
- Keep the device and its components out of the reach of children. If left within the reach of children, the power cable and the expression connection tube may pose a strangulation hazard. If swallowed, the small parts contained in the product may cause children to suffocate.

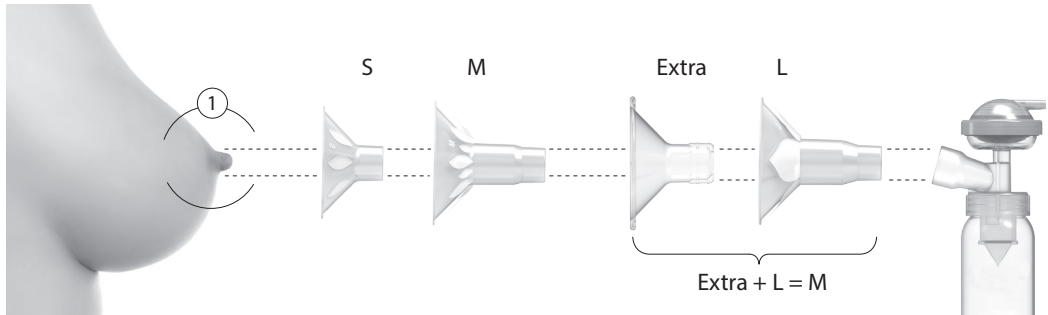
- Only place the device on flat and stable surfaces, away from cots and beds.
- Before connecting the power cable to the mains socket, ensure that the rating plate specifications regarding the power supply and the supplied power cable plug are compatible with your power mains and the corresponding socket. The rating plate of reference is located on the back of the product.
- Only use the supplied power cable.
- Connect the plug to a socket that is easily accessible by an adult only.
- To disconnect the device from the mains, unplug the power cable from the socket and only then disconnect the cable from the main body of the device. Do not pull the cable or the device, if connected, to unplug from the socket.
- Always keep the power cable away from hot surfaces.
- Keep the power cable away from animals (for example, rodents) that might damage the insulation.
- The supplied cable is fitted with a protective gasket to prevent the penetration of liquids into the device. Do not separate this gasket from the cable.
- Do not wash the device under running water or by immersion and protect it from water sprays or other liquids.
- The device is not suitable for use in the presence of a flammable anaesthetic mixture with air, oxygen or nitrous oxide; we recommend, therefore, not to use the device in premises where there are patients receiving home oxygen therapy, also at home.
- Do not use the breast pump sets for purposes other than the expression of breast milk.
- Do not handle the device with wet hands. Do not use the device in damp environments (for example, while taking a bath or shower). Do not immerse the device in water; in the event of immersion immediately disconnect the plug. Do not remove or touch the device immersed in water; unplug the power cable first. Immediately take the device to an authorised FLAEM service centre or to your trusted dealer.
- During use, ensure the kit is at such a distance from the main body to prevent it from falling.
- Never leave the product unattended while it is connected to a socket.
- Do not obstruct or introduce objects in the filter and its housing in the device.
- Never obstruct the air vents located on both sides of the device.
- Make sure there is no material obstructing the air vents before each use.
- Do not put any objects in the air vents.
- The device is fitted with an easy to inspect safety fuse in case of fault. Before doing so, unplug the power cable.
- Repairs, including the replacement of the supply cord, are to be carried out by FLAEM authorised personnel only, complying with the information provided by the manufacturer. Unauthorised repairs void the warranty and may be hazardous for the user.
- The average operating life of the compressor is 700 hours approximately.
- **WARNING:** Do not modify this device without authorisation from the manufacturer.
- The Manufacturer, the Dealer and the Importer shall be liable for safety, reliability and performance only if: a) the device is used in compliance with the instructions for use b) the wiring of the premises where the device is used is in compliance with safety regulations and current laws.
- The manufacturer must be contacted about any problems and/or unexpected events concerning operation and for any clarifications on use, maintenance or cleaning.
- Interactions: The materials used in the medical device are biocompatible in accordance with the provisions of Directive 93/42 EC as amended. However, the possibility of occurrence of allergic reactions cannot be entirely ruled out.
- The amount of time required to switch from storage to operating conditions is approximately 2 hours.
- The suction kits are items for personal use and intended to be used by one person only. Mothers should not use other mothers' devices.
- Do not perform expression for more than two consecutive sessions if no results are obtained
- In case of problems or pain, ask a breastfeeding specialist for advice
- Never use the device during pregnancy unless on prescription/recommendation of your doctor, since the expression of milk may induce labour.
- For mothers suffering from hepatitis B, hepatitis C or human immunodeficiency virus (HIV): the expression of breast milk does not reduce or eliminate the risk of transmission of the virus to the baby via the breast milk
- Never use devices, parts or components that are visibly dirty, contaminated by mould or other.
- Replace mouldy, damaged or worn components.
- Plastic bottles and caps become fragile if frozen and might break if they are dropped.
- Do not use the breast milk if the bottles or components are damaged.

PREPARING THE BREAST PUMP

Before using the **Latty** breast pump for the first time, thoroughly wash all the components of the kit, as described in the paragraph **“Cleaning and maintenance”**.

Then assemble the breast pump as follows:

1. Connect the expression valve (C9) to the internal duct of the shield cap (C5), pushing it against the sides all the way, so that it touches the base of the cap. Ensure that the two flaps of the valve are joined. In case they are crooked or there is an obvious empty space, repeat the operation.
2. Screw the shield cap (C5) on the milk bottle (C1).
3. Assemble the diaphragm unit as follows:
 - a. Insert the diaphragm membrane (C8.2) in the lower diaphragm (C8.1), so that the socket of the outer rim of the membrane completely covers the edge of the lower diaphragm.
 - b. Connect the upper diaphragm (C8.3) to the assembled components, pushing it all the way so that it seals the outside of the diaphragm membrane (C8.2).
4. Connect the assembled diaphragm system (14) by inserting the diaphragm fitting (C8.4) in the upper duct of the shield cap (C5).
5. Insert the shield of the desired diameter (C2, C3 or C4) in the suitable housing of the shield cap (C5), as shown in the picture.
6. Connect the expression connection tube (B) to the tube fitting housing (A5) on the main body (A) and to the tube connection on the diaphragm (C8.5).
7. Only for size M: a soft silicone adapter is available that offers more comfort (only for size M, 27 mm). In that case, using the 30 mm diameter shield (C4), apply the silicone adapter (C6) to the shield, inserting the internal cone all the way and fitting the curved border on the edge of the shield, making sure that it fits around the entire perimeter.
8. Insert the assembled kit in the handy bottle holder (A6) while performing the following operations.
9. Insert the power cable (A8) in the suitable socket on the back of the product (A7).
10. Insert the plug of the power cable (A8) in a mains socket that is easily reachable by an adult.
11. Place the main body (A) on a flat and stable surface, at such a distance that the connection tube (B) is not pulled during use.

Choosing a breast shield

Answer the following questions to find out if the available size of breast shield is the right one.

- a. Does the nipple move freely in the shield's duct?
- b. Is the areola tissue sucked into the shield's duct a little or not at all?
- c. Do you notice a rhythmic and gentle movement of the breast at each pump cycle?
- d. Do you feel the breast emptying completely?
- e. Is the nipple free of pain?

If the answer to any one of these questions is “No”, or in case of problems or pain during expression, ask the midwife or the lactation consultant for advice.

To know more

USE OF THE BREAST PUMP

Wash hands thoroughly with soap and water before touching the breast pump kit(s) and the breast. Avoid touching the inside of the containers or caps. Wash the breast with water or with suitable wipes.

After assembling the breast pump as indicated in paragraph "Preparing the breast pump", sit in a comfortable and relaxed position. Apply the shield tightly on the breast, ensuring the nipple is centred in the shield's inner cavity.

To know more

1. Unless you have used the device before and have applied personal settings, we recommend using the frequency adjustment knob (A2) setting it on "stimulation" mode. We also recommend using the intensity adjustment knob (A4) and turning it clockwise (minimum intensity).
2. Switch on the breast pump by using the switch (A1) located on the left-hand side of the control panel of the main body, setting it to "I": the Latty breast pump will start.
3. Gently turn the intensity adjustment knob (A4) anti-clockwise until the effects of suction on the breast and nipple are noticeable but not painful.

Normally, we recommend the stimulation stage to last at least 2 minutes: during this time, you can use the frequency adjustment knob (A2) to change the stimulation rhythm from 120 to 80 suction cycles per minute.

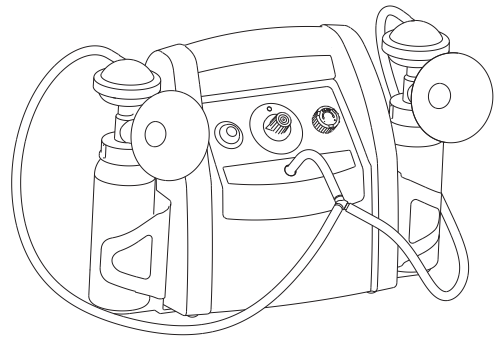
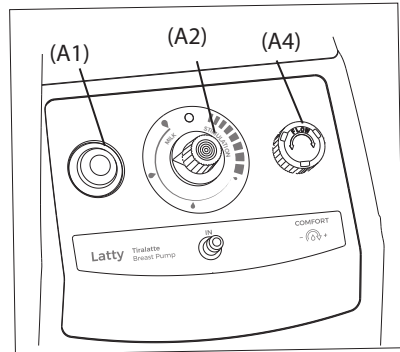
The setting of the stimulation frequency may vary from woman to woman, therefore, the user is advised to try various frequency intervals to find the one that is best for their breasts.

4. After the stimulation stage, or when the milk starts being expressed, further turn the frequency adjustment knob (A2) clockwise to switch to suction mode: the frequency of suction cycles decreases further and the kit's suction power increases. At this point, after setting the most comfortable frequency or the one that results in the best expression (visible through the shield that has been applied to the breast), you can further use the intensity adjustment knob (A4) so that the applied suction results in proper expression of milk.

Warning: In the event the intensity level set should cause discomfort or pain, turn the intensity adjustment knob (A4) clockwise, thus reducing the force applied to the nipple/breast.

It is possible to alternate milk expression between the breasts or, alternatively, milk can be expressed from both breasts by using the double expression kit, which may be purchased separately. (Full Twins kit code ACO0021).

5. At the end of expression, switch the breast pump off. Set the switch (A1) to "0".
6. Immediately after each use, thoroughly clean all the kit's components, as described in the paragraph "Cleaning and maintenance".

- ⚠️ Pushing the nipple shield too hard might affect the flow of milk.
- Do not express milk with too high and uncomfortable vacuum (painful). The pain, which may be a symptom of breast and nipple trauma, might decrease the milk flow.
- Do not try to remove the breast shield during expression. Switch the breast pump off and use a finger to detach the shield from the breast; then remove the breast shield.
- If expression is not comfortable or painful, switch the breast pump off, use a finger to detach the shield from the breast and, lastly, remove the breast shield.
- Do not shorten the tube between the breast shield and the breast pump.
- Do not use the Latty breast pump sets with an incompatible breast pump.
- Do not use the Latty breast pump with incompatible breast pump sets.

Storing the milk

It is possible to store the expressed milk in the fridge or in the freezer by closing the bottle (B1) with the supplied airtight cap (C5): the expressed milk can be stored in the fridge for a maximum of 48 hours at 4°C, or for up to 3 months, if stored in the freezer at -18°C.

- In order to prevent the proliferation of bacteria, we recommend feeding the expressed milk immediately after expression, or storing it immediately, opting for the most suitable solution.
- It is not recommended to defrost or heat up breast milk in the microwave oven or in a pan with boiling water. The high temperatures may cause serious burns in the baby's mouth. In addition, the composition of breast milk might be altered.

USEFUL ADVICE

To further stimulate the production of milk, we recommend taking a warm shower or applying a warm cloth on the breast just before expression.

To avoid liquid overflow, we recommend never filling the milk bottle above the measuring scale on the bottle.

CLEANING SANITISATION DISINFECTING

Switch off the device before any cleaning procedure and unplug the power cable from the socket.

- ⚠ Do not perform cleaning or sanitising operations on the breast pump when it is switched on; cleaning and sanitising operations must be performed in accordance with these instructions for use.
- Do not use abrasive liquid soap/dishwasher detergent to clean the breast pump, its components or the parts of the breast pump set.
- Only use drinking or bottled water for cleaning.
- Do not use tools to clean the expression valve (C9), as it might be damaged and result in the breast pump being unusable.
- Do not use abrasive products to clean the breast pump and its components.

DEVICE AND TUBE EXTERIOR

Only use a cloth moistened with antibacterial soap (non-abrasive and with no solvents of any sort).

ACCESSORIES

Separate all the components of the breast pump. Remove the expression valve (C9) from the duct inside the shield cap (C5) by using the sides and not the central closing flaps.

Note: to remove the silicone tube (B), first disconnect the diaphragm system (C8) from the shield cap (C5) to avoid overturning. Then grip it from the end that is attached to the upper diaphragm (C8.3), pulling gently.

SANITISATION

Before and after each use, sanitise the accessories by choosing one of the methods described below.

method A: Sanitise accessories **C1-C.2-C.3-C.4-C5-C6-C7-C8-C8.1-C8.2-C8.3-C9** under warm drinking water (approximately 40°C) with a gentle, non abrasive dish detergent.

method B: Sanitise accessories **C1-C.2-C.3-C.4-C5-C7-C8-C8.1-C8.2-C8.3** in the dishwasher with hot cycle. This, more aggressive, type of washing might discolour or, over time, cause the material to become opaque or decay.

method C: Sanitise accessories **C1-C.2-C.3-C.4-C5-C6-C7-C8-C8.1-C8.2-C8.3-C9** by immersing them in a solution of 50% water and 50% white vinegar, then rinse thoroughly under warm drinking water (approximately 40°C).

If you also wish to perform cleaning by DISINFECTATION, skip to the DISINFECTATION paragraph.

After sanitising the accessories, shake them well and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example hair dryer).

DISINFECTATION

After sanitising the accessories, disinfect them choosing one of the methods described below.

method A: Accessories **C1-C.2-C.3-C.4-C5-C6-C7-C8-C8.1-C8.2-C8.3-C9** may be disinfected.

The disinfectant must be an electrolytic chlorine oxidiser (active principle: sodium hypochlorite) specific for disinfecting, which is available in any chemist's shop.

Directions:

- Fill a container of suitable size to hold all the parts to be disinfected with a solution of drinking water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant and at the concentration chosen for the solution.
- Remove the disinfected parts and rinse with plenty of warm drinking water.
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

method B: Sanitise accessories **C1-C.2-C.3-C.4-C5-C6-C7-C8-C8.1-C8.2-C8.3-C9** by boiling them in water for 10 minutes; use demineralised or distilled water to prevent limescale deposits.

method C: Sanitise accessories **C1.1-C1.2-C1.3-C1.4-C2-C3-C4** with a hot steam steriliser for baby-bottles (not the microwave type). Perform the process thoroughly following the instructions of the steriliser. To ensure the disinfection is effective, choose a steriliser with an operating cycle of at least 6 minutes.

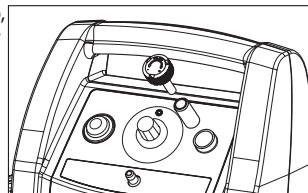
After disinfecting the accessories, shake them well and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example a hair dryer).

At the end of each use store the device with its accessories in a dry place away from dust. Do not store wet or humid components, as this might lead to the formation of mould.

AIR FILTERING

The device is fitted with a suction filter, located inside the pressure adjustment knob, which must be replaced when it is dirty or discoloured. To replace it, lift the intensity adjustment Knob (A4) and extract the Filter.

Do not wash or reuse the same Filter. Regular Filter replacement is necessary to assure proper compressor performance. The Filter must be regularly checked. Contact your dealer or authorised service centre for replacement filters.

Note: Only use original accessories and spare parts by Flaem; we disclaim any liability in the event of using non original spare parts or accessories.

TROUBLESHOOTING

Switch off the device before any procedure and unplug the power cable from the socket.

PROBLEM	SOLUTION
The motor does not work	<p>Check the mains power.</p> <p>Check that the plug of the power cable is well inserted in the mains socket and in the socket on the back of the product.</p> <p>Check that the device is switched on.</p>
Suction is faint or imperceptible	<p>Disassemble and assemble the breast pump set again, following the instructions.</p> <p>Check that all connections of the breast pump set and of the suction tube are securely fastened.</p> <p>Use the device in accordance with the instructions manual.</p> <p>Ensure the breast shield fully adheres to the breast.</p> <p>Ensure the flaps of the expression valve are closed.</p> <p>Check that the valve and the membrane are clean and undamaged.</p> <p>The ends of the tube must be securely fastened to the connector on the main body and to the opening in the membrane's cover.</p> <p>The diaphragm unit must be correctly assembled, as described in the instructions.</p> <p>Do not twist or block the tube during expression.</p> <p>All components must be completely dry.</p>
Blocked float	Decrease the suction intensity level
If there is leakage	<p>Place the breast pump set in the appropriate bottle holder.</p> <p>Switch off the device and disconnect the power cable from the socket.</p> <p>Use a moist cloth (not wet) to clean the breast pump and the membrane area.</p> <p>If there are visible residues, condensation or milk in the tube: clean all the components of the breast pump set as described in the chapter "Cleaning and maintenance".</p> <p>When the components are completely dry, assemble the breast pump kit once more.</p>

If, after checking the above-described conditions the device is still not working properly, we recommend taking it to your trusted dealer or nearest authorised FLAEM service centre.

TECHNICAL SPECIFICATIONS

Mod. P0117EM

Power supply:	230V ~ 50Hz 140VA
Maximum vacuum in the pump in expression mode:	-0.50 bar approx
Noise (at 1 m):	< 60 dB (A) approx.
Operation:	Continuous

Operating conditions:

Temperature:	min 5°C; max 40°C
Air humidity:	min 10%; max 95%
Atmospheric pressure:	min 69KPa; max 106KPa

Storage conditions:

Temperature:	min -25°C; max 70°C
Air humidity:	min 10%; max 95%
Atmospheric pressure:	min 69KPa; max 106KPa

Safety approvals:

Dimensions: 26(W) x 12(D) x 23.5(H) cm

Weight: 2.100 Kg

APPLIED PARTS

Type BF applied parts are: patient accessories (C2, C3, C4)

ELECTROMAGNETIC COMPATIBILITY

This device was designed to meet the current requirements for electromagnetic compatibility (EN 60 601-1-2). Electro-medical devices require particular care during installation and use relative to EMC requirements. Users are therefore required to install and/or use these devices following the manufacturer's specifications. There is a risk of potential electromagnetic interference with other devices, in particular with other test and treatment devices. RF mobile or portable radio and telecommunications devices (mobile telephones or wireless connections) might interfere with the operation of electro-medical devices. For additional information please go to www.flaem.it. The Medical Device may be subject to electromagnetic interference if there are other devices used for specific diagnosis or treatments. Flaem reserves the right to make technical and functional modifications to the product with no prior warning.

DEVICE DISPOSAL

In compliance with Directive 2012/19/EC, the symbol printed on the device shows that the device to be disposed of is considered waste and must therefore be "collected separately". Consequently, the user must take said waste (or have it taken) to the designated collection facilities arranged by the local authorities, or hand it to the retailer when purchasing an equivalent new device. Separate waste collection and the subsequent treatment, recycling and disposal procedures promote the production of devices made with recycled materials and limit the negative effects on the environment and on health caused by any waste mismanagement. Illegal disposal of the product by the user entails administrative penalties according to the transposition laws of directive 2012/19/EC of the member state or country where the product is disposed of.

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© Copyright by FLAEM NUOVA S.p.A., 2019 cod.18440-0 rev.09/2019
FLAEM NUOVA S.p.A. non risponde di ulteriori estensioni del periodo di garanzia assicurate da terzi.