TERMOFORO HEATING PAD

TRF20

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
1. Product Model: TRF20

2. Instruction for use & General device function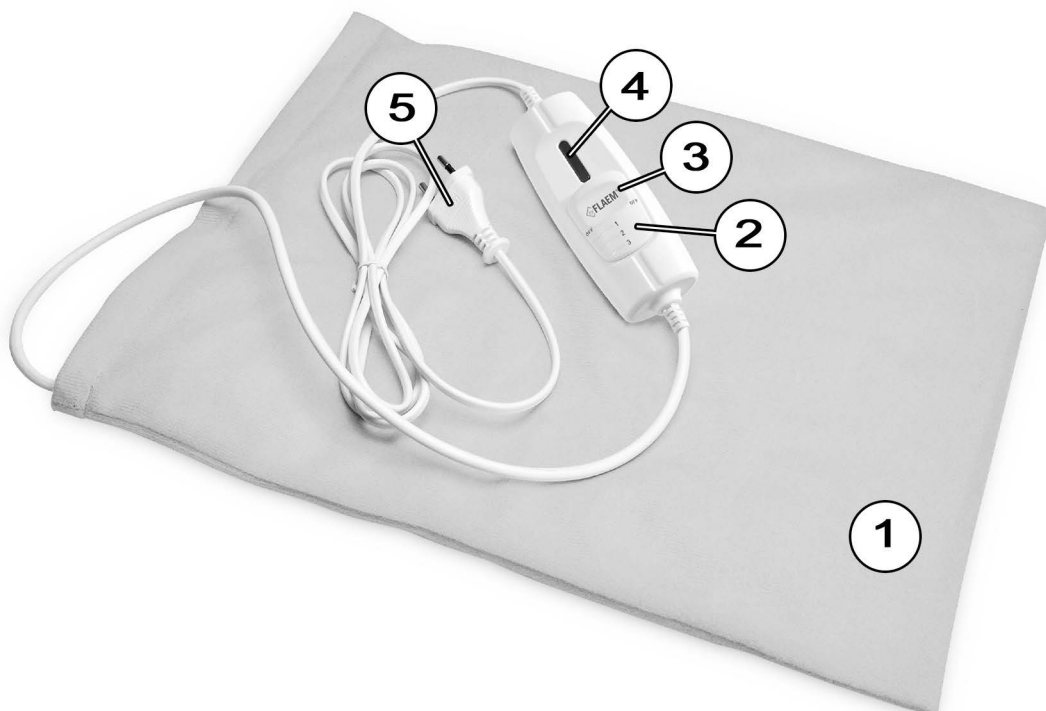

- 1) Removable Cotton Cover with bottoms
- 2) Three Temperature Settings: Low(1), medium (2) and high (3)
- 3) Heat Selection Switch
- 4) Led Indicator
- 5) Power Plug


Place the heating pad in the fabric cover and adjust it to the part of the body you want.

Place the plug of the heating pad into the socket.

Make sure that the voltage which is stated on the device is compatible with AC voltage.

Activate the heating pad by moving the switch from closing position (OFF) to the desired heating gradation from 1 up to 3 (low 1, Medium 2 and High 3).

The product has an advanced technology sensor thermostat to regulate automatic and to remain stable the preselected temperature levels

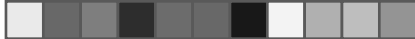The heating pad has an automatic deactivation system. After 60 minutes operation the device will turn off by itself. In case you wish to reactivate the use of the device just move the switch from the position which it was before the automatic deactivation in closing position (OFF). Then move the switch

gradation by selecting the desired temperature (1 or 2 or 3).

After completing the usage the power switch should be placed in closing position (OFF) in order to switch off the device.

Disconnect the plug of the heating pad from the socket when it is in not in use.

3. Important Warnings & Safety Information

Inspection of the heating pad

- Before of each use, the heating pad should be checked for any damages. In case of any damages will appear discard the heating pad or contact the authorized service for more information.

If your heating pad doesn't function (is not warming) please check the following:

- Make sure that plug of the heating pad is applied correctly and steadily in the socket.
- Make sure that electric wire is not cut or bend and applied correctly to the heating pad and to the plug.

The service or / and the repairing of the heating pad is only allowed by Qualified and specialist personnel of the authorization service, Any interference in the mechanism, control or wire of the heating pad will cancel the provided warranty.

Important Safety Instructions

- Do not sit or lay on the heating pad and don't bend the wire of the heating pad.
- Do not use the heating pad while sleeping.
- When the heating pad is used for an extended period, Heat Selection Switch should be adjusted to temperature setting Low (1).
- Do not ever puncture the heating pad with any sharp object.
- Do not ever try to fix or repair the heating pad.
- For any maintenance effort of the heating pad you should first disconnect the plug from the socket.
- Do not use the heating pad on persons insensitive to heat and other very vulnerable persons who are unable to react to overheating.
- Children under the age of three are not to use this appliance due to their inability to react to overheating.
- this appliance should not be used by persons who have pacemaker.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

Not to be used by very young children (0-3 years).

- The appliance is not to be used by young children over the age of three unless the Heat Selection Switch have been pre-set by a parent or guardian, or unless the child has been adequately instructed on how to operate Heat Selection Switch safely.

- The appliance must be used solely with the types of control indicated on the label.

- If the power cord is damaged, it must be replaced by an authorised technical service centre, so as to prevent all risks.

- In order to use together with other medical devices it is recommended to take the opinion of the treating physician.
- If you have poor blood circulation or you are a diabetic you should consult your physician before using it.
- Do not place the heating pad on parts of the body with inflammation or particularly sensitive.
- The excessive use of heating may cause irritation or burns.
- Do not use the heating pad together with liquids and ointment containing ingredients that produces heat.
- Do not use the heating pad without the removable cotton cover.
- Do not use the heating pad in the environment rich in pure oxygen or near equipment that provide or store oxygen.
- The device should not be exposed to extreme temperatures, to the sun or rain.
- The user should not touch the device with wet hands.
- Do not use if wet.

NOTE:

The heating pad can not be washed or chemically cleaned or wring out

4. Useful Advice


Care instruction of the heating pad

To clean the heating pad you must remove the plug from the socket. After removing the cover, using a slightly wet cloth or sponge clean possible stains or generally the heating pad. Do not use in any case cleaning material with solvent contents. In order to use again the heating pad make sure that it doesn't have the slightest moisture and it is completely dry.

Washing and Care Instruction for the fabric cover.

Remove cotton fabric cover from the heating pad.

Wash the cover by hand with mild soap in lukewarm water (40°C) Drip dry to keep it fluffy. Do not machine Wash or Dry. If cover becomes discolored, or worn replace it with a new one available from your heating pad make sure that it doesn't have the slightest moisture and it is completely dry.

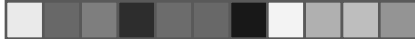Storage of the Heating pad

When not in use, store as follows:

When storing the appliance, allow it to cool down before folding;

Do not crease the appliance by placing items on top of it during storage;

Examine the appliance frequently for signs of wear or damage. If there are such signs or if the appliance has been misused, return it to the supplier prior to any further use;

When you don't use the device, it should be stored in a dry place.

This appliance is not intended for medical use in hospitals.

Disposal

The product must be disposed of appropriately, in compliance with European standard 2012/19 EC. The recyclable materials contained in the appliance must be recovered so as to prevent environmental pollution. . For further information contact the local disposal body or the appliance dealer..

5. Dimensions - Specifications


Mod.TRF20

Voltage	220-240V AC
Frequency	50/60HZ
Power	100W
Dimensions of Heating Pad	40 x 30 cm
Storage Conditions	10°- 40°C

The highest temperature which the heating pad will achieve is the highest limit according to the European Standards for the safety of the user.

6. Symbols Explanation

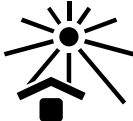	Read operator's manual		2006/95/EC Low Voltage Directive 2004/108/EC Directive EMC
	Do not use if folded or crumpled		Do not dry clean
	Class II		Do not iron
	Do not insert needles or pins		Do not use chlorine bleach
	Hand wash only		Keep dry
	Do not wash		Keep away from heat
	Not to be used by very young children (0-3 years).		

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GB - The warranty terms here specified are valid only in Italy for Italian residents. In all other countries, the warranty will be provided by the local dealer that sold you the unit, in accordance with the applicable laws.

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Flaem Nuova S.p.A