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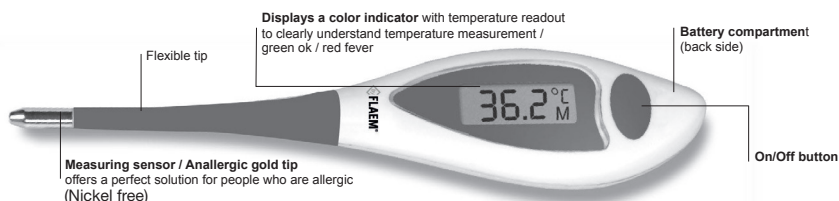
MT19U1

Important safety instructions

- The thermometer is suitable exclusively for measurement of the body temperature!
 - The minimum measurement time until the signalling tone [beep] must be maintained without exception!
 - Ensure that children and disabled persons do not use the instrument unsupervised; some parts are small enough to be swallowed.
 - Protect the thermometer from hitting and striking!
 - Avoid bending the tip more than 45°!
 - Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
 - Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
 - Do not intentionally submerge the thermometer in water or other liquids.
- In the event of splashes on the instrument, before taking a measurement dry with a clean cloth. Splashing water will not compromise the accuracy of the measurement.

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste. Illegal dumping of the product by the user entails the administrative sanctions stated in article 50 et seq of Legislative Decree n. 22/1997.

Important: check the operating instructions

Type BF applied part

Turning on the thermometer

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

Using the thermometer

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0,1°C in 16 seconds and that the thermometer is ready for reading. **Displays a color indicator** with temperature readout to clearly understand temperature measurement:

The LCD will display green color → ok

The LCD will display red color → fever

To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 3 minutes.

Storage of measured values

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

Technical Specifications

Type

Predictive thermometer

Measurement range

32.0 °C bis 43.9 °C Temp.

< 32.0 °C: display «L» for low (too low)

Temp. > 43.9 °C: display «H» for high (to high)

±0.1 °C between 34 °C and 42 °C

Measurement accuracy

Operating temperature

10 °C to 40 °C, humidity: 15% to 95% (noncondensing)

Self-test

Automatic internal check at a test value of 37 °C;

if there is a deviation of > 0.1 °C, «ERR» (error) is displayed

Display

Liquid crystal display (LCD) with three digits.

Smallest unit of display: 0.1 °C

Signalling tone

For signalling that the thermometer is ready to use and that the

temperature increase is less than 0.1 °C / 16 seconds

For storing the last measured value

temperature: -25 °C to +60 °C, humidity: 15% to 95% (noncondensing)

Memory

3.0V CR1632

Storage Environment

EN12470-3, clinical thermometers, ASTM E 1112

Battery

IEC 60601-1, IEC 60601-1-2

Reference to standards

EN12470-3, clinical thermometers, ASTM E 1112

This device complies with the requirements of the Medical Device Directive 93/42/EEC.

Technical alterations reserved.

Measuring methods

In the mouth (oral)

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breath evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

Approx. measuring time: 8 sec.!

In the anus (rectal)

This is the most reliable measuring method, and is especially suitable for infants and small children. Carefully insert the measuring sensor of the thermometer 2 to 3 cm into the anal aperture. **Approx. measuring time: 8 sec.!**

In the armpit (axillary)

A minimum measurement time of 3-5 minutes is recommended regardless of the beep sound. **Approx. measuring time: 8 sec.!**

At home, for measurement by parents, for all children measurement with electronic thermometer in the armpit is recommended. (Source: Italian Society of Pediatrics)

Cleaning and disinfecting

Use an alcohol swab or cotton tissue moistened with alcohol (70% Isopropyl) to clean the thermometer. Please never immerse into liquids!

Battery replacement

When the «▼» symbol (upside-down triangle) appears at the display field, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover from the thermometer.

Insert the new battery with the + at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store

WARRANTY: The warranty terms here specified are valid only in Italy for Italian residents. In all other countries, the warranty will be provided by GB the local dealer that sold you the unit, in accordance with the applicable laws.

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www.flaem.it

Microlife AG, Espenstrasse 139
9443 Widnau / Switzerland

Onbo Electronic (Shenzhen) Co., Ltd
No. 497, Ta Laneg Nan Road,
Ta Laneg Street, Baoan District,
Shenzhen China

CE 0044

Cod.16399 rev. 07/2014

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**FLAEM NUOVA S.p.A. - Via Colli Storici, 221