I Aspiratore Nasale per Bambini
GB Nasal Aspirator for Children
F Aspirateur nasal pour les enfants
NL Nasale Aspirator voor kinderen
D Nasensauger für Kinder
ES Aspirador nasal para niños
RU Носовой аспиратор для детей
PL Aspirator do nosa dla dzieci
KR 영유아용 콧물흡인기- 베이비 코푸리
GR Ρινική αναρρόφησης για τα παιδιά
TR Çocuklar için Burun Aspiratörü
CN 儿童鼻吸气
SA افطالل ۃخضم فن الـ

**PHTHALATES
& BPA FREE**

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GB NASAL ASPIRATOR FOR CHILDREN

device for aspiration of mucus from baby's nose

We are grateful for your purchase.

The **BABY NASAL ASPIRATOR Baby Flaem** is a patented device, practical, easy to use and hygienic. It actually only requires the simple action of one hand and does not require any mouth aspiration by mum as is the usual case with other aspirators found on the market, thus avoiding possible micro-biological contamination.

WHY SHOULD EXCESS MUCUS BE ASPIRATED?

Good breathing is the first defense which a baby has available to protect its wellbeing and curb infections. As a matter of fact, **the babies under 1 year old, breath exclusively via the nose**. So, if the nose is blocked, it is difficult for the baby to breathe, having problems sleeping and eating; it can also at times refuse to suckle and starts crying very often. Considering that **up to when it is 3 years old, the child is not able to blow its nose on its own**, mucus and entrapped bacteria, viruses and dust particles get clogged in the nasal cavities. There are two methods which are mostly advised to unblock the nose: **a good wash and the removal of mucus**; as for washing, we advise you to use the Flaem nasal rinses and for mucus removal, the baby nasal aspirator.

IMPORTANT WARNINGS

Read these instructions carefully and keep for future reference. Only use the device as described in this leaflet. It is important to read and understand the instructions for use. Contact your vendor or support centre for further clarification.

As for use by children and by persons who are not self-supporting, the device should be used under the strict supervision of an adult person who would have read this leaflet. Keep the device out of the reach of children.

NOMENCLATURE (ref. Fig. C)

1. Soft Ergonomically-shaped Tip (2 per pack)

2. Body with external Fins (2A) allowing an easy and safe grip, and Mucus collection chamber (2B)

3. Spring

4. Plunger

5. Sealing ring

CLEANING AND MAINTENANCE

Before each use, wash your hands thoroughly and clean the **Aspirator** as follows: open the **Nasal Aspirator** by turning the **Sealing Ring** (5) (Picture D), paying attention to the **Spring** (3) thrust.

Cleaning: Wash components with warm drinking water (approximately 40°) with non-abrasive mild dish detergent or directly in the dishwasher with hot cycle. The device can be treated with commonly marketed cold disinfectant, used according to their specific instructions.

Drying: After having washed or sanitized all components, shake them vigorously, and finally place them on a paper towel, or alternatively dry with a jet of hot air (for example using a hairdryer).

Assemble the device as shown on the relevant diagram (Picture C), after having lubricated with baby moisturizing oil the seal of the plunger (4).

INSTRUCTIONS FOR USE

Personal use is recommended, as well as frequent replacement of the nasal tip (1) to avoid any risk of infection.

a - Hold the Aspirator as if it were a syringe.

b - Place the Nasal tip (1) of the Aspirator inside the nostril of the child (Picture A).

c - Press with the thumb on the bottom of the Plunger (4) (Picture A); the air is expelled through the slots at the base of the device Body (2), thus avoiding the creation of a pressure towards the nasal orifice of the child. This phase can also be performed before placing the Aspirator at the nostril of the child.

d - Gradually release the Plunger (4) (Picture B), thus creating a vacuum that gently sucks the mucus from the nose of the child.

e - Repeat for the other nostril from point b.

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